
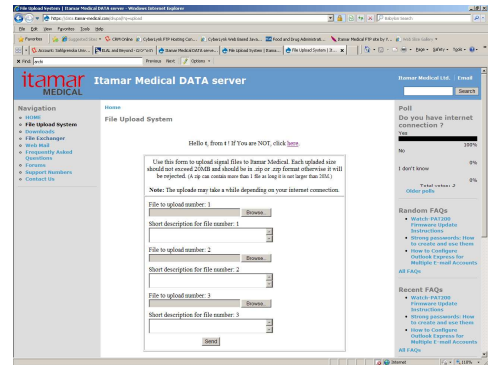


Data signal shipping

Sending data for evaluation to Itamar

- Select the S32 files you want to upload. These files may be reached by clicking the “ED2000 Data” icon located on your desktop.
- Right mouse click on the files, Select ‘Send to’ → ‘Compressed folder ‘ (only compressed files can be uploaded to the ftp server)
- Upload the compressed zip file to: <http://support.itamar-medical.com/upload>
- Fill in the required information
 - PI / Physician Name
 - Submitter Name
 - Site Name
 - Site Type
 - Country
 - City
 - Phone Number
 - E-mail
- Enter study code: (Supplied by Itamar Representative according to study code)
- Enter password: (itamarftp)
- Enter Submit
- Browse and enter the zip file
- Enter description for file
- Press the send button
- At the end of the upload process make sure to get a message: “List of successfully upload files” and the list of files you uploaded
- Note: If you have a laptop computer supplied by Itamar Medical, please don't connect it to the Internet as it is not protected by anti-virus software. You may burn the Data into a CD and then use it to send the Data through the protected computer

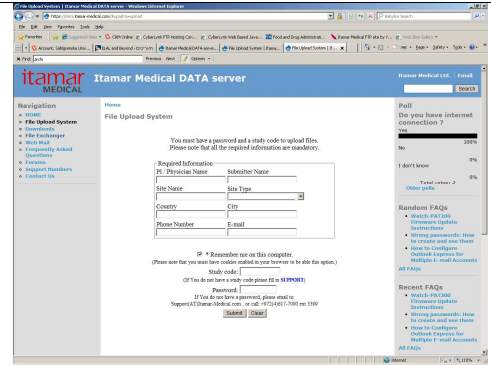

List of successfully uploaded files:

File Name (size)	File description