

WatchPAT™ 300 Initialization

On this page, you will find helpful resources that are necessary for the proper initialization and operation of "WatchPAT™ 300".

zzzPAT Users Only

Step 1: zzzPAT Installation

Full zzzPAT Installation

1. Download zzzPAT using the link below (Patch to upgrade from previous versions is available as well)
2. Extract the zip file and launch the "Setup" file
3. Follow On-screen instruction and complete installation

[Download zip file for full installation](#)

zzzPAT Patch Installation – If Applicable

1. Back-up all WatchPAT study files to a safe location
2. Close zzzPAT
3. Download zzzPAT patch using the link below
4. Run the patch file zzPATpatch.exe

[Download zip file for upgrade patch installation](#)



Itamar Medical Ltd.

9 Halamish St. P.O Box 3579, Caesarea 3088900, Israel | T: +972-4-617-7000 | F: +972-4-627-5598

US Toll Free: 1-800-206-6952 | www.itamar-medical.com

Step 2: USB Device Driver (FTDI) Installation

Please Note: You Must Install USB Device driver (FTDI) prior to connecting the device to the PC

1. Follow the WatchPAT 300 FTDI Driver installations instruction located [here](#)

Driver location:

C:\Program Files (x86)\Itamar medical\zzzPAT\Misc\CDM21228_Setup.exe

2. Run the driver installer and follow the downloaded instructions
3. The WatchPAT 300 is now ready to be used

Step 3: Download the new Operations Manual

Click the link below to view and download the new Operations manual

[Download WatchPAT 300 Operations Manual here – USA Version](#)

[Download WatchPAT 300 Operations Manual here – European Version](#)

[Download zzzPAT Operations Manual here](#)

