

Step 7

Apply the "Unified" sticker at the back of the device on top of the current sticker.

Important: This step is important to distinguish between upgraded devices and regular ones, as they require different probes.



Step 8

Connect the device to the new wrist strap.



Step 9

Connect the uPAT probe to the device.

Note: The device will no longer work with old PAT® probes - only with uPAT® probes.



Step 10 - Getting started

Note:

The WatchPAT™ Unified works with zzzPAT version 4.4.64 and above.

Make sure to upgrade the zzzPAT software to the latest version using the link given here.

- Perform a Device Test and verify that you receive a **'PASS'** result
- Update the zzzPAT software to the latest software here: www.itamar-medical.com/UnifiedUpgrade
- Download the Operation Manual here: www.itamar-medical.com/UnifiedUpgrade
- You can download the patient instructional video here: www.itamar-medical.com/UnifiedUpgrade

itamar
medical



WATCH OVER YOUR SLEEP

Congratulations! You have upgraded your WatchPAT™200 to the WatchPAT™ 200 Unified device.

*For any questions and/or support please contact our USA help desk at:
1-888-7ITAMAR*

itamar
medical

www.itamar-medical.com

 **Itamar Medical Ltd.**
9 Halamish St. POB 3579
Caesarea Ind. Park, 3088900, Israel
Tel +972 4 6177000
Fax +972 4 6275598

Itamar Medical Inc.
842 Upper Union St., Suite 2
Franklin
MA 02038, USA
Tel 1 888 748 2627

 **MEDES Ltd.**
5 Beaumont Gate, Shenley Hill,
Radlett, Hertfordshire WD7 7AR, England
Tel: +44 20 8123 8056
Tel/Fax: +44 19 2385 9810

REF OM2100300 Edition 3

Congratulations -
Easily upgrade to the
WatchPAT™200 Unified
Home Sleep Test Device

Upgrade Instructions

The upgrade kit that you received contains the following items:

Description	REF
Wrist strap	AS21515XX
uPAT cable	AS00605XX
uPAT probe	AS01734XX
Step-by-Step instructions	OM2193330
Quick reference cards	OM2193370
Plug for bracelet port	MP22724XX
"Unified" Sticker	IL2298510
Tamper-proof bracelet cable (optional)	AS00655XX
*WP200 firmware upgrade	step 6 - online
*zzzPAT software upgrade	step 10 - online

Please follow these instructions in order to upgrade your WatchPAT™ 200 device to the WatchPAT™ Unified device:

Step 1

Detach the WatchPAT™ 200 device from the current wrist strap.



Step 2

Remove the PAT® cable from the WatchPAT™ 200 device by unscrewing the small screw all the way and then pulling the cable gently.



Step 3

Attach the new uPAT cable to the device and secure it using the same small screw from the previous step.



Step 4

If you are not using the Tamper-Proof Bracelet:

- A) Insert the Plug for the Tamper-Proof Bracelet connector.
- B) Push the flat part (with the 3 lines) inside the gap just above the connector.
- C) Inset the plug into the connector and push it in. You can now expose the connector without detaching the plug when needed.



Step 5

Optional: If you are using the Tamper-Proof Bracelet: Connect the Bracelet cable.



Step 6

Upgrade the device's firmware (embedded software) by connecting the device to a PC with internet connection and following this link: www.itamar-medical.com/UnifiedUpgrade Follow the instructions on the website in order to upgrade the software.