

Instructions on Creating and Retrieving a WatchPAT One Study with zzzPAT




Launch zzzPAT program
Default Login and Password:
Login: 111
Password: 111

zzzPAT - Login

1. Create New Study...

- Select "File > New Study"
Or Select the Icon (Ctrl+N)

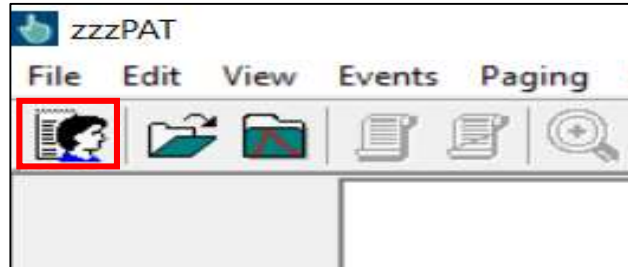

2. Fill out the mandatory information including:

- Patient ID
- Device SN (Can be found on the side of WatchPAT ONE box)




- Enter 4 number PIN
- Email for notification
- Complete other patient information
- After filling in all mandatory information, click "Register WP-ONE" to finalize the initiation

3. New Study Question...

You will receive a pop-up window that reads: "The new information has been successfully saved. Do you wish to prepare a new study?"



Select No and continue to next step.
WatchPAT One is ready for patient.

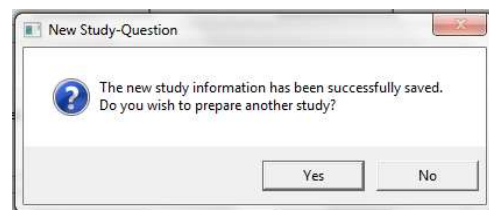


Retrieve Study Data from WatchPAT One into zzzPAT

Open zzzPAT application and press
File>Load Study and Analyze

or click on  from the main menu.

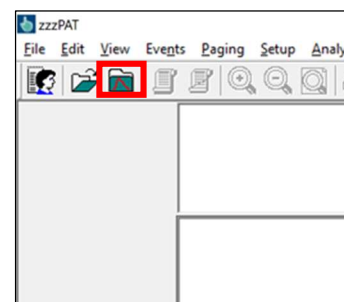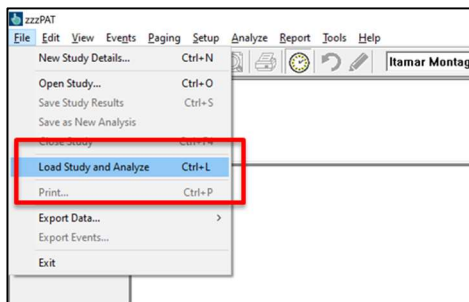

- A window will open with all registered patients whose studies were not yet retrieved from the web server.
- Select study to download and the recorded data will be loaded into the zzzPAT database for selected patient.
- The recorded data is automatically analyzed.
- The user can subsequently review, edit, add “Diagnosis and Recommendations” and produce a Sleep Report.
- Previously loaded studies can be opened and reviewed.

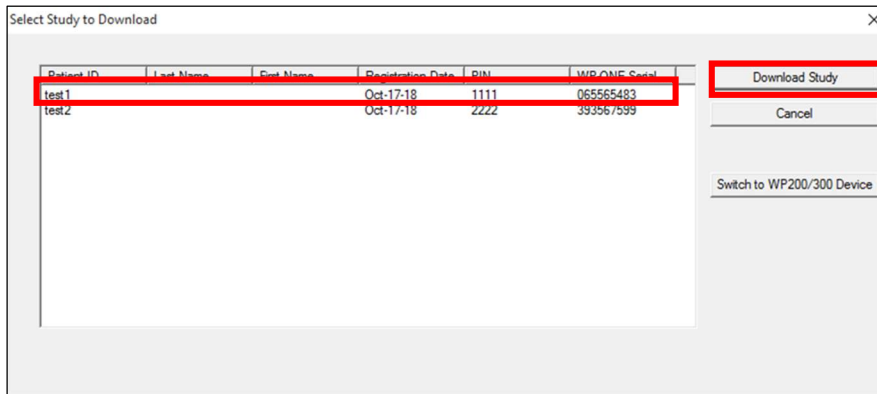

- You can always switch back to WP200 or WP300 by clicking on this button

