



Instructions on Preparing and Retrieving WatchPAT One Study with CloudPAT

CloudPAT Short Instructions – Refer to CloudPAT manual for more


1 Log In

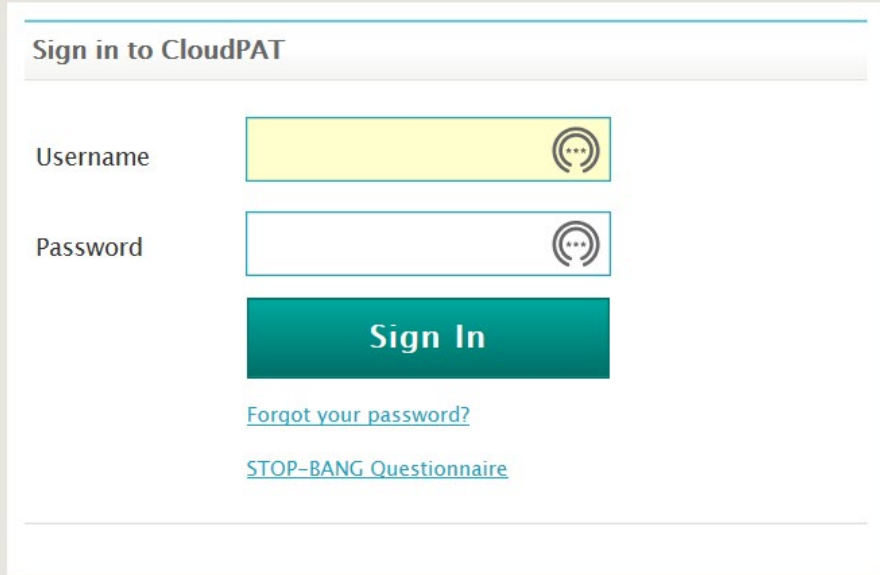

2 Create New Study

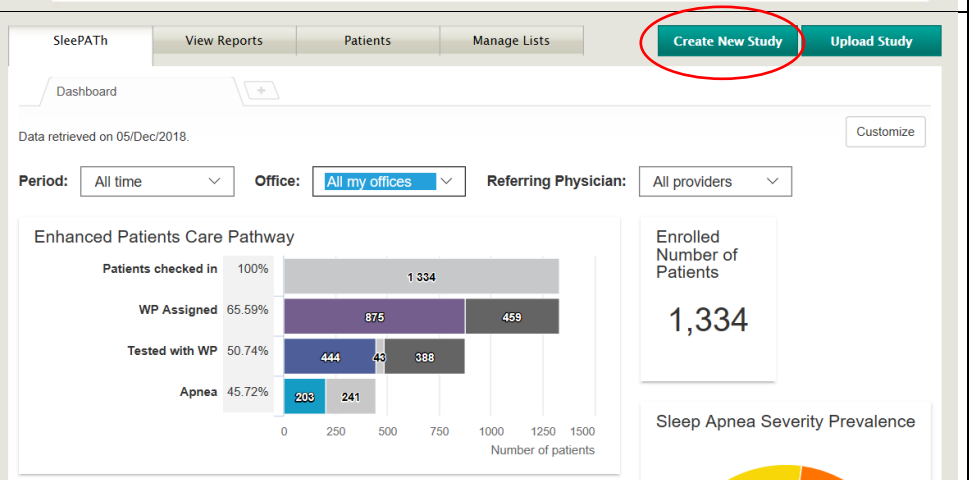

3 1. Enter Patient ID and demographics
2. Click on more details and enter Height and Weight at a minimum
3. Attach Files if necessary
4. Click on Register WP1

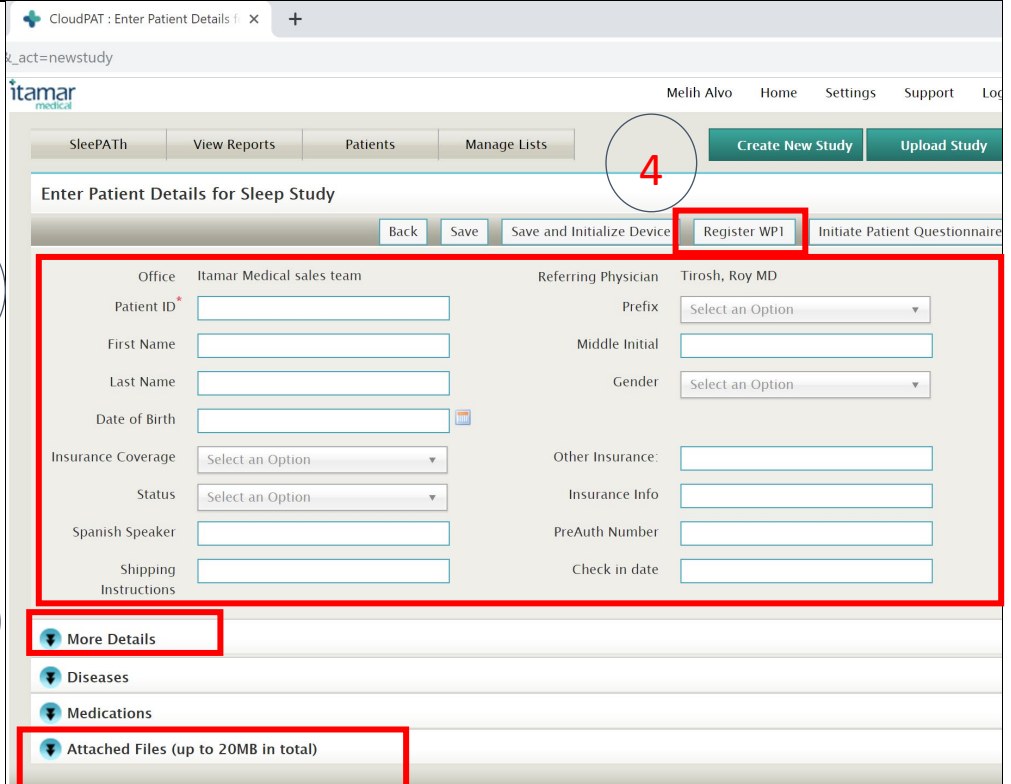

4 Fill out the mandatory information including:

- Device SN (Can be found on the side of WatchPAT ONE box)
- PIN
- Max Number of PIN Retries
- PIN Type
- Email for notification
- Assign Interpreting Physician now or after completion

Pls Note:

Define a PIN for your patient based on PIN selection criteria (4 first/last digit of SSN, credit card, mobile phone etc.)

2- Click "Proceed with registration"

5


Please note messages

- Registration was successful
- You can click on **Back** button if you wish to see the patients data and add additional information

6


Device is now ready for Patient
Be sure to:

- Have them download the WatchPAT One application to their smart phone (Apple or Android)
- Give them a Triple A battery
- Let them know all the instructions for using the WatchPAT one is within the app
- Keep the phone within 15 feet of their bed and keep the phone plugged in
- Hit the End Study button in the morning when they wake up.

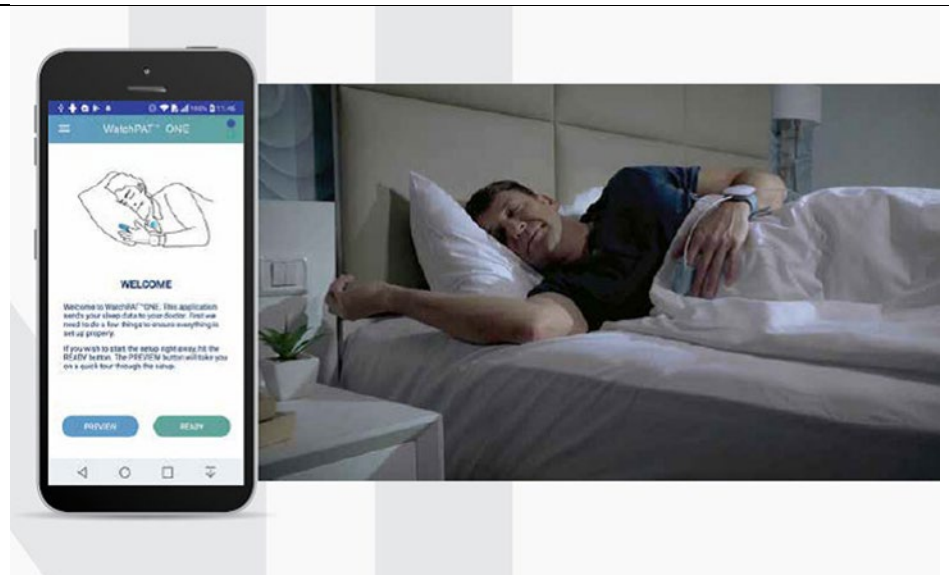


Retrieving Data on WatchPAT One device
 CloudPAT Short Instructions – Refer to CloudPAT manual for more


1 Log In

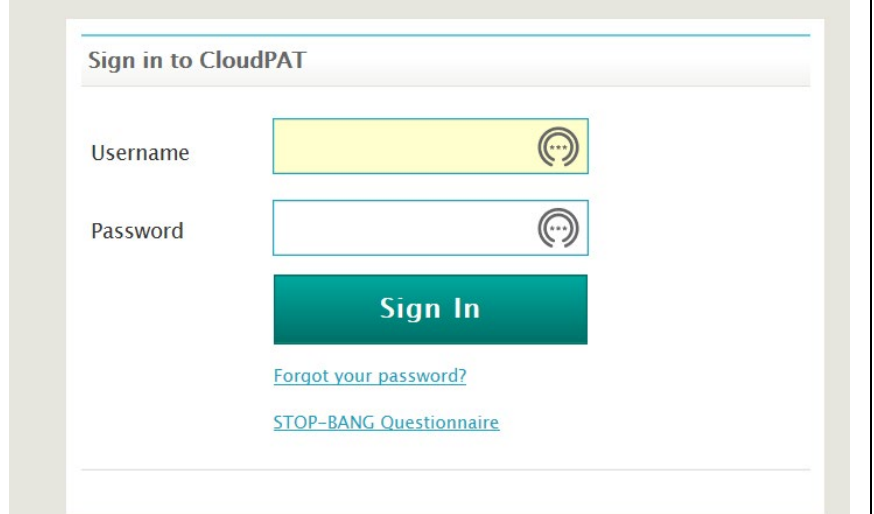

2 In the morning sign back in to retrieve your patients test information
 Click on View Report

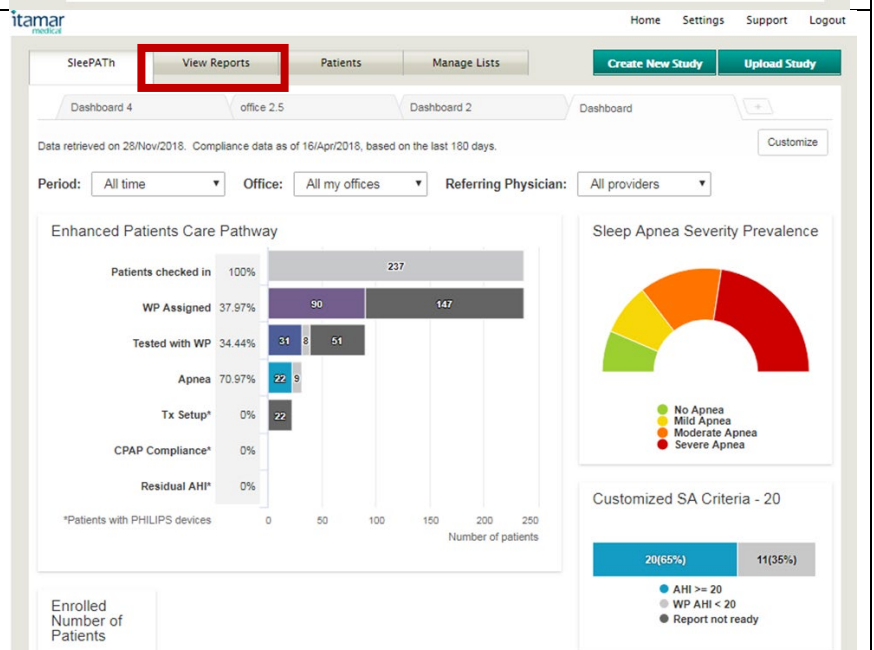

3 Click on the Patient ID

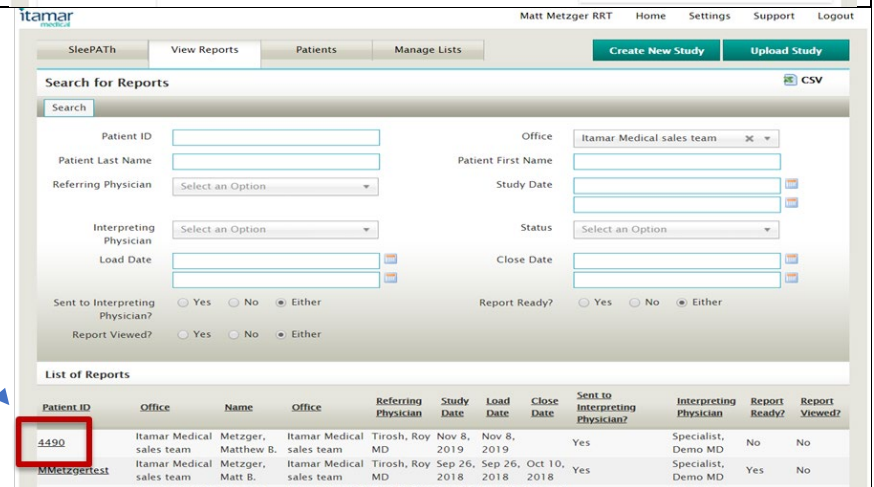

4 Report Details and Assigning Physician

- Select the Interpreting Physician
- Check the Min Desat – 3 or 4%
- Click Save

File is sent to Interpreting Physician if not selected during Setup

You can view the Report by clicking on Analysis File

Getting Interpreted Test Report

- Check for completed interpretations by clicking on the View Reports tab
 - Notification by Email or simply sign in to check
- Then click on the Patient ID once you see the Report Ready has Yes

Patient ID	Office	Name	Office	Referring Physician	Study Date	Load Date	Close Date	Sent to Interpreting Physician?	Interpreting Physician	Report Ready?	Report Viewed?
4490	Itamar Medical sales team	Metzger, Matthew B.	Itamar Medical sales team	Tirosh, Roy MD	Nov 8, 2019	Nov 8, 2019	Nov 19, 2019	Yes	Specialist, Demo MD	Yes	Yes

Open the Report

- Click on Results file and then Right click on Report to see option to **Print**, or **Save** to a location on your computer system. This will be saved as a PDF.