


CloudPATTM **Featured with SleePATh**TM **Module**

Addendum to Web Software Operation Manual for Software Version 2.7

Itamar Medical REF OM2200001

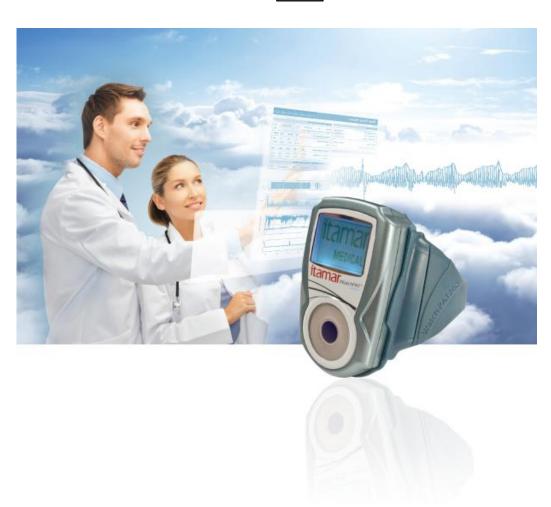

WatchPATTM is a trademark of Itamar Medical, Ltd.

Ronly Caution: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner

Copyright © 2011-2019 by Itamar Medical Ltd.

Software

Version:

2.7

This manual and the information contained herein are confidential and are the sole property of Itamar Medical Ltd. Only Itamar Medical Ltd. or its licensees have the right to use this information. Any unauthorized use, disclosure or reproduction is a direct violation of Itamar Medical's proprietary rights.

DISCLAIMER

Itamar Medical Ltd. shall not be held responsible in any manner for any bodily injury and/or property damage arising from operation or use of this product other than that which adheres strictly to the instructions and safety precautions contained herein and in all supplements hereto and according to the terms of the warranty provided in the License Agreement in Appendix F. Screen layouts are subject to change from time to time without notification and without being altered in this Web software operation manual as long as the intent of the content has not been altered significantly.

This product and/or method of use, is covered by one or more of the following US patents: 6319205, 6322515, 6461305, 6488633, 6916289, 6939304, 7374540, as well as any pending US patent applications and corresponding patents and/or applications filed in other countries.

Itamar Medical Ltd.
9 Halamish St., P.O.Box 3579
Caesarea Ind. Park, 3088900, Israel
Tel + 972 4 617 7000
Fax + 972 4 627 5598
www.itamar-medical.com
support@itamar-medical.com

EN ISO 13485:2016

Table of Contents

1	Introduction to CloudPAT TM Version 2.7				
	1.2	zzzPAT S/W – Definition			
	1.3	CloudPAT S/W – Definition			
	1.4	CloudPAT Version 2.7 Overview	6		
	1.5	Terminology	7		
	1.6	Minimum requirements	7		
2 3 4	Ove	ePATh TM Module Overviewerview of PAP Compliance Data in CloudPAT*ient Detailed Compliance Data*	8 9		
	4.2	Patient Compliance section	10		
5		2 PAP Device Information	11 11 11 11		
	5.1	Creating and modifying dashboard			
	5.25.35.4	Data included in dashboard Define patient group Patient Care Pathway Funnel Chart	14		
	5.5	Enhanced Patient Care Pathway Funnel Chart			
	5.6	Compliance Chart*	18		
Appendix A: Troubleshooting					
	Provider: Create list of patients' group from SleePATh dashboard				
A	Appendix C: License Agreement				

Table of Figures

Figure 1 - CPAP Compliance in Patient Details Window	9
Figure 2 - PAP Device Window	
Figure 3 - Compliance Graph Tooltip, period of interest and Ablation Date mark	11
Figure 4 - Typical SleePATh Dashboard	12
Figure 5 - Dashboard widgets selection popup	
Figure 6 - Dates of data used for dashboard display	
Figure 7 - Dashboard patient population group filter	
Figure 8 - Patients Care Pathway Funnel Chart	
Figure 9 - Enhanced Patient Care Pathway Chart	

Itamar Medical Ltd.

Edition Table

Edition	Date	Description	Chapter	Pages
1	July 2018	Release	All	All
2	Oct 2018	Add: Enhanced Patient Care Pathway chart	5.5	16
3	Jan 2019	Note re: filling compliance usage gaps CPAP Compliance, <u>Under 30 Days</u>	3 5.5	8 16



Note

This Operation Manual is an Addendum to Web Software Operation Manual OM2200000 describing additional functionality introduced with the release of CloudPATTM Software Version 2.7.

Web Software Operation Manual OM2200000 is applicable unless changes are specified in this Addendum.

1 Introduction to CloudPAT™ Version 2.7

1.1 Intended Use of WatchPAT™ devices

There's no change to the intended use of WatchPATTM devices vs OM2200000.

1.2 zzzPAT S/W – Definition

There's no change to zzzPAT S/W – Definition vs OM2200000.

1.3 CloudPAT S/W – Definition

There's no change to CloudPAT S/W – Definition vs OM2200000.

1.4 CloudPAT Version 2.7 Overview

Refer to the basic functionality of CloudPAT Version 2.7 to OM2200000.

The CloudPAT horizon is expanded beyond a tool for conducting WatchPAT Sleep Study and now includes monitoring of treatment compliance and effectiveness by integrating information from CPAP provider.



Note

The expanded capabilities are currently available only for patients with Philips' PAP devices given by specific DME providers.

The following CloudPAT version 2.7 expanded capabilities are available:

a. New Dashboard Widget: Enhanced Patient Care Pathway Funnel Widget

Web Software Operation Manual OM2200000 together with this manual provides the information necessary for routine use of CloudPAT Version 2.7 software.

1.5 Terminology

- o **API** Application Programming Interface
- PAP Compliance Data Server Data server system that collects usage & other measurements from PAP devices and provides API for obtaining summary of compliance data for specific identified patient
- o **PAP Compliance Data -** Usage & other measurements from PAP devices, including hours of usage per night and Residual AHI per night.
- o **PAP -** Positive Airway Pressure
- o **CPAP -** Continuous Positive Airway Pressure
- Residual AHI Apnea Hypopnea Index as calculated by the PAP device/provider.

1.6 Minimum requirements

An internet connection using a PC workstation with Microsoft Win7 and above, running at a minimum upload speed of 1 Mbps and supported browser (Internet Explorer version 11 or higher, Edge, Chrome or Firefox) is required.

2 SleePATh™ Module Overview

SleePATh Module is a Patient Care Pathway dashboard enabling physicians to track their patients' sleep apnea, CPAP compliance and treatment effectiveness*. Physician can easily focus on the patients group of interest based on their care pathway status.

With the introduction of SleePATh™ Module, CloudPAT integrates Check-in procedure and STOP-BANG screening questionnaires, initiation of WatchPAT sleep study, sleep data management including study analysis, obtaining PAP daily usage compliance data and daily AHI score from PAP device for monitoring treatment effectiveness.



Refer to section 2 SleePAThTM Module Overview for detailed description.

3 Overview of PAP Compliance Data in CloudPAT*

CloudPAT is configured to interface with PAP Compliance data servers for obtaining patients' daily PAP Compliance Data, including usage (h) per night and residual AHI per night.

To obtain the patients' daily PAP Compliance Data, first the patient has to be identified by the PAP Compliance data servers (see 4.1).

CloudPAT is obtaining PAP compliance data **daily** for all identified patients actively using PAP device. Subsection 4.2 describes how to identify if patient is actively using PAP device.

Patient's daily PAP device usage and residual AHI (provided by PAP device) are displayed in CPAP Compliance section in Patient Details screen (see 4.2.4).

Patients' PAP Compliance data is also used for generating the data for the SleePATh dashboard – Compliance Chart and Patient Care Pathway funnel chart (see 5.4 Patient Care Pathway Funnel Chart and 5.6 Compliance Chart*).



Note

PAP Compliance data will be available one day after patient's identification by PAP Compliance data servers.

^{*} Currently available only for patients with Philips' PAP devices given by specific DME providers.



Note

Gaps in daily usage compliance data are considered as zero (0) minutes of usage.

Residual AHI is only calculated from daily data received from CPAP Compliance Data Server

* Currently available only for patients with Philips' PAP devices given by specific DME providers.

4 Patient Detailed Compliance Data*

The following enhancements were implemented to Patient Details Window to enable the connection between patients and their PAP data:

- 1) PAP Device Button
- 2) CPAP Compliance Section

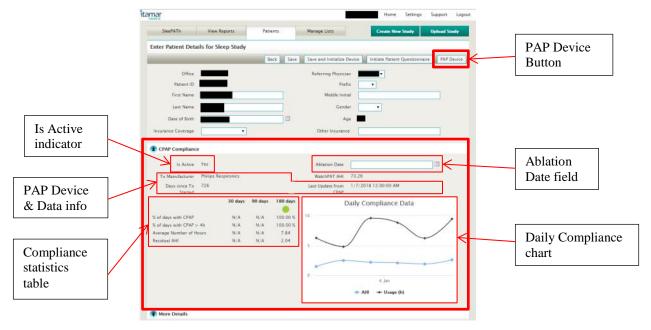


Figure 1 - CPAP Compliance in Patient Details Window

* Currently available only for patients with Philips' PAP devices given by specific DME providers.

4.1 PAP Device Button

CloudPAT patients must be identified by the PAP Provider server prior to reception of their CPAP data.

Clicking on the PAP Device button opens the PAP Device window, automatically filling-in the Last Name, First Name and Date of Birth as entered to the CloudPAT system.

Clicking on Submit button, CloudPAT system sends the patient details to the PAP Compliance server. If patient is uniquely identified by the PAP Compliance Server, the Device Serial Number of the PAP device allocated to the patient is returned and included in Patient Details. The window is automatically closed.



Figure 2 - PAP Device Window

When the PAP device Window is reopened for the patient, after successfully identified by the PAP Compliance server, the Device Serial Number is displayed, together with the message that the PAP device was already associated with this patient.



Note

It is strongly recommended to confirm that the PAP Device Serial Number of the patient matches the number displayed in the window.

If mismatch found, report to your administrator.

If patient was not identified, or uniquely identified, by the PAP Compliance Sever, the system returns with error message, and PAP data for this patient is not available.

4.2 Patient Compliance section

The Patient Compliance section includes the following subsections:

- a. Is Active indicator
- b. PAP Device information
- c. Ablation Date field
- d. Compliance statistics table
- e. Daily Compliance chart

The Patient Compliance section is available only after the patient is identified by the PAP Provider server and PAP data is received for the patient.

4.2.1 Is Active indicator

Is Active field provides indication that user is actively using CPAP based on the time from the last CPAP update. When patient enters Inactive state, system stops polling for compliance data.



Note

When patient is In Active = No, and regained CPAP usage, the operator shall re-establish retrieval of compliance data clicking on PAP Device button and submit connection request to the server.

4.2.2 PAP Device Information

This subsection provides information on the CPAP Device Manufacturer, number of days since device has been set up for patient, and date/time of last update from CPAP.

4.2.3 Ablation Date field

User can update the patient's ablation date. The ablation date will be marked on graph and used as a period filter of the dashboard.

4.2.4 Compliance statistics table

After patient is associated with PAP device, the system retrieves the patient's compliance data. The Compliance section provides the following specific CPAP compliance data and effectiveness of the therapy over last 30, 60 and 180 days:

- 1) % of days with CPAP: Days for which CPAP use was recorded
- 2) % of days with CPAP > 4h: CPAP use was recorded for more than four hours
- 3) Average Number of Hours: Average number of hours only on days CPAP use was recorded (including nights of no use)
- 4) Residual AHI: Average AHI, only on days CPAP use was recorded

Compliance color coded icon is provided for each period:

- RED- Less than four hours per night of average use
- ORANGE Average AHI is over 10 and is at least half of the AHI observed in the WatchPAT study
- YELLOW Average AHI is over 10 but is less than half of the AHI observed in the WatchPAT study
- GREEN Average AHI is under 10

4.2.5 Daily Compliance Chart

Patient Daily Compliance Data graph shows usage behavior and treatment effectiveness displaying CPAP hours of use and AHI per night over the last 180 days. Operator can zoom in marking the period of interest, and zoom out clicking on Reset Zoom button in the graph area.



Figure 3 - Compliance Graph Tooltip, period of interest and Ablation Date mark

5 SleePATh™ Module

SleePATh Module is a Patient Care Pathway dashboard enabling physicians and provide office admins to track their patients' sleep apnea, CPAP compliance and treatment effectiveness*. Physician can easily focus on the patients group of interest based on their care pathway status. Refer to Figure 4 for typical SleePATh dashboard layout.

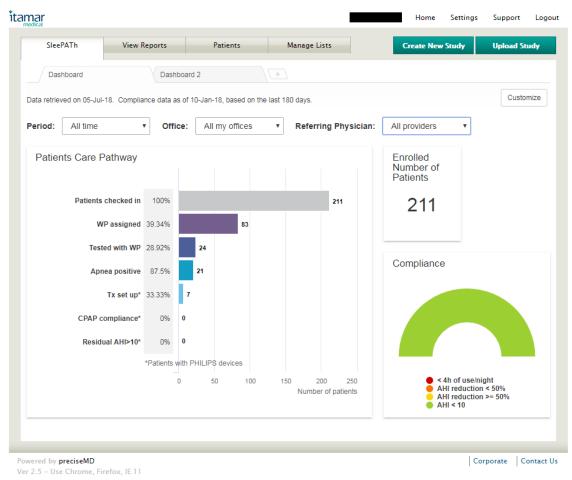


Figure 4 - Typical SleePATh Dashboard

With the introduction of SleePAThTM Module, CloudPAT now covers 5 stages of Sleep Patient Care Pathway:

- a) Patient Checked-in
- b) Initiation of WatchPAT sleep study
- c) Sleep data upload and study analysis
- d) Monitor PAP daily usage compliance data
- e) Monitor daily AHI score from PAP device for monitoring treatment effectiveness.



The SleePATh dashboard is customizable, and can include the following widgets:

- 1) Patient Care Pathway Funnel
- 2) Compliance Chart*
- 3) Number of Enrolled Patients

^{*} Currently available only for patients with Philips' PAP devices given by specific DME providers. Compliance is assessed after 30 days since PAP device setup.



Note

Offices that do not have the connection to the compliance module should not add the Compliance Cart to their dashboard although is included in the available dashboard widgets as chart will always be empty.

Users can create their own dashboards.

5.1 Creating and modifying dashboard

To create a new dashboard, click on the "+" tab, and assign a name to the dashboard.



Clicking on *Add widget* button, a popup window is displayed, allowing the selection of a widget. Clicking on a widget will place it on the dashboard, and user can arrange the layout of the dashboard screen by dragging it to desired location.



Figure 5 - Dashboard widgets selection popup

Widgets can be removed by clicking on the *x* on its upper right corner. When all widgets are added and placed on their location, click *Save*.

To switch between dashboard, click on the desired tab.

5.2 Data included in dashboard

The dashboard provides information on the data which was used to create the statistics. **Data retrieved on**: CloudPAT's system time.

Compliance data as of*: the most recent date compliance data was obtained. **Based on the last** 30 days*: days used for the analysis of PAP compliance and treatment effectiveness.

Data retrieved on 01-Jul-18. Compliance data as of 09-Jan-18, based on the last 30 days.

Figure 6 - Dates of data used for dashboard display

* Currently available only for patients with Philips' PAP devices given by specific DME providers. Compliance is assessed after 30 days since PAP device setup.

5.3 Define patient group

User can focus on specific patient population group that will be presented on the dashboard defining combination of 3 filters:

- A. **Period** Filtering by specific period based on:
 - a. Admittance date: Patient admitted over the last N weeks/months where N=All time, 1 week, 2 weeks, 1 month, 3 months, 6 months, 12 months, 24 months and 36 months.
 - b. Ablation date: Patient had ablation over the last N days where N=30 days, 90 days and 180 days.
- B. **Office** either all offices the user belongs to or specific office.
- C. **Referring Physician** either all physicians or specific one.



Figure 7 - Dashboard patient population group filter

5.4 Patient Care Pathway Funnel Chart

Patient Care Pathway Funnel (see Figure 8) is a bar chart that provides statistics on the selected patient group, along seven stages starting from their check-in towards assessment of their treatment effectiveness.

For each stage along the pathway, the total number of patients in this stage is provided as well as the percentage of them from the previous stage.

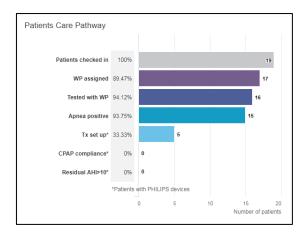


Figure 8 - Patients Care Pathway Funnel Chart

The following stages are displayed in the funnel

- 1) Patients checked in: Number of patients in selected group (percentage is always 100%).
- 2) **WP assigned**: Patients assigned with initialized WatchPAT device, including patients returned the device and WP study was uploaded to CloudPAT but yet not analyzed. The number of patients is displayed and their percentage of the checked-in patients.
- Tested with WP: Number of patient successfully completed the WatchPAT Study and successfully analyzed, and their percentage out of the patients assigned with WatchPAT.
- 4) **Apnea positive**: Number of patient that were positively diagnosed with sleep apnea, i.e. AHI above threshold¹, and their percentage of all WP tested patients.
- 5) **Tx Setup*:** Number of patients provided with Philips' PAP devices, and their percentage out of the patients diagnosed with Sleep Apnea.
- 6) **CPAP compliance*:** Number of patients that have slept with CPAP at least 4 hours per night over at least 21 out of the last 30 days or with average use of 4 hour per night over the last 30 days and their and percentage out of number of patients provided with CPAP device.
- 7) **Residual AHI>10 *:** Number of patients with High Residual AHI (reported by the CPAP) and their percentage out of the patients maintaining CPAP compliance.
 - * Only for patients with Philips' PAP devices by specific DME providers are included in the statistics. Compliance is assessed after 30 days since PAP device setup.

If the specific office or all offices that has been chosen to be displayed are not one of the specific DME providers, only the first 4 bars will be displayed, and the compliance stages will be hidden.

Clicking on any of the bars in the Patient Care Pathway bar chart will display a list of patients included in the chosen group, based on the selected criteria (see 5.3 Define patient group). Clicking on a patient within the list displays the Patient Details window.

_

¹ AHI Threshold is configurable per office. Refer to your office admin to determine the AHI threshold defined for your office(s).

5.5 Enhanced Patient Care Pathway Funnel Chart

The Enhanced Patient Care Pathway Funnel is a stacked bar chart that provides statistics on the selected patient group, along four stages starting from their check-in towards assessment of their Sleep Apnea (see Figure 9).

For each stage along the pathway, the total number of patients in this stage is provided as well as the percentage of them from the previous stage. Each stage is built from 1-3 subsections, main/first section and additional 1-3 complementary sections, that all together sum to the number of patients in the main/first section of the previous stage. The first (main) section is assigned with unique color, while the complementary sections are in gray levels.

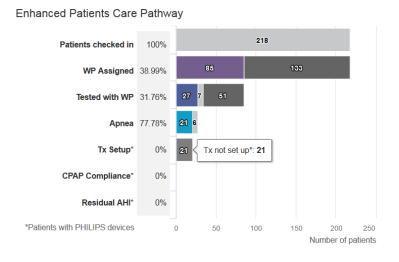


Figure 9 - Enhanced Patient Care Pathway Chart

The following stages are displayed in the funnel

1) Patients checked in: Number of patients in selected group (percentage is always 100%).

2) WP assigned:

- a. <u>WP assigned</u> (main): Patients assigned with initialized WatchPAT device, including patients returned the device and WP study was uploaded to CloudPAT but yet not analyzed.
- b. <u>WP not assigned</u>: Patients yet not assigned with WatchPAT device.

3) Tested with WP:

- a. <u>Tested with WP</u> (main): Patients successfully completed the WatchPAT Study and successfully analyzed.
- b. <u>WP Study Failed</u>: Patients that their WP study was uploaded to CloudPAT but no AHI was calculated.
- c. Not tested yet: Patients that their WP study yet not uploaded to CloudPAT.

4) Apnea:

- a. <u>Apnea Positive</u> (main): Patients that were positively diagnosed with sleep apnea, i.e. AHI above the office-defined threshold.
- b. Apnea Negative: Patients that their AHI is below the threshold.
- c. Report not ready²: Patients that their study was successfully uploaded to CloudPAT but their report is yet not ready.

² Applicable only for offices configured not to show analysis file, and therefore, the AHI is only taken from locked report. For all other offices, this section will always be zero.

5) Tx Setup*:

- a. <u>Tx Setup</u>: Number of patients provided with Philips' PAP devices, and their percentage out of the patients diagnosed with Sleep Apnea.
- b. Tx not set up: Patients that yet not provided with Philips' PAP device.

6) CPAP compliance*:

- a. <u>Complying</u>: Number of patients that have slept with CPAP at least 4 hours per night over at least 21 out of the last 30 days or with average use of 4 hour per night over the last 30 days and their and percentage out of number of patients provided with CPAP device.
- b. <u>Inactive</u>: Patients that there is no data on device usage for more than 30 days.
- c. Not Complying: Number of patients that have <u>not</u> slept with CPAP at least 4 hours per night over at least 21 out of the last 30 days or with average use of 4 hour per night over the last 30 days and their and percentage out of number of patients provided with CPAP device.
- d. <u>Under 30 Days</u>: Patients that have been on CPAP device for fewer than 30 days.

7) Residual AHI*:

- a. <u>Residual AHI>10</u>: Number of patients with High Residual AHI (reported by the CPAP) and their percentage out of the patients maintaining CPAP compliance.
- b. Residual AHI<=10

Clicking on any of the sections of the bars in the Patient Care Pathway bar chart will display a list of patients included in the chosen group, based on the selected criteria. Clicking on a patient within the list displays the Patient Details window.

^{*} Only for patients with Philips' PAP devices by specific DME providers are included in the statistics. Compliance is assessed only after 30 days since PAP device setup.

5.6 Compliance Chart*

The Compliance half-pie chart displays the overall compliance and treatment effectiveness for the selected group of patients.



Note

The Compliance Chart is relevant **only** for specific DME providers. If these DME provides are not among the offices assigned to the user, the Compliance Chart will be empty. Therefore, it is not recommended to add to the dashboard.

- 1) < 4h of use/night: None-complying patients that are failing to use the PAP for 4h/night on average. The color of the half pie segment for this category is Red ().
- 2) AHI reduction < 50%: Patients with none effective treatment, that are using the PAP for more than 4h/night on average, but their Residual AHI (reported by the PAP device) is not reduced to 50% of what it was diagnosed with WatchPAT. The color of the half pie segment for this category is Orange ().
- 3) AHI reduction >= 50%: Patients with effective (but not optimal and >= 10) treatment, that are using the PAP device and get their Residual AHI reduced to below 50%, but are still have not reached optimal AHI level. The color of the half pie segment for this category is Yellow (_).
- 4) **AHI < 10:** Patients with Optimal treatment that are using the PAP device and have reached target Residual AHI levels. The color of the half pie segment for this category is Green ().

Clicking on any of the half-pie segment displayed in the Compliance chart will display a list of patients included in the chosen group, based on the selected criteria(see 5.3 Define patient group). Clicking on a patient within the list displays the Patient Details window.

* Currently available only for patients with Philips' PAP devices given by specific DME providers.

Appendix A: Troubleshooting



Note

Make sure you are using Chrome, Firefox, Edge or Internet Explorer 11 or above as your browser.

Trouble	Possible	Solution	Error Message
	Cause		
Patient Detail	s screen		
PAP Device	CloudPAT	Change patient's details entered	
not found for	Patient details	in CloudPAT to match patient's	
patient	not matching	details on Compliance Server	
	patient's	_	No patient has been found. Please verify the information.
	details on		
	Compliance		
	Server		
PAP Device	Partial	Update patient's details on	
not found for	CloudPAT	CloudPAT to include First and	Please enter full name and date of birth, or the device serial number.
patient	Patient details	Last name, Date of Birth or PAP	riedse enter full name and date of birth, of the device serial number.
		device serial number.	
PAP Device	Multiple	Include PAP device serial	
not found for	patients on	number Update patient's details	
patient	Compliance	PAP device serial number.	
	Server match		Multiple patients have been found. Please refine the search fields.
	CloudPAT		
	Patient details		
Patient's	Patient state is	Click on PAP device and submit	
Compliance	In Active	to restart data retrieval for the	

Itamar Medical Ltd.

Trouble	Possible Cause	Solution	Error Message
data not updated	Ottabo	patient.	
PAP Device button not available	Patient has no study	Upload WP study	
PAP Device button not available	Office not configured to be Philips DME	Contact your Administrator	
CPAP data not presented for patient	PAP Device not associated to patient	Click PAP Device button.	
CPAP data not presented for patient	CPAP Data yet not obtained from PAP server	Wait 1 day to allow daily batch retrieval of the data.	

Appendix B: New CloudPAT Assistant "cheat sheets"

Provider: Obtain Patient PAP Compliance Data*

		Step	Action(s)	Comment
Locate Patient	1.	Identify the desired patient	Search the patient, highlight the patient and click to open Patient Details screen.	Search for Reports Prices ID Report Newed Prices ID Report New ID
	2.	Verify that PAP data can be obtained	Verify that the patient has at least one WP Study.	Previous Studies
PAP Device	3.	Open PAP Device window	Click on PAP Device button	Enter Patient Details for Steep Study Eacl
Connect	4.	Search patient in PAP Compliance server	 Verify that all patient details are displayed on the window Submit button. 	PAP Device Things Represent V North No. State Last Name Date of Simb. 6.27/2001

^{*} Currently available only for patients with Philips' PAP devices given by specific DME providers.

Provider: Create list of patients' group from SleePATh dashboard

		Step	Action(s)	Comment
Dashboard	1.	Open dashboard	Click on SleePATh tab in the main menu.	SleePATh View Reports Dashboard
Period	2.	Select the period of interest	Click on the Period pull down menu and select desired period.	SieePATh View Reports Patients Manage Lists Dashboard Data retrieved on 01-Jul-18. Compliance data as of 09-Jan-18, based on the last 180 days. Period: All time All time Patier 1 weeks 1 months 6 months 12 months 12 months 24 months 36 months 36 months 36 months 17 months 18 do post ablation 19 do post ablation 19 do post ablation 10 do post ablation
Office	3.	Select the Office of interest	Click on the Office pull down menu and select desired office. *If you have only one office skip this step.	Period: 12 months Office: All my offices Referring Physician: All providers
Physician	4.	Select the Referring Physician of interest	Click on the Referring Physician pull down menu and select desired Referring Physician.	Period: 12 months v Office: All my offices v Referring Physician: All providers v
Stage	5.	Select the state of Patients Care Pathway	Within the Patients Care Pathway chart, click on the bar graph associated with the desired state in pathway.	Patients Care Pathway Patients checked in 100% We assigned 100% Tasked with WP 100% Agnee prositive 100% 6 1 2 3 Number of patients.
Compliance	6.	Select the compliance and effectiveness group	Within the Compliance half-pie chart, click on the segment associated with the desired compliance and effectiveness group.	Compliance 44 of use/night AHI reduction < 50% AHI reduction >= 50% AHI < 10
List	7.	Get list of patients	List of Patients window is opened. Data can be exported to excel clicking on CSV icon.	

Appendix C: License Agreement

License To User From Itamar

IMPORTANT - PLEASE READ THIS LICENSE AGREEMENT CAREFULLY BEFORE INSTALLING OR OTHERWISE USING THE LICENSED SOFTWARE (AS DEFINED BELOW) OR THE PRODUCT WITH WHICH YOU RECEIVED THIS LICENSE AGREEMENT. THIS LICENSE AGREEMENT APPLIES TO (a) ALL LICENSED SOFTWARE, (b) ALL LICENSED PRODUCTS (AS DEFINED BELOW), AND (c) ALL THIRD PARTY PRODUCTS INTO WHICH A LICENSED PRODUCT OR LICENSED SOFTWARE IS INCORPORATED. SHOULD YOU HAVE ANY QUESTIONS CONCERNING THIS LICENSE AGREEMENT, PLEASE CONTACT THE VENDOR FROM WHICH YOU PURCHASED THE LICENSED SOFTWARE, LICENSED PRODUCT, OR PRODUCT INTO WHICH A LICENSED PRODUCT OR LICENSED SOFTWARE IS INCORPORATED. YOU MAY ALSO CONTACT ITAMAR AT THE ADDRESS PROVIDED AT THE END OF THIS LICENSE AGREEMENT.

This License Agreement is a legal agreement between you (as an individual, company, organization or other entity) and Itamar Medical Ltd. ("Itamar"). By installing, copying, or otherwise using the Licensed Software, and/or by using the Licensed Product or third party product into which a Licensed Product or Licensed Software is incorporated ("Third Party Product"), you agree to be bound by the terms of this License Agreement with respect to the Licensed Software and Licensed Products. If you do not agree to the terms of this License Agreement, including, without limitation, the Restrictions on Use as provided in Section 2 do not install, use or copy the Licensed Software or use the Licensed Product or the Third Party Product.

The Licensed Software and the Licensed Products are protected by US patent laws, trade secret laws, copyright laws, and international treaty provisions as well as other intellectual property laws and treaties. Therefore, you must treat the Licensed Software and the Licensed Products like any other copyrighted and protected material or product. All title to the Licensed Software and all intellectual property rights in and to the Licensed Software and the Licensed Products shall remain with Itamar.

1. **DEFINITIONS**

- 1.1. "Licensed Product(s)" means the Watch-PAT200, the PAT Probe, Integrated Snore & Body position sensor, Tamper-proof Patient Identification Bracelet and the corresponding components of any Third Party Product with which this License Agreement was received. Some Licensed Products are stand-alone products and some Licensed Products are incorporated as components within Third Party Products, in each case sold or otherwise made available, by Itamar and/or third parties. If you have received this License Agreement with a Third Party Product, this License Agreement applies only to the Licensed Product incorporated as a component within such Third Party Product.
- 1.2. "Licensed Software," means the zzzPAT software, CloudPAT software, and the associated media and accompanying materials provided to you with such software. Some Licensed Software is a stand-alone product and some Licensed Software is incorporated as a component within a Licensed Product, in each case sold or otherwise made available, by Itamar and/or third parties. If you have received this License Agreement with a Licensed Product, which incorporates the Licensed Software as a component within such Licensed Product, this License Agreement applies to the Licensed Software.

2. GRANT OF LICENSE AND RESTRICTIONS ON USE

- 2.1 Itamar hereby grants you a non-exclusive right to use the Licensed Software, solely for its intended use in sleep medicine (with the term "sleep medicine" including Cheyne-Stokes respiration as well as research in sleep medicine and Cheyne-Stokes respiration) (i) with the Licensed Product(s) and (ii) in accordance with the provisions of this License Agreement and the instructions provided in the documentation accompanying the Licensed Software and the Licensed Product You may make one copy of the Licensed Software solely for backup or archival purposes, or transfer the Licensed Software to a single hard disk, provided you keep the original solely for backup or archival purposes. However, you may not cause any Licensed Software, which is not designed for use on a server, to execute or be loaded into the active memory or media of more than one computer at any one time.
- 2.2 Any use of the Licensed Software and/or Licensed Product other than as set forth in Section 2.1 above is strictly forbidden. Without derogating from the generality of the above, you may not:
- Distribute, reproduce, copy, assign, rent, lease, or otherwise transfer the rights granted to you under this License Agreement to any third party except explicitly as set forth in this License Agreement;
- Reverse engineer, recompile, or disassemble, as applicable, the Licensed Software or the Licensed Product, except as expressly permitted by applicable law; or
- Modify in any manner the Licensed Software and/or the Licensed Product unless obtaining the prior written consent of Itamar.

3. TRADEMARKS

Cardio-PATTM, Sleep-PATTM and all trademarks and logos, which appear on or in connection with the Licensed Software and/or the Licensed Products, as may be amended from time to time, are, unless stated otherwise, trademarks of Itamar. No right, license, or interest to such trademarks are generated or granted hereunder other than the limited right to use provided herein, and you agree that no such right, license, or interest shall be asserted by you with respect to such trademarks. You may not remove or destroy any copyright, trademark, logo or other proprietary marking or legend placed on or contained in the Licensed Software or a Licensed Product.

4. LIMITED WARRANTIES AND DISCLAIMERS

- a. <u>Against</u> Infringement. Itamar hereby warrants to you that it has the right to grant you the license to use the Licensed Software and/or the Licensed Product and to enter into this License Agreement and that neither the Licensed Software nor the Licensed Product(s) infringes the intellectual property rights of any third party.
- b. As to Licensed Product. Itamar warrants that the Licensed Product, with which this License Agreement was delivered, will be free from defects in design, materials and workmanship for a period of one year from the date of delivery of the Licensed Product to you. If the Licensed Product contains a defect in design, materials or workmanship and such Licensed Product is returned to Itamar within one (1) year of delivery of the Licensed Product to you, Itamar will repair or replace the Licensed Product, or issue a credit for the purchase price of the Licensed Product, with the choice to repair, replace or credit being within the sole discretion of Itamar. The foregoing repair, replacement or credit remedy will be your sole remedy for breach of the warranty set forth in this Section 4(b).
- c. As to Licensed Software. Itamar warrants that for a period of ninety (90) days from the date of delivery of the Licensed Software to you, the Licensed Software will, under normal use, be free from defects in materials and workmanship and will perform substantially as it is intended to perform. If during such

ninety (90) day period, the Licensed Software has a defect in materials or workmanship or does not perform substantially as it is intended to perform, Itamar shall (a) attempt to correct or assist you around errors with efforts which Itamar believes suitable to the problem, (b) replace the Licensed Software with a functionally equivalent software, or (c) issue a credit for the purchase price of the Licensed Software, with the choice to correct or assist, replace or credit being within the sole discretion of Itamar. The foregoing correct or assist, replacement or credit remedy will be your sole remedy for breach of the warranty set forth in this Section 4(c).

d. Limitation of Warranties. The warranties contained in Sections 4(b) and 4(c) above do not cover damage to the Licensed Products or the Licensed Software caused by accident, misuse, abuse, negligence, failure to install in accordance with Itamar's installation instructions, failure to operate under conditions of normal use and in accordance with the terms of the documentation accompanying the Licensed Product and/or the Licensed Software, failure to maintain in accordance with applicable documentation accompanying the Licensed Product and/or the Licensed Software, alteration or any defects not related to materials or workmanship, or in the case of Licensed Products, design, materials or workmanship. This warranty does not cover damage which may occur in shipment. This warranty does not apply to Licensed Products and/or Licensed Software not purchased new. This warranty does not apply to any Licensed Product or any individual parts of a Licensed Product which have been repaired or altered by anyone other than Itamar or a person or entity authorized by Itamar to repair Licensed Products.

While every reasonable effort has been made to ensure that you will receive Licensed Software that you can use, Itamar does not warrant that the functions of the Licensed Software will meet your requirements or that the operation of the Licensed Software will be uninterrupted or error free. Itamar is not responsible for problems caused by changes in the operating characteristics of the hardware or operating system software you are using, nor for any problems in the interaction of the Licensed Software with non-Itamar software.

ITAMAR HEREBY DISCLAIMS, WITH RESPECT TO THE LICENSED PRODUCTS AND THE LICENSED SOFTWARE, ALL OTHER WARRANTIES AND CONDITIONS, WHETHER EXPRESS OR IMPLIED, INCLUDING, BUT NOT LIMITED TO, ANY WARRANTIES OR CONDITIONS OF OR RELATED TO MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE, ACCURACY OR COMPLETENESS OF INFORMATION, LACK OF NEGLIGENCE AND CORRESPONDENCE TO DESCRIPTION.

5. LIMITATION OF LIABILITY

- (A) TO THE MAXIMUM EXTENT PERMITTED BY APPLICABLE LAW, EXCEPT FOR DAMAGES ARISING UNDER SECTION 4(A) ABOVE, IN NO EVENT SHALL ITAMAR BE LIABLE TO YOU FOR DAMAGES IN EXCESS OF THE PURCHASE PRICE YOU PAID FOR THE LICENSED SOFTWARE, THE LICENSED PRODUCT OR THE APPLICABLE THIRD PARTY PRODUCT. THE FOREGOING LIMITATION SHALL BE APPLICABLE REGARDLESS OF WHETHER THE ACTION GIVING RISE TO SUCH DAMAGES IS IN TORT, CONTRACT, STRICT PRODUCTS LIABILITY, OR OTHERWISE.
- (B) IN NO EVENT SHALL ITAMAR BE LIABLE FOR ANY SPECIAL, INCIDENTAL, INDIRECT OR CONSEQUENTIAL DAMAGES WHATSOEVER ARISING OUT OF OR IN ANY WAY RELATED TO THE USE OF OR INABILITY TO USE THE LICENSED SOFTWARE AND/OR THE LICENSED PRODUCT AND/OR THE THIRD PARTY PRODUCT, OR THE PROVISION OF OR FAILURE TO PROVIDE SUPPORT SERVICES BY ITAMAR, EVEN IF ITAMAR HAS BEEN ADVISED OF THE POSSIBILITY OF SUCH CONSEQUENTIAL DAMAGES. THE FOREGOING

DISCLAIMER OF CONSEQUENTIAL DAMAGES SHALL BE APPLICABLE REGARDLESS OF WHETHER THE ACTION GIVING RISE TO SUCH DAMAGES IS IN TORT, CONTRACT, STRICT PRODUCTS LIABILITY, OR OTHERWISE.

(C) IN ORDER TO BE ENTITLED TO INDEMNIFICATION HEREUNDER IN CONNECTION WITH AN INFRINGEMENT CLAIM, YOU MUST (i) NOTIFY ITAMAR IN WRITING PROMPTLY UPON BECOMING AWARE OF AN INFRINGEMENT CLAIM OR THE POSSIBILITY THEREOF, (ii) GRANT ITAMAR SOLE CONTROL OF THE SETTLEMENT, COMPROMISE, NEGOTIATION AND DEFENSE OF ANY SUCH ACTION, AND (iii) PROVIDE ITAMAR WITH ALL INFORMATION RELATED TO THE ACTION THAT IS REASONABLY REQUESTED BY ITAMAR. NOTWITHSTANDING THE FOREGOING, ITAMAR SHALL HAVE NO INDEMNIFICATION OBLIGATIONS WITH RESPECT TO ANY INFRINGEMENT CLAIM TO THE EXTENT ARISING FROM YOUR USE OF THE LICENSED PRODUCT AND/OR LICENSED SOFTWARE IN CONJUNCTION WITH OTHER HARDWARE OR SOFTWARE WHERE USE WITH SUCH OTHER HARDWARE OR SOFTWARE GAVE RISE TO THE INFRINGEMENT CLAIM.

6. TERMINATION

Without prejudice to any other rights or remedies, Itamar may terminate this License Agreement immediately if you fail to comply with any of its terms and conditions. In the event of such termination, you must, within ten (10) business days of receiving notice of termination from Itamar, cease all use of the Licensed Software and destroy all copies thereof, and cease all use of the Licensed Product (including Licensed Product incorporated within Third Party Product).

7. TRANSFERABILITY

You may only transfer or assign the rights and obligations hereunder together with the Licensed Software and/or the Licensed Product or Third Party Product as a whole, without retaining any rights or, subject to Sections 2 and 3 above, any obligations arising after the date of such transfer or assignment, or retaining any installed or uninstalled copy of the Licensed Software, the Licensed Product or the Third Party Product. Any attempt by you to rent, lease, sublicense, assign or transfer any of the rights, duties or obligations hereunder in any other way is forbidden and shall be null and void.

8. SEVERABILITY

Should any term or provision of this License Agreement be declared void or unenforceable by any court of competent jurisdiction in any country or countries, such declaration shall have no effect on the remainder of this License Agreement in such country or countries, or on this License Agreement in other countries.

9. NO WAIVER

The failure of either party to enforce any rights granted to it hereunder or to take action against the other party in the event of any breach hereunder shall not be deemed a waiver by that party as to subsequent enforcement actions in the event of future breaches.

10. GOVERNING LAW AND JURISDICTION

This License Agreement is governed by the laws of the State of New York, excluding its conflict of laws principles. The United Nations Convention on Contracts for the International Sale of Goods shall not apply to any of the transactions contemplated by this License Agreement.

11. ENTIRE UNDERSTANDING

This License Agreement represents the complete and exclusive understanding between you and Itamar concerning the license by Itamar to you of Licensed Software and Licensed Products and supersedes all prior agreements and representations between the parties with respect to the subject matter hereof, unless specifically stated otherwise in a writing signed by Itamar and you. This License Agreement may not be amended other than by a written agreement specifically intended for this purpose and signed by Itamar and you.

Note: Should you have any questions concerning this License Agreement, or if you desire to contact Itamar for any reason, please write to: Itamar Medical Ltd., 9 Halamish St., P.O.Box 3579, Caesarea Ind. Park, 3088900, Israel, Facsimile: +972-4-627 5598, or visit Itamar's web site at www.itamar-medical.com.