


ZZZPAT For Watch-PAT[™] Software Extended And Illustrated Manual

Itamar Medical REF OM2197450

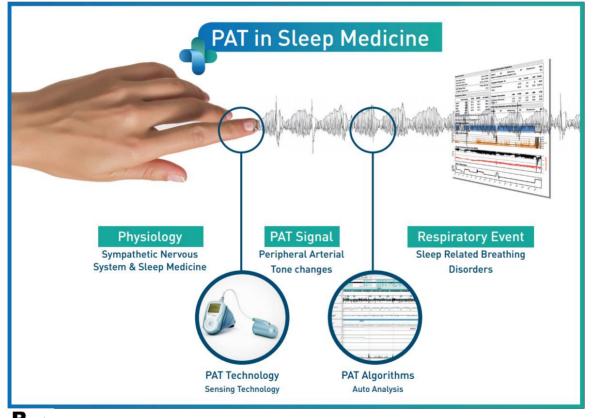

R^{only} Caution: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner

Copyright © 2022 Itamar Medical Ltd. All rights reserved. WatchPAT and PAT are trademarks or registered trademarks of Itamar Medical Ltd., a subsidiary of ZOLL Medical Corporation, in the United States and/or other countries. All other trademarks are the property of their respective owners.

DISCLAIMER

Itamar Medical Ltd. shall not be held responsible in any manner for any bodily injury and/or property damage arising from operation or use of this product other than that which adheres strictly to the instructions and safety precautions contained herein and in all supplements hereto and according to the terms of the warranty provided in the License Agreement available at https://www.itamar-medical.com/Images/licensewp.pdf.

This product and/or method of use, is covered by one or more of the following US patents: 6319205, 6322515, 6461305, 6488633, 6916289, 6939304, 7374540, as well as any pending US patent applications and corresponding patents and/or applications filed in other countries.

Itamar Medical Ltd. 9 Halamish Street, PO 3579 Caesarea 3088900, Israel Tel: International + 972-4-617-7000, US 1-888-7ITAMAR Fax + 972 4 627 5598 www.itamar-medical.com support@itamar-medical.com



EN ISO 13485:2016 See appendix D for contact information of the regulatory authorized representative

Edition Table

Edition	Date	Description	Chapter	Pages
1	Jan 2020	Initial	All	All
		Based on OM2197434 Ed.23		
2	May 2020	Intended use WP1	1.3	12
		Transferring a Study to Itamar Medical	5.6	101-102
		Manage WatchPAT One Registered	6.10	114
		Devices		
3	June 2021	Updated SQL Express Version	2.3	13
		Update ODI % calculate value	3.3	42
			4.6	89
			App B	129
		New Report Layout	4.6.1	91-94
		Added cybersecurity risk related note	3.3.5	40
			3.3.9	47
4	Feb 2022	Detailed report	3.3.7	45-46
			4.6.8	105
		Report: Additional page with Pulse rate and Oximetry histograms	4.6.2	90-97
		WP1 Multi study: register additional nights	4.1.2.2	57-59
		on same patient set-up page and summary report	4.6.9	108-109
		Patient report according to gender	4.6.7	103
		Pack and Send to Itamar: Additional	5.6	111
		Problem type		
		Check status of WP1 registered devices	4.2	64-65
		Search option by Patient Id and/or Device	6.10	123-124
		SN in WP1 download and manage screens		
5	Oct 2022	Change of wording to Intended Use	1.3	10

Table of Contents

1	INTRODUCTION TO THE ZZZPAT10
1.1	Intended Use/ Indications for Use of the Watch-PAT20010
1.2	Intended Use/ Indications for Use of the Watch- PAT200U, WP300, and WP110
1.3	The zzzPAT S/W – Definition10
1.4	Overview11
2	INSTALLATION
2.1	Overall Description of zzzPAT Software13
2.2	Hardware Requirements13
2.3	Software Requirements14
2.4	Installing zzzPAT software14
2.5	Defining the WatchPAT drive (WP200(U) only)21
2.6	Using zzzPAT Software in a secure way22
3	SETTING UP ZZZPAT CONFIGURATION
3.1	Setup>Directories
3.2	Setup>User Settings
3.3	Setup>General Settings
4	USING ZZZPAT
4.1	Preparing a New Study52
4.2	Managing Patient Studies64
4.3	The Display Screen71
4.4	Signal Display Options78
4.5	Review, Analysis and Report Study82
4.6	Reports

5	EXPORTING DATA	
5.1	File>Export Data	
5.2	File>Export Events - Creating *.txt File	
5.3	File>Export Manual Events	
5.4	Tools>Export/Delete	
5.5	Tools>Export General Settings	
5.6	Transferring a Study to Itamar Medical	
5.7	Displaying WatchPAT Device Information (WP200(U)/WP300)	114
5.8	Upgrading WatchPAT Device firmware (WP200(U)/WP300)	114
6	TOOLS	
6.1	Tools>Export/Delete	
6.2	Tools>Import	
6.3	- Tools>Backup	
6.4	Tools>Restore	
6.5	Tools>Move Files to New Location	
6.6	Tools>Export General Settings	
6.7	Tools>Import General Settings	
6.8	Tools>User Administration	
6.9	Tools>Export Activity log	
6.10	For WP1: Tools>Manage WatchPAT One Registered Devices	
7	DATABASE WIZARD	
7.1	Database Tools	
7.2	User Administration	
7.3	Configuration Tools	

8	TROUBLESHOOTING	129
APP	ENDIX A: LICENSE AGREEMENT	134
APP	ENDIX B: TECHNICAL SUPPLEMENT	135
APP	ENDIX C: KEYBOARD SHORTCUTS	140
APP	ENDIX D: REGULATORY REPRESENTATIVE	141
APP	ENDIX E: INDEX	142

List of Figures

	4.5
Figure 1 - zzzPAT installation dialog box	
Figure 2 - zzzPAT License Agreement	
Figure 3 - Customer Information dialog box	
Figure 4 - Database Properties dialog box	
Figure 5 – Database Destination dialog box	
Figure 6 – Choose folder dialog box	
Figure 7 – Application folder dialog box	
Figure 8 – Ready to Install dialog box	
Figure 9 - Select WatchPAT Drive dialog box	
Figure 10- zzzPAT database Wizard dialog box	
Figure 11 - User Administration dialog box	
Figure 12 - User Details dialog box	.25
Figure 13- Set Directories dialog box	. 27
Figure 14 - Save settings for curent user question	. 28
Figure 15- Manage Montage of zzzPAT display	. 28
Figure 16 - Setting colors of zzzPAT display	. 30
Figure 17 - Setting "Options"	
Figure 18 - Showing Segment Information	
Figure 19 - General Settings Setting Dialog Box – Events Tab	
Figure 20 - Save Settings for curent user question	
Figure 21 - General Settings Dialog Box – Clinical Data	
Figure 22 - General Settings Dialog Box – Custom Fields Tab	
Figure 23 - Define Values List Dialog box	
Figure 24 - General Settings Dialog Box – General Options Tab	
Figure 25 – Label example	
Figure 26 - General Settings Dialog Box – Security Tab	
Figure 27 - General Settings Dialog Box – Analysis/Statistics Parameters Tab	
Figure 28 - General Settings Dialog Box – Report Appearance Tab	
Figure 29 - General Settings Dialog Box – Report Appearance Tab Figure 29 - General Settings Dialog Box – Report Translation Tab	
Figure 30 - General Settings Dialog Box – HL7 Configuration	
Figure 30 - General Settings Dialog Box – HL7 Configuration Figure 31 - General Settings Dialog Box – HL7 Configuration/Modify fields dialog	
Figure 32- General Settings Dialog Box – WP Device Configuration	
Figure 33 - Login Dialog Box	
Figure 34 - New Study Dialog Box for WP300	
Figure 35 – WP300 Device Test results Dialog Box	
Figure 36 - WP Not Loaded Dialog Box	
Figure 37 - New Study Termination Question	
Figure 38 - New Study Dialog Box for WP1/WPONE-M	
Figure 39 - New Study Termination Question	
Figure 40 - Demographic Details Dialog box	
Figure 41 - Patient Clinical Data Dialog box	
Figure 42 - More Study Details Dialog box	
Figure 43 – List of WP1's registered patients	
Figure 44 - Loading Study dialog box	
Figure 45 - Select Patient Study dialog box	. 67
Figure 46 - Select Analysis dialog box	. 67

Figure 47 - Select studies options dialog box	68
Figure 48 - zzzPAT display screen	
Figure 49 - Selecting the All night Window	72
Figure 50 - All Night pop-up Menu	73
Figure 51 - Signal Properties - All Night window	74
Figure 52 - All Night pop-up menu	75
Figure 53 - Active channel pop-up menu	76
Figure 54 - zzzPAT Screen - Status Bar	77
Figure 55 - Montage screen	78
Figure 56 - Set y-Scale dialog box	79
Figure 57 - Set y-Scale – per channel dialog box	80
Figure 58 - Change event color	83
Figure 59 - Add Event dialog box	84
Figure 60 - Rigt Click on event	85
Figure 61 - Rigt Click on segment	85
Figure 62 - Event Type Search options	86
Figure 63 - Select Event dialog box	86
Figure 64 – Draw Background of Events	87
Figure 65 – Editing sleep stages	
Figure 66 - Report Toolbar	
Figure 67 - Clinical Diagnosis dialog box	
Figure 68 - New Analysis Warning message	
Figure 69 - Snoring and Body Position Statistics	
Figure 70 - AHI Severity Graph	
Figure 71 - First page of Sleep Report	93
Figure 72 - Second page of Sleep Report	
Figure 73 - Third page of Sleep Report	
Figure 74 - Fourth page of Sleep Report	
Figure 75 – Fifth Page of Sleep Report (Optional)	
Figure 76 - Sleep Report for Selected Time Range Title	
Figure 77 - Event Report	
Figure 78 - Detailed Event Report	
Figure 79 - Sleep Indices report	
Figure 80 - Patient Follow-up Report	
Figure 81 - Report for Patient	
Figure 82 – Sample Page from Detailed Report	
Figure 83 - Multi-night Summary Report	
Figure 84 - Print dialog box	
Figure 85 - Launching Transfer Files	
Figure 86 - Prepare and Send Study Dialog Box	
Figure 87 - List of Problem Types	
Figure 88 - Saving Case Study	
Figure 89 - Sending Study confirmation	
Figure 90 - Data Transfer Progress	
Figure 91 - Data Transfer Successfully Completed	
Figure 92 – Device Information Dialog Box	
Figure 93 - Export Dialog box	
Figure 94 - Delete studies options Dialog box	
1 Gui 77 Detete studies options Dialog von minimum minimum minimum minimum	

Figure 95 - Study Selection Dialog box	. 117
Figure 96 - Import Dialog Box	. 118
Figure 97 - Select Archive Dialog box	. 119
Figure 98 - Restore Dialog box	. 120
Figure 99 – Move Files dialog box	. 121
Figure 100 - Export General Settings Dialog Box	. 122
Figure 101 - Import General Settings Dialog Box	
· · · ·	. 123
Figure 103 – Manage WP1 registered devices window – visible in the list	. 124
Figure 104 – Manage WP1 registered devices window – hidden from the list	
Figure 105 - Database Wizard Login	. 125
Figure 106 - Database Tools Wizard dialog box	. 126
Figure 107 - Configuration Tool Wizard	

List of Tables

Table 1 - User Permissions	26
Table 2 - Troubleshooting, Installation	129
Table 3 - Troubleshooting, zzzPAT	
Table 4 - Troubleshooting, Shared Access Mode zzzPAT	
Table 5 - Troubleshooting, Utilities	

1 Introduction to the zzzPAT

Note: Throughout this document, the references WatchPAT[™], WatchPAT and WP are referring to all 4 devices (WP200, WP200U, WP300 and WP ONE (WP1)) unless specified otherwise

1.1 Intended Use/ Indications for Use of the Watch-PAT200

The Watch-PAT200 (WP200) is non-invasive home care device for use with patients suspected to have sleep related breathing disorders. The WP200 device is used as a diagnostic aid for the detection of sleep-related breathing disorders and sleep staging (Rapid Eye Movement (REM) Sleep, Light Sleep and Deep Sleep). The device generate a peripheral arterial tonometry ("PAT") Respiratory Disturbance Index ("PRDI"), Apnea-Hypopnea index ("PAHI") and PAT sleep staging identification ("PSTAGES") and optional snoring level and body position discrete states from an external integrated snoring and body position (SBP) sensor. The device's "PSTAGES" and SBP provide supplemental information to its PRDI/PAHI. The device's "PSTAGES" and SBP are not intended to be used as the sole or primary basis for diagnosing any sleep related breathing disorder, prescribing treatment, or determining whether additional diagnostic assessment is warranted.

The WP200 is not indicated for children less than 17 years old.

1.2 Intended Use/ Indications for Use of the Watch- PAT200U, WP300, and WP1

The Watch-PAT200U (WP200U), Watch-PAT300 (WP300) and Watch-PATONE (WP1) devices are non-invasive home care device for use with patients suspected to have sleep related breathing disorders. The WP200U, WP300 and WP1 are a diagnostic aid for the detection of sleep related breathing disorders, sleep staging (Rapid Eye Movement (REM) Sleep, Light Sleep, Deep Sleep and Wake), snoring level and body position. The devices generates a peripheral arterial tonometry ("PAT") Respiratory Disturbance Index ("PRDI"), Apnea-Hypopnea index ("PAHI"), Central Apnea-Hypopnea index ("PAHIc"), PAT sleep staging identification (PSTAGES) and optional snoring level and body position discrete states from an external integrated snoring and body position sensor. The device's PSTAGES and snoring level and body position provide supplemental information to its PRDI/PAHI/PAHIc. The device's PSTAGES and snoring level and body position are not intended to be used as the sole or primary basis for diagnosing any sleep related breathing disorder, prescribing treatment, or determining whether additional diagnostic assessment is warranted.

PAHIc is indicated for use in patients 17 years and older. All other parameters are indicated for 12 years and older.

Note: The presentation of pAHIc is subject to regulatory approval in the country

1.3 The zzzPAT S/W – Definition

The zzzPAT is an analysis software package used with the WP devices to aid in diagnosis of sleep related breathing disorders, detects REM, Light Sleep, Deep Sleep and Wake stages and measure snoring intensity and body position states. The zzzPAT

S/W displays the signals recorded by the WP devices, automatically identifies breathing disordered events, sleep stages and snoring and body position data and generates a comprehensive report for the physician.

1.4 Overview

Obstructive sleep apnea syndrome (OSAS) is considered a major public health problem. The prevalence of the syndrome is estimated at 2% to 5% in the adult population. It is characterized by recurrent events of complete or partial obstruction of the upper airways during sleep, often leading to hypoxemia, and/or arousals associated with sympathetic nervous system activation. The diagnosis and assessment of the sleep apnea patient is based on the Respiratory Disturbance Index (RDI), the number of Apneas, Hypopneas and Respiratory Effort Related Arousals (RERA) per hour of sleep and/or apnea-hypopnea index (AHI), along with sleep architecture. The common consequences of this sleep disruption are daytime sleepiness, poor daytime performance and increased vulnerability to accidents. Cardiovascular complications such as systemic/pulmonary hypertension, ischemic heart disease and arrhythmias are the major sequel of OSAS in the adult population.

The WP device is worn on the wrist and utilizes a plethysmographic based fingermounted probe, to measure the PAT (Peripheral Arterial Tone) signal. The PAT signal is a measurement of the pulsatile volume changes in the fingertip arteries which reflects the relative state of the arterial vasomotor activity, and thus indirectly the level of sympathetic activation. Peripheral arterial vasoconstriction, which mirrors sympathetic activation, is shown as attenuation in the PAT signal amplitude. The PAT signal is recorded continuously and stored in the device along with pulse rate (derived from the PAT signal), together with either data from oximetry channels integrated into the uPAT probe (WP200U/WP300/WP1), or from a built-in pulse-oximetry sensor (WP200) and an actigraph (embedded in the device). Snoring and Body Position signals are generated from the SBP/RESBP integrated sensor. The RESBP (Respiratory Effort Snoring and Body Position) sensor records the subject's chest movement signal in addition to the snoring and body position signals that are included with the SBP sensor. Following the sleep study, in an offline procedure, the recordings are automatically downloaded and analyzed using the proprietary zzzPAT software.

The zzzPAT algorithms use the four WP channels: PAT, Pulse Rate, actigraphy and Oxygen saturation (either recorded in WP200 or derived in WP200U/WP300/WP1) for the detection of sleep related breathing disorders and sleep staging (Rapid Eye Movement (REM), Light Sleep, Deep Sleep and Wake). In WP200U, WP300 and WP1, using the RESBP's respiratory movement channel in addition to the other WP channels, allows further identification of central apnea.

The zzzPAT uses WP's snoring and body position channels to generate snoring level and body position discrete states. The software issues comprehensive reports of the study, with statistics and graphic presentation of the results. The whole night data can be viewed and the automatically detected events can be revised manually.

Note: Throughout this document, the Snore and Body Position sensor is referring to both SBP sensor and RESBP sensor unless specified otherwise. In WP200(U)/WP300, the use of Snore and Body Position sensor is optional and according to physician preference. The

use of RESBP sensor is subject to regulatory approval in the country.

This manual provides the information necessary for routine use of the zzzPAT software.

Restrictions

The tracings and calculations provided by the WP systems are intended as an aid for Sleep Breathing Disorders diagnosis. They are explicitly not to be regarded as a sole incontrovertible basis for clinical diagnosis.

- The zzzPAT software should be used only on compatible computers that meet the requirements specified in this document.
- Running other programs, commercial or customized, simultaneously with the zzzPAT may interfere with its proper function.
- Sleep/Hibernate features should be disabled in case of shared database configuration.

2 Installation

2.1 Overall Description of zzzPAT Software

zzzPAT is a proprietary PC software developed specifically for managing and analyzing data recorded by the WP device. The software displays and stores the recorded signals, and provides a set of analytical functions for interpretation purposes.


In WP200(U)/WP300 a USB cable is used in order to read the data recorded by the WP on the internal memory card. The zzzPAT S/W automatically detects the data on the internal memory card once the WP is connected to the PC via the USB communication cable. In WP1, an internet connection is used in order to read the data recorded by the WP1. The zzzPAT S/W automatically opens a list with all registered patients that the study was not yet retrieved from the web server.

Please refer to Section 2.2 for required PC configuration.

zzzPAT can operate in two modes:

Standalone - for use on a single PC with a local database.

Shared Access - for use in a networked environment where multiple zzzPAT stations access a single, shared database.

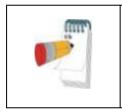

Note

It is strongly advised to coordinate the setting of **shared access** zzzPAT operation mode with an Itamar Medical representative. Extra training is crucial for proper operation.

The zzzPAT mode of operation is determined during installation as further described in the Installation section of this operation manual.

2.2 Hardware Requirements

Hardware configuration:

Computer Pentium 4 3GHz or higher 1 available USB port (WP200(U)/WP300) XGA screen resolution (minimum 1024 x 768 pixels) RAM 1GB or higher Access to the local Network (WP1)

Disk space requirements:

Standalone installation

 10GB minimum / 60GB recommended disk space on Files folder and at least 1.2GB on boot drive

Shared installation

 SQL DB drive – 1.2GB if using our default MS SQL Express installation and enough for 1 year worth of studies (500 KB / study). Shared Files folder for raw data signal files - enough for 1 year worth of studies (30 MB / study).

2.3 Software Requirements

Operating System supported:

Windows Server 2008 Service Pack 1 and above Windows 8 Windows 10

MS SQL Server supported:


MS SQL Express 2017 (default) MS SQL Server 2008 Standard/Enterprise (existing installation) MS SQL Server 2012 Standard/Enterprise (existing installation) MS SQL Server 2016 Standard/Enterprise (existing installation)

Microsoft .NET Framework required:

Microsoft .NET Framework 2 Microsoft .NET Framework 4

Hotfixes required:

Hotfix KB2999226 (Universal CRT) must be installed before the installation can be begin.

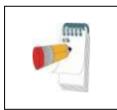

Note

Software components needed for the proper installation of zzzPAT should be installed prior to zzzPAT installation (OS service packs or latest OS updates).

2.4 Installing zzzPAT software

zzzPAT can be installed either by CD or by downloading from Itamar website (itamarmedical.com).

This process will install a single Icon on the desktop, which opens the zzzPAT analysis and reporting software.

Quick start instructions: CD installation:

- Insert the zzzPAT CD into the computer's CD-ROM drive.
- If the CD-ROM drive is set to run automatically, the installation will begin automatically. If installation does not begin, refer to the section below.
- Follow the instructions on the screen.

If the installation does not start automatically:

- Double-click the My Computer icon on your desktop.
- Double-click the CD-ROM icon labeled zzzPAT.
- Double-click 'Setup.exe'.
- Follow the instructions on the screen.

Web Download:

• Go to Itamar website (itamar-medical.com) and download the zzzPAT software. In case of any issues with the download or installation process, please refer to the Clinical Account Specialist representative in your area.

You are now ready to run the zzzPAT software. See section <u>Using zzzPAT Software in a</u> <u>secure way</u>.

2.4.1 Step by step installation instructions

Prior to installation, verify that you are in full system administrator mode with full privileges.

1. Insert the zzzPAT installation disk into the CD drive. The installation program is activated, and the following dialog box appears. If the installation does not start automatically refer to section 2.4.



Figure 1 - zzzPAT installation dialog box

2. Click **Next**. The License Agreement appears.



Figure 2 - zzzPAT License Agreement

3. After reading the agreement, select **I accept the terms in the license agreement** and click **Next** to continue.

討 zzzPAT - InstallShield Wizard		×
Customer Information Please enter your information.		6
User Name:		
Organization:		
InstallShield	< Back Next >	Cancel

Figure 3 - Customer Information dialog box

4. Enter user name, company name and Click **Next** to continue. The 'Database Properties' dialog box appears.



Figure 4 - Database Properties dialog box

2.4.2 Installations types

The zzzPAT software can be installed as a standalone or as a client/server configuration.

- "Standalone" intended for users that do not require sharing that database with other zzzPAT stations.
- "Client/Server" intended for network installations with multiple zzzPAT stations accessing a central zzzPAT database over the network. If you wish to install this mode it is strongly advised to coordinate the installation with an Itamar Medical representative. Extra training is crucial for proper operation.
 - The "Server" mode is selected **only once prior** to installing the "Clients". This installation should be done on the dedicated computer which will serve as the server for the zzzPAT clients.
 - The "Client" mode is selected **only after** installing the "Server" and is **repeated for all clients**.

2.4.2.1 Standard - Standalone installation only

The zzzPAT Installer will assist in selecting a destination location for the zzzPAT database (signal files). If the necessary folder does not exist, zzzPAT Installer will create the folder.

Databas	e Destination
Click Ne	ext to install to this folder, or click Change to install to a different folder.
P	Install zzzPAT database to:
-	c:\tamar medical\zzzpat\database\ Change
	' s folder will hold the data files (50MB per study). Please ensure that e is enough disk space for your future studies.

Figure 5 – Database Destination dialog box

If you wish to change the default folder, select the Change button. The 'Choose Folder' dialog appears.

Choose Folder
Please choose the database folder DO NOT USE MAPPED DRIVES!!!
c:\jtamar medical\zzzpat\database
Directories:
OK Cancel

Figure 6 – Choose folder dialog box

Make sure you are not selecting a mapped drive. Select **OK** to Proceed.

The zzzPAT Installer will assist in selecting a destination location for the zzzPAT application files. If you wish to change the default folder, select the Change button.

🛃 zzzPAT -	InstallShield Wizard
	on Folder kt to install to this folder, or click Change to install to a different folder.
This fe	Install zzzPAT to: C:\Program Files (x86)\Itamar medical\zzzPAT\ Change blder will hold all zzzPAT components (programs, utilities, help files, etc.)
InstallShield -	< Back Next > Cancel

Figure 7 – Application folder dialog box

The last dialog before standalone installations summarizes your selections. After reading this summary, press **Install** to begin the installation.

😸 zzzPAT - InstallShield Wizard	x	
Ready to Install the Program The wizard is ready to begin installation.	5	
Click Install to begin the installation. If you want to review or change any of your installation settings, dick Back. Click Cancel to exit the wizard.		
InstallShield < Back Cancel		

Figure 8 – Ready to Install dialog box

At the end of the installation you should restart your computer.

Note
A login name and password are needed to operate this software. The default administrator login information is: Login name: 111 Login password: 111 To modify the administrator login name and password, and to add/modify additional users, refer to section 2.6.

	Warning
<u>/!</u>	 Do not attempt to modify the zzzPAT software. It is recommended to close other programs when using zzzPAT to prevent possible conflicts.

For any other installation except the Standalone please refer to the **Installation guide.pdf** (can be found in the installation CD).

2.4.3 Upgrading from previous versions

2.4.3.1 Upgrading using the Patch program

You can find the Patch program inside the installation CD or in <u>http://www.itamar-medical.com</u> web site under the Support>Downloads section. The Patch program will update your software to the latest version.

2.4.3.2 Upgrading a Standalone Installation


im	Note
	Uninstalling zzzPAT does not remove the zzzPAT database, however, it is strongly recommended to backup the database before upgrading zzzPAT.

- Backup the entire database of the existing zzzPAT installation (see Database Wizard, Backup section 7.1.3).
- Uninstall the previous version using 'Add/Remove Programs' applet from the control panel.
- Select 'Setup.exe' from the zzzPAT installation CD-ROM to start the zzzPAT installation program.
- Follow stages in section 2.4.1.

2.4.3.3 Upgrading a Shared Database Installation

Use the Patch program to upgrade the clients unless you need to uninstall and re-install from the beginning.

Activating the Shared Database installation on an older version zzzPAT station will update the zzzPAT software only – the database will not be updated through the installer and will require a separate upgrade process that is done automatocally during the first time zzzPAT is launched after the upgrade.

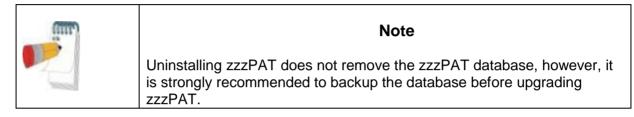

- Backup the entire database of the existing zzzPAT installation (see Database Wizard, Backup section 7.1.3).
- Uninstall the previous zzzPAT version using 'Add/Remove Programs' applet from the control panel.
- Select 'Setup.exe' from the zzzPAT installation CD-ROM to start the zzzPAT installation program.
- Follow stages in section 2.4.1.

2.5 Defining the WatchPAT drive (WP200(U) only)

To prepare a new study (**File>New Study Details** from zzzPAT), you must first define the WatchPAT drive.

Connect the device to the PC's USB drive using the supplied USB cable.

- Click **File>New Study Details** in zzzPAT. A dialog box will open (see Figure 34).
- Click **Set WatchPAT Drive** in the 'New Study' dialog box. The following window will appear:

Select WatchPAT Drive	
Select WatchPAT Drive:	ОК
	Cancel
	Show All Drives


Figure 9 - Select WatchPAT Drive dialog box

• The WP200(U) device is identified by the zzzPAT, and is listed in this window. Click **OK** to confirm the drive selection.

	Note
(IIII)	If you have more that one WP300 connected to the computer at the same time you will get an error. Additionally, if you try to access the device for New Study while both WP200(U) and WP300 are connected you will receive the following error:
	New Study-Error
	WP300 and WP200 devices are connected. Please verify that just 1 WP device is connected at a time.
	ОК

2.6 Using zzzPAT Software in a secure way

In order to maintain the security of the information stored by the zzzPAT, it is recommended that you set new administration Login and Password, replacing the default setting.

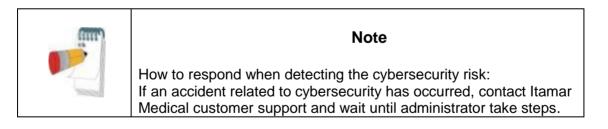

2.6.1 Setting new login and Password for the zzzPAT Administrator

You can either use the Active Directory Service option to define the users or you can use the User Administration functionality as described below:

- Select **Tools>User Administration** from zzzPAT or navigate through the installation folder of zzzPAT to the dltools.exe from the BIN folder.
- Log in using the system default settings (in Shared Access mode this process needs to be repeated twice – once for the shared access database and once for the local database

- Login: 111
- Password: 111
- A dialog box will open Figure 10 for **dbtools.exe** Figure 11 User Administration dialog box or **Tools>User Administration**.

😰 zzzPAT Database V	Vizard		×
tramar medical		Database Tools User Administration Configuration Tools	
DB Connection: Loca	i		Exit

Figure 10- zzzPAT database Wizard dialog box

- Click 'User Administration' icon
 Image: The 'User Administration' dialog box will open.
- Select 'Super User'.
- Click Edit User.
- Change the default Login from 111 to the new login.
- Change the default Password from 111 to a new password.
- Update the personal information.
- Click Save User.



Note

In Shared Access mode changing the administrator login for the shared access database needs to be performed only once. Changes to the local database administrator login needs to be changed in each of the zzzPAT stations.

2.6.2 Setting a New user

Setting new user Login and Password and defining the permissions allowed for each user operating the zzzPAT.

If the users are defined by ADSI see 3.3.4 for instructions on how to set new users and define permissions.

- Launch the 'Database Wizard' and Click 'User Administration' icon select Tools>User Administration from zzzPAT.
- The following dialog box will open:

U	User Administration						
Γ	All Users						
L.	First Name	Last Name	ID	Login	Title	Status	Created by
L.	John	Smith	1	1	d	Active	zzzPAT
	Super Jonathan	User Lee	111 123	111 jonathan		Active Active	zzzPAT zzzPAT
	Deac	tivate User	Activate User	Add U	ser	Edit User	
	DB Connection:	Local					Exit

Figure 11 - User Administration dialog box

 Note
The column "Created by" denotes if the user was defined by zzzPAT or was defined by Active Directory service (in this case "Created by" is ADSI). "Activate User" / "Deactivate User" will not be functional for user that was defined by Active Directory service.

• Click Add User. The following dialog box will open:

User Details		×
User Details <u>F</u> irst Name: <u>I</u> D: <u>L</u> ogin: <u>P</u> assword: <u>Co</u> nfirm:	Last <u>N</u> ame:	<u>S</u> ave User Canc <u>e</u> l <u>C</u> lear all Fields Clear Signature
Digital Signature:	<u>B</u> rowse	
Extended Permissions		
Patient ID Modification	Database <u>M</u> aintenance	
Finalizing <u>R</u> eports	User Administrator	

Figure 12 - User Details dialog box


- User Details: Fill all user details fields. Demographics information is optional.
- Set Digital Signature file to be saved in the report (when report is set as reported). Use the Browse button in order to select signature image file (jpg or png).
- **Extended Permissions:** This section defines the functions each user is allowed to perform with the zzzPAT (see Table 1).
- Extended Permissions for users defined by Active Directory service and set to be defined by ADSI cannot be set in zzzPAT (see 3.3.4).

Extended Permissions	Allowed activities	
No extended permissions	Basic User:	
selected	 Prepare a new study 	
	 Load study into zzzPAT 	
	Issue reports	
Patient ID Modification	Basic User plus:	
	 Modify a patient's ID 	
Finalizing report	Basic User plus:	
	 Finalizing the report. This action locks the 	
	current analysis, so that no further changes can	
	be made to the analysis and report.	
Database Maintenance	Basic User plus:	
	 Database export, import, delete, backup, restore 	
	and upgrade functions. (See section 5)	
	Customizing capabilities for the General Settings	
User Administrator	Basic User plus:	
	 Adding users to zzzPAT and defining their 	
	permissions	

Table 1 - User Permissions

2.6.3 Deactivating a user:

- Click on the User's name in the 'User Administration' dialog box.
- Click Deactivate User.

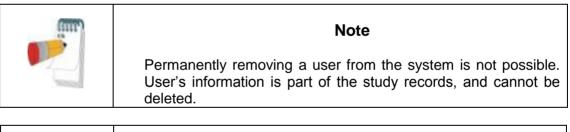



Note

The original user (originally named "Super User") cannot be deactivated. It can only be modified with a new name and password to ensure continued access with System Administrator's privileges. The change of the password of Super user should be handled with care.

3 Setting Up zzzPAT Configuration

3.1 Setup>Directories

Displays the zzzPAT working directory, the name of the currently connected database, the files directory (signal files data) and the USB drive.


Directories Setup		X
Working Directory	\Program Files (x86)\Itamar medical\zzzPAT\Data\	ОК
Database	LDIANA-SSD-LTVZZZSQL	DB Logged Users
Files Directory	C:\temp\	
USB Drive		

Figure 13- Set Directories dialog box

Select "DB Logged Users" in order to see who is connected to the shared database. The list will contain all computers' names that use the same shared database and have the zzzPAT application up and running.

3.2 Setup>User Settings

The user can change the following Setup parameters by opening the 'User Settings' dialog box from **Setup>User Settings**. 'User Settings' setup parameters are stored in the zzzPAT database for each user (either the local database in a standalone installation or the shared access database in a Shared Access installation).

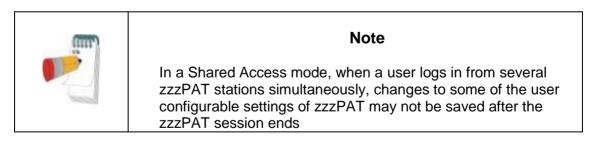

When all settings changes are completed, click **OK** to close the Settings dialog box. The following dialog box will open:

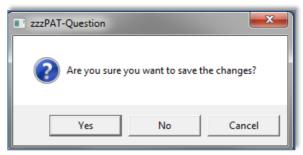


Figure 14 - Save settings for curent user question

Clicking **Yes** will save the changes. Clicking **No** will exit 'User Settings' Dialog Box. Clicking **Cancel** will return the last displayed dialog box.

3.2.1 Setup>User Settings>Manage Montage

A montage defines which channels will be presented on the zzzPAT screen, in what order and size, which channel is displayed in "All Night" window and other parameters.


A number of montages may be defined and used according to the specific needs. The selection of the current montage can be easily done through the main screen's toolbar (see 4.3).

User Settings	×
Manage Montage Colors Options	
Current Montage Itamar Montage Save Save as	
Montage List: Itamar Montage(Default) Set as Default	
OK Cancel Help	

Figure 15- Manage Montage of zzzPAT display

To Save the current montage:

• Click the "Save" option.

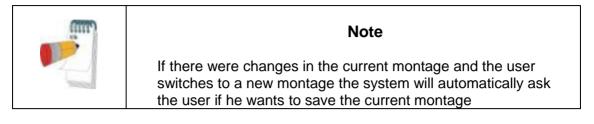

To create new montage:

• Click on "Save as..." to give a name to the new montage. The current montage will be saved to a new montage and can be edited (see 4.3.1 and 4.4).

To delete a montage:

• Select the montage and click on "Delete".

	Note				
	The Itamar montage cannot be modified or deleted. If you make a change and try to save it the following dialog will be opened:				
m	ZZZPAT-Error				
	Itamar montage can't be modified. Press 'Yes' to save the changes as another montage Press 'No' to cancel the changes.				
	Yes No				
	If you press YES the montage will be saved as a new montage.				

To set a montage as the default montage:

• Select the montage and click on "Set as Default". This montage will be used from now on when zzzPAT is opened.

3.2.2 Setup>User Settings>Colors

Colors Setup page (Figure 16) consists of a list of all the display elements that can be colored, allowing the user to personalize the screens. The example area provides a preview of the selected color scheme.

To change a color:

- Select a display element from the list.
- Click the **Change Color** button to select a new color. The preview is displayed in the 'Example' area.

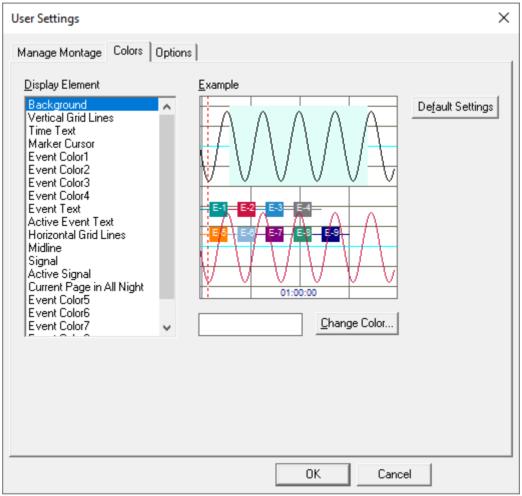


Figure 16 - Setting colors of zzzPAT display

· · · · · ·	Note
	Be careful not to choose a color that blends into one of the background elements

3.2.3 Setup>User Settings>Options

Open report - defines if and when the 'Sleep Report' or 'Sleep Indices' report is displayed automatically.


User Settings	
Manage Montage Colors Options	
Open report after ''Load Study and Analyze''	
C "Sjeep Report" report	
Dpen report after "Open Study"	
C "Sleep <u>R</u> eport" report	
User defined AHI according to :	
G <u>3</u> % (ASDA) C <u>4</u> % (Medicare)	
Enable Manual Scoring	nual Events
Show Segment Information	
OK Cancel	Help

Figure 17 - Setting "Options"

You can select to **open a report** either after "Load Study and Analyze" by selecting 'Open report after "Load Study and Analyze" or/and for existing study by selecting 'Open report after "Open Study". Both options contain radio-buttons that enable you to select between Sleep Report and Sleep Indices report.

- When selecting "Sleep Report" report, the sleep report opens automatically.
- When selecting "Sleep Indices" report, the sleep indices report appears automatically on display, and the sleep report opens upon command.

You can select **ASDA or Medicare** protocol for calculating pAHI and pRDI indices on a user level, see 3.3.6.

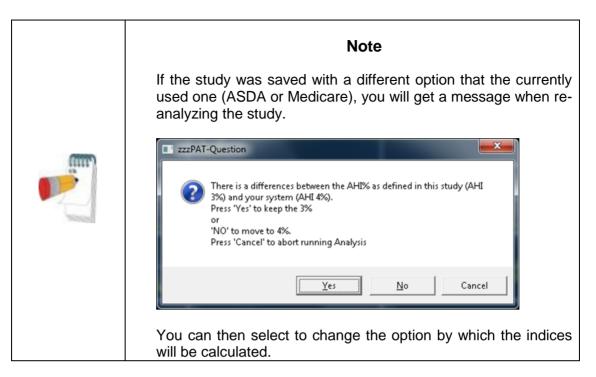

You can select "**Enable Manual Scoring**" if you want to be able to edit manually the events and sleep stages (see 4.5.8).

You can select "**Enable Export Manual Events**" if you want to be able to export manual event changes (used by sleep specialists working with CloudPAT web service).

You can select "**Show Segment Information**" if you want to automatically see the information of the selected segment's active signal, i.e. begin, end and duration of segment as well as maximum and minimum values when relevant.

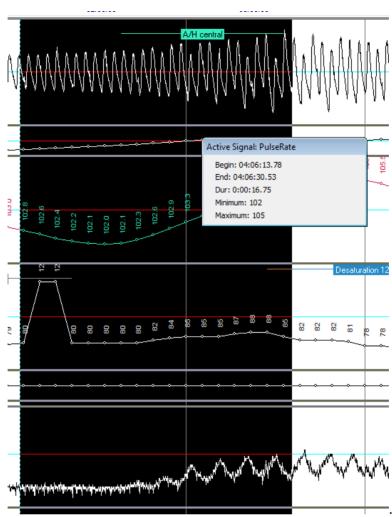


Figure 18 - Showing Segment Information

3.3 Setup>General Settings

Only a user, with 'User Administration' permission, can change the following Setup parameters by opening the 'General Settings' dialog box from **Setup>General Settings**. 'General Settings' are stored in the zzzPAT database (either the local database in a standalone installation or the shared access database in a Shared Access installation).

These settings are global. Modified settings become available to all users.

Report Appearan			Configuration WP Device Configuration
Events Clinic	al Data Custom Fields:	General Options	s Security Analysis/Statistics Paramete
Event	Long Name	Short Name	▲ <u>M</u> odify
A/H central	A/H central	A/H cent	
A/H obstructive	A/H obstructive	A/H obst	Add
A/H unclassified	A/H unclassified	A/H uncl	
Wake	Wake	Wake	Remove
Desaturation	Desaturation	Desat.	
Push Button	Push Button	Push B.	Undo Remove
Excluded	Excluded	Excld.	Undo Remove
Resp. Event	Resp. Event	R.E.	the second secon
REM	REM	REM	Default Settings
L.Sleep	Light Sleep	L.Sleep	E
D.Sleep	Deep Sleep	D.Sleep	
Accept Signal	Accept Signal	Acpt.Sig	
Reject Signal	Reject Signal	Rjet.Sig	
Sleep	Sleep	Sleep	
CSR	CSR	CSR	
Apnea	Apnea	Apnea	
Hypopnea	Hypopnea	Hypopnea	
Movement	Movement S2	Movement	
S2 S4	52 S4	S2 S4	
54 MT	54 MT	54 MT	
M I Light-Off	M I Light-Off	L. Off	
Light-On	Light-On	L. On	
Lignt-On Leg Movement	Lignt-Un Lea Movement	L. Un Leg.M.	
PI M	PI M	PLM	·
Display Long National States 1	ame C Displa	y <u>S</u> hort Name	

Figure 19 - General Settings Setting Dialog Box – Events Tab

When all settings changes are completed, click **OK** to close the General Settings dialog box. The following dialog box will open:

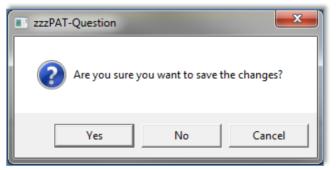


Figure 20 - Save Settings for curent user question

Clicking **Yes** will save the changes. Clicking **No** will exit 'General Settings' dialog box. Clicking **Cancel** will return the last displayed dialog box.

3.3.1 Setup>General Settings>Events

A list of Event types is listed in the events tab (Figure 19). The list includes event type, long name, short name and category for each event.

To Modify an Event in the Settings list:

- Click on the Event in the list box. The **Modify** and **Default Setting** buttons become enabled.
- Click the **Modify** button. 'Modify Event Definition' dialog opens.
- Enter the required names in the fields.
- Click 'OK' to confirm and exit.

Default Settings reverts to the default event list.



Note Clicking the **Default Settings** will revert to default settings losing all user-defined settings.

3.3.2 Setup>General Settings>Clinical Data Medical History

A preliminary list of medical conditions, medications, clinical diagnoses and clinical recommendations are listed in the Clinical Data tab while the selected type is selected in the "Predefined Clinical List" pull down menu (Figure 21).

neral Settings		×
Report Appearance Events Clinical D	Report Translation HL7 Configuratio	n WP Device Configuration surity Analysis/Statistics Parameters
	al History, Medications, Clinical Diagnosis and Clinical Recor from the drop-down list below in order to view the items belo Medical History Medical History	
Medical History: Anxiety disorder Day time sleepiness	Medications Dinical Diagnosis Clinical Recommendations	<u>M</u> odify
Depression Frequent arousals Frequent arousals gasp Ischemic heart disease Morbid obesity Morning headaches	ng for air	E E Default Settings
Narcolepsy Nocturia None Obesity Osteoporosis Peptic ulcer		
PLMD RLS Sobiooptropio		-
Add to List		Add
	ОК	Cancel Help

Figure 21 - General Settings Dialog Box – Clinical Data

To add a description to the list:

- Type the description in the 'Add to List' box. The **Add** button becomes enabled.
- Click Add.

To modify a description:

- Click on the description.
- Click **Modify**. The 'Modify' dialog box opens with the selected description name.
- Type in the desired changes.

To remove a description from the list:

- Click on the description.
- Click Remove.
- Click **OK** or **Apply**.

Default Settings reverts to the default list.



Note

Clicking the **Default Settings** will revert to default settings losing all user defined setting.

3.3.3 Setup>General Settings>Custom Fields

Up to three 'Custom Fields' can be defined and named by the user. These 'Custom Fields' can be used to organize your studies in the 'Select Patient Study' dialog box (Figure 45) and during the Export/Import process. These custom fields are available to all users.

Report Appearance Rep	oort Translation	HL7 Conf	iguration	WP Device Configuration
Events Clinical Data Cus	tom Fields	General Options	Security	Analysis/Statistics Parameters
✓ Use Custom Field <u>1</u> Named As:	Case Number	∐alues		
☑ Use Custom Field <u>2</u> Named As:	Site	V <u>a</u> lues		
▼ Use Custom Field <u>3</u> Named As:	SW version	Vajues		

Figure 22 - General Settings Dialog Box – Custom Fields Tab

To enable a custom field:

- Check the box to the left of the custom field. The **Values** button becomes enabled.
- Write the title of your choice for this field in the text box (for example CHF).
- Click the Values button, the following dialog box will open:

Values For Medical Ast. List of Values OK Value to Add Add> FC1 FC2 FC3 Cancel FC4 Remove < Clear List	Define Values List	man 1	×
	Value to Add Add>	FC1 FC2	Cancel


Figure 23 - Define Values List Dialog box

To insert a value into the 'List of Values' field:

- Write a value you wish to insert into the 'Value to add' text box, the **Add** and **Remove** buttons will become enabled.
- Click Add.

To delete a value from the list of values you created:

- Click on the value and click **Remove**.
- Click **OK** to save settings.

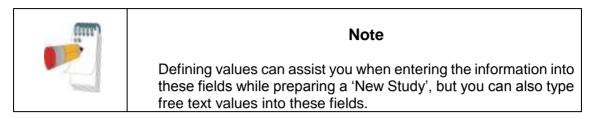

3.3.4 Setup>General Settings>General Options

Used to set up reminders to backup the database, alerts when available disk space is below 1GB and in order to enable features that not enabled by default (import packed studies, multi-night studies (WP200(U),WP300), tamper-proof testing (WP200(U),WP300) and login options).

Report Appearance	Report Translation	HL7 Configuration WP Device Configuration
Events Clinical Da	ata Custom Fields	General Options Security Analysis/Statistics Paramet
Prompt for backup	every 30 days 🗖 Pr	rompt for <u>a</u> rchive when disk space less than 1GB (~60 studies)
🔽 Enable Automatic	Database Backups 🔽 Er	nable Packed Studies Import
Tamper-Proof Brace	let	
· ·	Proof Testing'' option	Default Set to On 📀 Default Set to Off
Multiple Nights		
Enable "Multiple	Nights" option	hts (8 hr per night and no need to charge between nights)
	O Up to <u>3</u> nigl	hts (10 hr per night and need to charge between nights)
Login		
Enable Active Dir	rectory Service (ADSI) Login	zzzPAT Users Group:
Define Exten	nded Permissions Using ADSI	
	-	
	ication Group:	Database Maintenance Group: zzzPATDB
Patient <u>I</u> D Modifi	·)	
Patient <u>I</u> D Modifi Finalizing <u>R</u> eport	s Group:	User Administrator <u>G</u> roup:
-		
Finalizing <u>R</u> eport		
Finalizing <u>R</u> eport	Database Login	User Administrator <u>G</u> roup:
Finalizing <u>R</u> eport	Database Login	
Finalizing <u>R</u> eport	Database Login	User Administrator <u>G</u> roup:
Finalizing <u>R</u> eport	Database Login	User Administrator <u>G</u> roup:

Figure 24 - General Settings Dialog Box – General Options Tab

• Check "Prompt for backup every ..." and set the numbers of days to get a reminder message after the last backup of the database was performed.

- Check "Enable Automatic Database Backups" in order to automatically make a backup of the database files when exiting zzzPAT (see Tools>Backup). The process of backing is very quick (no raw data files are backed up) and it is recommended to leave the option checked.
- Check "Prompt for archive when disk space ..." to get an alert message when available disk space is lower than 1GB.
- Check "Enable Packed Studies Import" if you have access to packed studies from ftp uploads. By enabling this option you can import studies that were uploaded by ftp by selecting Tools>Import Packed studies option from the main menu.
- WP200(U)/WP300: Enable "Tamper-Proof Testing" if you want to use the Patient Identification Bracelet. By enabling this option you can select to use the bracelet while preparing a New study (see Preparing a New Study).
 - Select "Default Set to On" if you want to have the "Tamper-Proof Testing" option selected by default.
 - Select "Default Set to Off" if you do not want to have the "Tamper-Proof Testing" option selected by default.
- WP200(U)/WP300/WP1: Check "Enable Multiple Nights" option if you want to enable the WP recording up to 3 nights in a row while preparing a New study (see Preparing a New Study). There are two options for Multi Nights study:
 - WP200(U) only: Up to two nights with no charging needed between the nights (maximum length of study is approximately 8 hours)
 - Up to three nights when charging is necessary between the nights (maximum length of study is approximately 10 hours).
 - The WP200(U)'s power supply must be supplied to the patient.
 - Extra batteries must be supplied in case of WP300 and WP1.



Note


The "Enable Tamper-Proof Testing" and "Enable Multiple Nights" options are available only when the WP internal S/W is version 2.2182 and higher.

- Select "Enable Active Directory Service (ADSI) Login" if you wish to use the Active Directory Service for login to zzzPAT (using your domain name and password). The zzzPAT users group name must be defined if this option is selected
- Select "Define Extended permissions Using ADSI" check box if you wish to control user's extended permissions by ADSI and not zzzPAT database. zzzPAT has 4 internal groups for permissions (see 2.6.2).

• Check "Print label during New Study" and set the label Printer and the number of labels if you wish to have labels automatically printed when you prepare "New Study Details". This feature will work only with pre-defined label printers. The printers must print at 3 secs/label at a minimum and must be defined by Itamar Medical. The information printed on the label (2x5 cm) will include the following fields: Patient ID, Patient First and Last name, Date of Compact Flash preparation and Coverage (field that may be used for additional data such as device #, etc.). The labels may be attached to the WP suitcases as to provide easy identification of the designated patient after the WP device have been initialized with the patient details.

<u> (</u>	1
ID:012345678901234]
Name:John Smith	2
Date: 12/12/05	2
1	1
8	2

Figure 25 – Label example

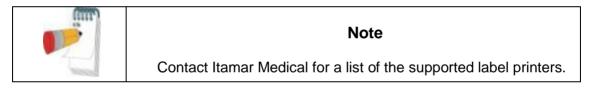

3.3.5 Setup>General Settings>Security

Used to enforce the Security of the application by allowing the user to define one or more of the following:

- Check Protect Data with Password option in order to ensure that all the studies are protected by encryption. This means that only authorized personnel can have access to the study and all export of information must be done through the zzzPAT application.
- Check Automatic zzzPAT Screen-Lock with Login window every 10/20/30/40/50/60 minutes option in order to lock the users screen after the defined amount of time. The user will be able to re-login to the application by inserting the password.
- Check Strong password (8-20) characters, lower/upper case/digits/nonalphanumeric option in order to force the use of stronger and more secure passwords by the users of the application. When this option is checked, every new user in the system will be asked to create a strong password, i.e:
 - at least 8 characters long
 - containing at least one character from 3 out of the following 4 groups:
 - Lowercase letters
 - Upper case letters
 - o digits (0-9)
 - non-alphanumeric symbols (`~! @ # \$ % ^ &, etc.)

• Check **Password will expire in X days** (X between 1 to 365) option in order to define the time after which the user will be prompted to replace the password. The new password must be different than the last 3 passwords.

Note
It is recommended to keep the options below. Strong password and password expiration option will not affect users logging using Active Directory Service (ADSI). If password is expired, the access is disabled. After XX time, the application locks. Password should meet the defined regulation. No trace of the personal data is left.

- Check **Use Patient ID only** option in order to omit the patient information other that the ID to be stored on the device upon device initialization. If this option is checked you must use the same database for uploading the study in order to retrieve patient information (other that the ID) entered upon device initialization.
- Check **Use a new System generated ID only** option in order to have completely anonymous data stored on the device upon device initialization. The system will generate a unique number that will be used instead of the entered patient ID. If this option is checked you must use the same database for uploading the study, otherwise the upload will fail.

Itamar Medical Ltd.

ieral Settings
Report Appearance Report Translation HL7 Configuration WP Device Configuration Events Clinical Data Custom Fields General Options Security Analysis/Statistics Parameters
<u>P</u> rotect Signals Files with Password <u>A</u> utomatic zzzPAT Screen-Lock with Login Window every 10 minutes
zzzPAT Login Password Policy Strong Password (8-20 characters,lower/upper case/digits/non-alphanumeric) Password will Expire in: 90 day(s)
Patient Information Security Default settings: Patient information inserted during device initiation is stored on the device.
You may opt out by selecting from the following settings. Image: Use patient ID only. • Patient info will no longer be stored on the device.
 Full patient data retrieval contingent upon use of the same DB (shared or local), used for device preparation. Use a new System generated ID
 Unique System ID will be anonymous and non-identifiable and will not correspond with other pre-existing patient identifiers. Note: The study cannot be uploaded in cases where device initiation and upload are performed using two different DBs.
OK Cancel Help

Figure 26 - General Settings Dialog Box – Security Tab

3.3.6 Setup>General Settings>Analysis/Statistics Parameters

Used to modify parameters that affect analysis or report.

		1 1170 0	. 1 .	
	Report Translation			VP Device Configuration alysis/Statistics Parameters
Events Clinical Data C	Custom Fields	General Options S	ecurity An	alysis/ Statistics Farameters
Snoring Threshold: 45 dB				
AHI Index according to:	• <u>3</u> % (ASDA)	С <u>4</u> % (Medicare) 🔽		values in the
ODI Index according to:	C 3%	• 4%	3% report	
Oxygen Saturation (in Report):	€ <=88	○ <88		
⊢ AHI Severity Graph———				
	_ 5	- 15 30 _	>	
(Use AASM values Norm	al Mild	- 15 <u>Moderate</u> 30 —	Severe	
C Manually Defined:	5 Mid	- 15 30	Savara >	
Nom		Moderate	Jevele	

Figure 27 - General Settings Dialog Box – Analysis/Statistics Parameters Tab

- Select snoring threshold to be used in calculating snoring statistics (3rd page of report).
- Select the ASDA or Medicare protocol for calculating pAHI and pRDI indices on a site level. The automatic analysis will calculate respiratory events for pAHI and pRDI indices using desaturations of 3% and above (ASDA definition) or desaturations of 4% and above (Medicare definition) for all studies for this site. The Sleep report will show the calculated pAHI and pRDI indices with a note to indicate the protocol used for the calculation. Mark V in order to include both 3% and 4% results within the report.

Note

The default settings to use ASDA or Medicare protocol for pAHI/pRDI calculation is according to site level and is defined in General Setting (Analysis/Statistics Parameters). A user may change this protocol (for his use only) by selecting one of the protocols under 'User defined AHI according to' checkbox in User settings. A user may change this protocol for specific study during new study preparation by selecting one of the protocols under 'AHI index according to' in Study details screen. In order to apply this change on existing studies in the database, you need to rerun the analysis.

	Note			
	NOLE			
	If the study was saved with a different option that the currently used one (ASDA or Medicare), you will get a message when re- analyzing the study.			
(1117)	zzzPAT-Question			
	 There is a differences between the AHI% as defined in this study (AHI 3%) and your system (AHI 4%). Press 'Yes' to keep the 3% or 'NO' to move to 4%. Press 'Cancel' to abort running Analysis 			
	Yes No Cancel			
	You can then select to change the option by which the indices will be calculated.			

- Select the ODI index value to calculate ODI indices on a site level. The automatic analysis will calculate events for ODI indices using desaturations of 3% and above or desaturations of 4% and above for all studies for this site. The Sleep report will show the calculated ODI indices with an indication of the value used for the calculation. Note that ODI 3% calculation is available only when AHI calculation value is 3%.
- Select the comparison operator for oxygen saturation level value of 88 (<88 or <=88).
- Select the AASM values or manually define the ranges to be used in calculating the AHI Severity graph (3rd page of report). See section Report>Sleep Report for more details.

3.3.7 Setup>General Settings>Report Appearance

Used to change the basic Report appearance, adding a logo, affiliation, header and footer.

General Settings
Events Clinical Data Custom Fields General Options Security Analysis/Statistics Parameters Report Appearance Report Translation HL7 Configuration WP Device Configuration
Logo Current Logo:
Headers and Footer
Eooter:
O Dne page Sleep Report (results only) Image Sleep Report (2-3 pages) O Two pages Sleep Report (all 3 pages)
✓ Draw Report Title Background △ Automatically Generate Report File on 'Set as Reported' Selection The Report File will be created under folder: c:\itamarmedicaldb\Database\Archive □ Customized <u>Report</u>
OK Cancel Help

Figure 28 - General Settings Dialog Box – Report Appearance Tab

To add a logo to the sleep report:

- Click the **Set Logo...** to browse for the desired logo file. File may be in bitmap (BMP extension) or JPEG (JPG or JPEG extensions) formats. Highlight the logo file and click **Open** to add the icon. Note that actual lcon size in the report does not exceed 140X100 pixels, therefore larger images will be resized for proper printing.
- To clear the selected logo click Clear Logo.

To add a header/footer to the sleep report:

- Type in the desired header text in the Headers field.
- Type in the desired footer text in the Footer Field.

To set the number of pages of the Sleep Report:

- Select "1 page Sleep Report" to see the results (statistics and signals) page.
- Select "2 pages Sleep Report" (default) to see patient, physician and study information in addition to the "1 page Sleep Report".
- Select "4 pages Sleep Report" to see sleep/wake states, sleep stages and respiratory indices charts in addition to "2 pages Sleep Report".

To set the Printer page size of the Sleep Report:

• Select "A4" or "Letter" page size.

To add additional historgrams for pulse rate and oximetry:

 Select "Oximetry and Pulse Rate Histogram Page in Sleep Report". The additional page will be added to the end of the sleep report (see section 4.6.2).

To show the bitmap on the top of Sleep report:

• Select "Draw Report Title Background".

To generate the Report File on 'Set as Reported' Selection:

 Check the option of "Automatically generate report file on 'Set as Reported' selection". Checking it enables one of the 2 options of saving the study: as RTF or PDF (the default option). This implies that every time a study is 'Set as reported' automatically a copy of the reported study is saved in the Archive folder (found under the database folder, the default being C:\Itamar Medical\zzzpat\database\Archive) in the chosen format.

To show your company's customized Sleep Report (provided by Itamar according to your specifications):

• Select "Customized Report". When checking this option the customized report will replace the default zzzPAT report.

To define where the detailed report is saved:

- Click "Browse" and select a folder. The detailed report will be saved as a PDF file in this folder.
- See section 4.6.8 for more information about the detailed report.

3.3.8 Setup>General Settings>Report Translation


Used to change the sleep report headings.

	Custom Fields General Options Security Report Translation HL7 Configuration	Analysis/Statistics Parameters WP Device Configuration
Report Appearance Pefault Report Label Sleep Study Report Patient Information First Name: Last Name: ID: Birth Date: Age: Gender: Insurer: BMI: Address: Sleep Study Information Study Date: S/H/A Version: Study #: Referring Physician Information Patient Name: E-mail:	Current Report Label Current Report Label Sleep Study Report Patient Information First Name: Last Name: Last Name: ID: Birth Date: Age: Gender: Insurer: BMI: Address: Sleep Study Information Study Date: S/H/A Version: Study #: Referring Physician Information Patient Name/ID: Patient Name: E-mail:	WP Device Configuration
Work Phone: Mobile Phone: Fax: Medical Information Medical History Medications Comments	Work Phone: Mobile Phone: Fax: Medical Information Medical History Medications Comments	

Figure 29 - General Settings Dialog Box – Report Translation Tab

To translate report headings to a different language:


- All report field labels may be changed by selecting the desired report field and clicking **Modify...** In the dialog box that opens you can type the new text to be used in the reports.
- Click **Default Settings** to revert to the default text for all report field labels.

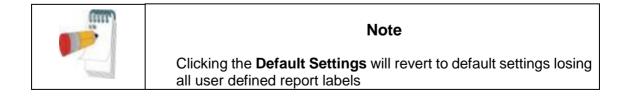

3.3.9 Setup>General Settings>HL7 Configuration

Used to enable and configure communication with the LIS (Laboratory Information System). If the LIS option is enabled, results and report PDF files can be exported to LIS through the Export dialog, see "Tools>Export/Delete".

 Note
In order to enable this dialog the HL7 Service must be already installed on the PC where zzzPAT DB is installed.

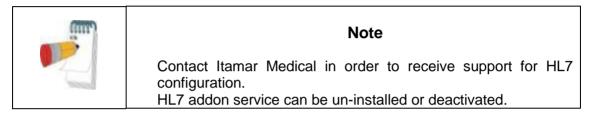

eneral Settings			×
Events Clinical Data Report Appearance	Custom Fields Report Translation	General Options Security HL7 Configuration	Analysis/Statistics Parameters WP Device Configuration
LIS Application Name:	L	LIS <u>F</u> acility:	Enable HL <u>7</u>
Application Name: zz	zzPAT F	Fa <u>c</u> ility: Itamar-Medical	<u>C</u> lear History Data
Sending Port: 2	575 L	LIS <u>H</u> ost: localhost	Define Fields
Listener P <u>o</u> rt: 3	085 F	Polling Freg.: 30 seconds	
<u>N</u> um. of Retries: 5			
🔽 Send ACK	🗖 Log Text	to System Event Viewer	
Receive ACK	🗖 <u>B</u> asic Sec	ourity	
selecting this option, report	te that report PDF contains t PDF files will be saved un	Brows s protected health information (PHI). B nder this path and it is under your	
Fesponsibility to manage the Show Order Number HL7 Service Status: Not In: Note: LIS - Laboratory Inform	stalled		
		OK Cancel	Help

Figure 30 - General Settings Dialog Box – HL7 Configuration

- "LIS Application Name" is the identifier of the LIS used for HL7 messages. If "Basic Security" option is selected, this field is mandatory and only inbound messages with this identifier are intercepted by the zzzPAT application.
- "LIS Facility" is the identifier of the LIS facility used for HL7 messages. If "Basic Security" option is selected, this field is mandatory and only inbound messages with this identifier are intercepted by the zzzPAT application.
- "Sending Port" is the network port that the LIS listens on.
- "Listener Port" is the network port that the zzzPAT application listens on.
- "Num. of Retries" defines the number of retries when sending messages.
- "Polling Freq." defines how long to wait before retrying.
- "Send ACK" option defines whether or not to send an ACK.
- "Receive ACK" option defines whether or not to receive ACK on outgoing messages.
- "Send Report PDF File" option defines whether or not to send the zzzPAT sleep report as a pdf file along with the test results.
 If option is selected, a path must be entered through the "Browse" function, to be used as a depository for the pdf files.
 It's recommended to use a network path. Make sure appropriate users have write permissions to this path.
- "Show Order number" option enables to search in "New study" dialog the patient by its order number.



Warning

HIPPA Warning: The report PDF contains protected health information (PHI). By selecting this option, report PDF files will be saved under the selected path and it is under your responsibility to manage and protect these files.

Note

The study report must be viewed in order for the pdf file to be created. It is recommended to use option 'Open report after "Load Study and Analyze" in order to automatically open the report, see <u>Setup>User Settings>Options</u>. If study is re-analyzed, manually edited or analysis parameters are changed, the report should be viewed after the changes so the pdf can be recreated.

- "Log Text to System Event Viewer" option defines whether or not to log the full message text.
- "Basic Security" option defines whether or not to handle security. If this option is selected then only messages coming from the "LIS Application Name" and "LIS Facility" are intercepted by the zzzPAT application.

- "Disable HL7" function stops the HL7 service. The function is supported only for stand-alone zzzPAT and only if user has admin permissions on running PC and zzzPAT has been started with the "Run As Administrator" option. For Shared database cases, have your IT person contact Itamar Medical for instructions.
- "Clear History Data" function cleans up all messages in queue and should be used in case of a malfunction in order reset the HL7 service.
- "Define Fields" enables selecting which fields will be sent in result message and also changing the field names.

Default Report Label	Current Report Label	- Modiy
🕑 Height	Height	Delad Setings
🕑 Weight	'w/eight	Cejaa seorgi
Patient BMI	Patient 8MI	Select All
Pabert Age	Pabert Age	
Neck Circumference	Neck Cecumierence	Clear Ait
Epworth Score	Epworth Score	Save
Pacemaker	Pacemaker	
Medications	Medications	Cancel
Diseases	Diceases	
2 Diagnose	Diagnosis	
Recommendations	Recommendations	
Contrent	Convivent	
Coverage	Coverage	
Custorel	Custom1	
Custon2	Custom2	
Custon3	Custom3	
SwVersion	SWVersion	
Havenion .	HWVersion .	
Anabris/Version	AnalysisVersion	
2 DeviceSeriaNumber	DeviceSeiaNumber	
TotalStudyTime	Tota/StudyTime	
SleepTime	SleepTime	
ValdSleepTwe	ValidSleepTime	
🔄 pRDI	pRDI	
10 o441	p4H9	
2 001	ODI	
REM	REM	
····	1-C	

Figure 31 - General Settings Dialog Box – HL7 Configuration/Modify fields dialog

- "Modify" function can change the name of the field
- "Default Settings" will change names back to default
- "Select All" will select all fields to be sent in result message
- "Clear All" will clear all fields so none of the fields will be sent in result message (however, at least one field must be selected)
- "Save" will save all changes
- "Cancel" will cancel all changes



Note

For outbound messages (studies results) the LIS application name is "zzzPAT" and Facility is "Itamar-Medical".

3.3.10 Setup>General Settings>WP Device Configuration

Used to select which type of WatchPAT devices are to be used and to set up WP1 configuration settings.

eneral Settings Events Clinical Data Report Appearance	Custom Fields General Options Security Analysis/Statistics Parameters Report Translation HL7 Configuration WP Device Configuration
Devices in Use Type of Devices: Default Device:	✓ WP-0NE WP-0NE
WatchPAT ONE Config Geographical Region: Email Address for Notifications:	Iration US xxxxx@itamar-medical.com
Policy Number of Retries Error Handler:	3 © End © Continue
PIN Code Type:	PIN
	OK Cancel

Figure 32- General Settings Dialog Box – WP Device Configuration

- "Type of Devices" is used to define WP devices to be supported by the zzzPAT
- "Default Device" is used to set the default device
- WP1: "Geographical Region" is used to select the geographical region of the study so the study files will saved in the corresponding FTPS server
- WP1: "Email Address for Notifications" is used for setting an email address to which notifications are sent when study is completed
- WP1: "Number of Retries" is used to set the number of retries the patient has available when inserting the PIN code during setup
- WP1: "Error Handler" is used to set the action in case number of retries is exceeded, End study or Continue
- WP1: "PIN code type" is used to select the type of code to be used by the patient. This 4 digit code can be the last 4 digits of Social security, Credit Card, Mobile number, etc.

4 Using zzzPAT

4.1 **Preparing a New Study**

Preparing the Patient file is a mandatory stage in the preparation of the WP devices for a sleep study. Before proceeding, make sure that the WatchPAT drive was defined as described in Section 2.5.

4.1.1 Launching zzzPAT


- Launch 'zzzPAT' by clicking the zzzPAT icon 💟 on your desktop.
- If the zzzPAT icon is launched the following login dialog opens:

zzzPAT - Login	X
C Active Directory Service (ADSI)	OK Close zzzPAT
Login Password	Change
DB Connection: Local	



- Enter Login and Password. When a Shared Access mode is installed, the login screen allows the user to choose to which database the zzzPAT will connect.
- The login dialog has two options for Login type:
 - Active Directory (current domain server).
 - Users from zzzPAT database

When Active Directory option is used, the application will check if group "zzzPAT" exists in current domain and the user name is a member of the group. If so, it will authenticate the password. If user is new to zzzPAT then the user will be defined in zzzPAT database with normal permissions. Administration zzzPAT permissions can be set only via zzzPAT's User Administration function.

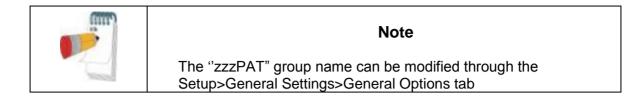



In Shared Access mode it is highly recommended to always connect to the Shared database - this will ensure all data is stored in one central location allowing future retrieval from all connected stations. Local database should be selected only if network access is not available, or for special purposes (training, travel, etc...).



Note

In Shared Access mode, if data was stored on the local database, it is recommended to export the data into the Shared Access database as soon as practical.

- If applicable, select the desired database to connect to.
- Enter your login name and password and click **OK** to continue.

4.1.2 Preparing a New Study

The New study window will be opened with fields according to default device type (WP200(U)/WP300/WP-ONE) according to the configuration.


4.1.2.1 Preparing a New Study (WP200(U)/WP300)

- Make sure the device is connected to the PC with the zzzPAT software using the USB cable.
- Click **File>New Study Details** in zzzPAT, or click the 'New Study' icon 📃 in the tool bar.
- The 'New Study' dialog box appears:

🕎 Nev	v Study	×
	Patient [D: (Mandatory) First Name: Last Name: # Date of Birth: Age: Gender: Male Date of Birth: Age: Gender: Male Female Height: ' '' Weight: Ib. Neck Circ.: " Epworth: Pacemaker Demographics Clinical Data	Save To WatchPAT Clear Page <u>R</u> ead From WatchPAT <u>R</u> un Device Test E <u>v</u> it
	Physician Ligense: Ligense: Last Namg: Demographics	
-₩+	Study Coverage: Payment Code: Payment Code: Study with Tamper-Proof Testing Multiple Nights (up to 2 nights: 8 hr per night and no need to charge between nights) Run WP Device Test while 'Saving to WatchPAT' Switch to WP-ONE Device Mgre	
	WP300 Data Initialized for patient 000000000. DB Connection: Local	

Figure 34 - New Study Dialog Box for WP300

- Fill the mandatory Patient ID in the **Patient** fields.
- Insert any additional information if needed, the rest of the fields are optional.
- Select "Pacemaker" if the patient has an implantable pacemaker. The zzzPAT will automatically detect segments where the PAT pulses seem to be paced, i.e. very low pulse rate variations (near to stable) and exclude these segments from the analysis.

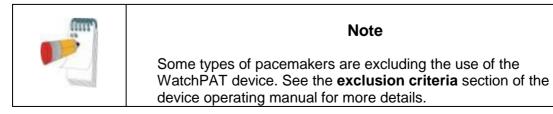

- Select "Study with Tamper-Proof Testing" if you want to use the Patient Identification Bracelet. By enabling this option you can use the bracelet in order to verify that the identified patient is indeed the one sleeping with the device (see Tamper-Proof testing in WP Operation Manual). This option is shown only if the feature is enabled in Setup>User Settings>Options.
- Select "Multiple Nights" option in order to run up to 3 nights with the same WP device (see Multi-night study in WP Operation Manual). This option is shown only if the feature is enabled in Setup>General Settings>General Options.
- WP300: The "Run Device Test" option appears only when a device is connected. By selecting this option, the connected device will be tested and the results will appear in a separate window. This test will perform the same "Test Device" operation run from a standalone device (see Operator Tests in WP Operation Manual).
- WP300: The "Run device test while saving.." option appears only when a device is connected. By selecting this option, the connected device will be tested automatically when user selects the 'Save to WatchPAT' and the results will appear in separated window (see Figure 35). This test will perform the same "Test Device" operation run from a standalone device.
- To register a study for the WP1 device use the "Switch to WP-ONE Device" button
- Click the **Save to WatchPAT** button.




Note

If the Patient ID field was left unfilled, a message will be displayed requesting the need to be filled. Fill in the missing information and press the **Save To WatchPAT** button again.



The units used for weight and height in the 'New Study' dialog box are defined by the regional settings of the PC.

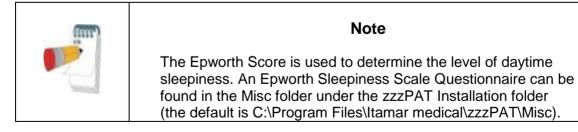

Devi	ice Test Results		×
	Device SN:	100279	
	Patient ID:	08529_1	
	Version	4.0.1527	
	PAT	used	
	battery	low	
	SBP	missing	
	File	loaded	
		_	
	Test Result:	FAILED 🗙	
		Close	
		<u></u>	

Figure 35 – WP300 Device Test results Dialog Box

If WP contains data (either a night study that has not been loaded to the zzzPAT Database or new patient data that has been prepared but not used in a study), the following dialog box opens:

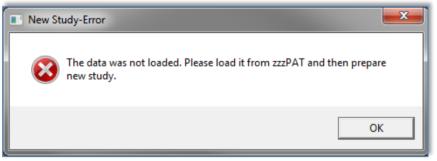


Figure 36 - WP Not Loaded Dialog Box

• After saving the patient information to the WP the following message appears:

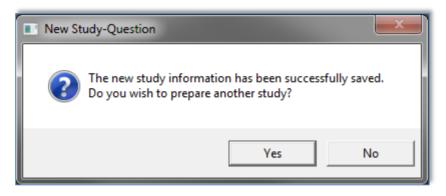


Figure 37 - New Study Termination Question

Click **Yes -** if you wish to prepare additional study. Click **No -** if you wish to exit the 'New Study' dialog box.

• Disconnect the USB cable from the device.

4.1.2.2 Preparing a New Study (WP1)

- Make sure that there is an active internet connection
- Click **File>New Study Details** in zzzPAT, or click the 'New Study' icon 😰 in the tool bar.
- The 'New Study' dialog box appears:

New Study	11,76.8		-	×
Patient				Register WP-ONE
<u>I</u> D:	(M	landatory)		Clear <u>P</u> age
First <u>N</u> ame:		Last Name:	#4	
Date of <u>B</u> irth: r	mm / dd / yyyy	Gender:	⊂ <u>M</u> ale ⊂ <u>F</u> emale	
Height:	' ''	<u>W</u> eight:	lb.	
Nec <u>k</u> Circ.:		Epworth:	Pacema <u>k</u> er	E <u>x</u> it
	Demogr <u>a</u> phics		Clinical <u>D</u> ata	<u>L⁷n</u>
Physician				
Li <u>c</u> ense:				
Firs <u>t</u> Name:		Last Nam <u>e</u> :	<u>Å4</u>	
	Demogr <u>a</u> phics			
-Study]
Coverage:		Payment Code:		
Device SN:				
PIN: Email for		PIN Type:	PIN 💌	
Notifications:				
		Switch to WP300) Device More	
WP-ONE	Ready for pre	pare New Study.		1
DB Connection: I	Local			

Figure 38 - New Study Dialog Box for WP1/WPONE-M

- Fill the mandatory fields: Patient ID, Device SN and PIN.
- Make sure the Email address appears on screen is the correct address for receiving email notifications (Email address is defined during setup).
- Insert any additional information if needed, the rest of the fields are optional.
- Select "Multiple Nights" option in order to run up to 3 nights with the same WP1-M device by chaging the probe each night. This option is shown only if the feature is enabled in Setup>General Settings>General Options. When "Multiple Nights" is checked, the selection of "2 Nights" or "3 Nights" will appear and the default selected option will be according to what is defined in General Options. Additional mandatory text boxes will appear to define the additional probes' SN numbers.

im	Note
	In the "Select Study to Download" window (see Figure 43), all registered devices with multi-night data will list the device SN number followed by the probe SN number(s) in parentheses.

- Click the **Register WP-ONE** button to register the device and the probes. Registration will be unsuccessful in the following cases:
 - \circ If one of the SN numbers is invalid or missing.
 - If one of the SN numbers is already registered
- To initiate study for WP300 or WP200(U) Device use the "Switch to WP200/WP300 Device" button.

(intr)	Note
	If same WP1 device SN is already registered, an error will appear to notify user to insert correct/other device SN. Fill in the correct information and press the Register WP-ONE button again.

Note
The units used for weight and height in the 'New Study' dialog box are defined by the regional settings of the PC.

(intr)	Note
	The Epworth Score is used to determine the level of daytime sleepiness. An Epworth Sleepiness Scale Questionnaire can be found in the Misc folder under the zzzPAT Installation folder (the default is C:\Program Files\Itamar medical\zzzPAT\Misc).

• After registering the patient information to the Web Server the following message appears:

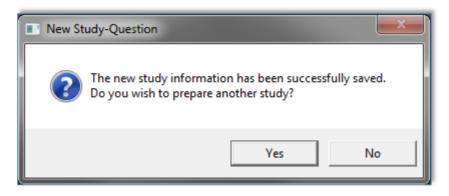


Figure 39 - New Study Termination Question

Click **Yes -** if you wish to prepare additional study. Click **No -** if you wish to exit the 'New Study' dialog box.

4.1.3 New Study screen features

Besides the main screen fields required for preparing a new study, there are additional fields that allow thorough documentation of the patient's past and current medical condition. In addition, zzzPAT enables you to load patient details from previous studies. Other features in this screen enable organizing the studies into groups using categories of your choice.

4.1.3.1 Synchronizing the WP device clock with the zzzPAT software (WP200(U))

Select the "Synchronize WatchPAT clock with PC clock while Saving to WatchPAT" while preparing a new study if the WP device clock needs to be set (see Figure 34).

4.1.3.2 Documenting additional information

To add demographic information:

Click the **Demographics** button (see Figure 34). The following dialog box will open:

Demographic Details					
Street: City:		OK Cancel			
State: Country: Phone:	Zip Code	Clear All			
E-mail:]				

Figure 40 - Demographic Details Dialog box

• Insert the information into the appropriate fields and click **OK** to save.

To add optional clinical information:

• Click the **Clinical Data** button. The following dialog box opens:

Clinical Data	×
Medical History	ок
List of available medical conditions: Patient's medical conditions:	Cancel
Anxiety disorder	
Depression	
Frequent arousals	<u>Clear All</u>
Frequent arousals gasping for air	
Morbid obesity	
Morning headaches Narcolepsy	
Nocturia	
None	
(Please double click on the list to add an item to the Medical History box)	
Medications	
List of available medications: Patient's medications:	
Anti-acid Anti-arrhythmia	
Anti-coagulants	
Anticonvulsants Anti-depressants	
Diuretics	
Hypoglycemic Inhaled steroids	
Insulin None	
None Therapy	
(Please double click on the list to add an item to the Medications box)	




- Double click on the relevant items listed on the left, to add them to the Patient information or add information in free text.
- After adding the selected data click **OK** to save.

To remove an item from your selection:

- Highlight the item you want to delete.
- Hit delete key or right click and choose 'Delete' from the pop-up menu.
- Click **OK** to continue.

4.1.3.3 Retrieving patient information from previous studies

This feature allows retrieving patient information from previous studies into current 'New Study' dialog box. In addition it enables viewing the list of all patients in the zzzPAT database.

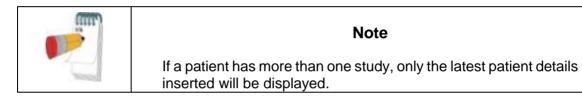

To view a list of all the patients in the zzzPAT database:

- In the 'New Study' dialog box (see Figure 34), leave the fields: 'patient ID', 'first name' and 'last name' empty.
- Click , the 'Search Results' dialog box will open with a list of all the patients in the zzzPAT database.
- Select the study you need and click **OK**. The patient's information will be retrieved into the 'New Study' dialog box.

To view patients by selected categories:

- Enter patient ID, first or last name into the 'New Study' dialog box.
- Click dialog box will open with a list of patients that will match the criteria inserted above.
- Select the patient you need and click **OK**. The patient's information will be retrieved into the 'New Study' dialog box.

4.1.3.4 Study details


To assign a study to a group or category for classifying patients by special group/category:

• Click the **More** button in the **Study (optional)** section of the 'New Study' dialog box. The following dialog box will open:

			*	OK
				Cancel
			*	Clear All
Custom Fields				
Custom Field 1:		Ŧ		
Custom Field 2:		Ŧ		
Custom Field 3:	Í	Ψ.		


Figure 42 - More Study Details Dialog box

• You can enter free text in the 'Comments' field and select an existing study group from the list you previously prepared in the 'Study Custom Fields' or insert free text into the fields of the **Custom Fields** section.

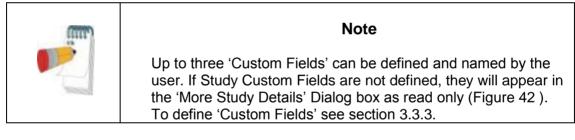


• You can select ASDA or Medicare protocol for calculating pAHI and pRDI indices on a study level, see 3.3.6.

4.1.3.5 Reviewing and editing information stored on a WP (WP200(U), WP300)

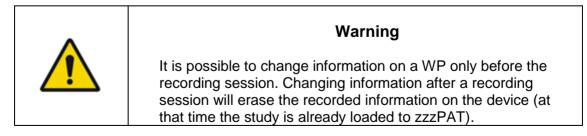

- Insert the WP into the USB drive.
- Open 'New Study' application. Click **Read From WatchPAT** button.

- If the information does not need to be edited, click **Exit**.
- Make any necessary changes and click **Save to WatchPAT**.

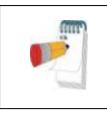


Note

Study details, including patient information, can be viewed and modified after the study is loaded from the WP to the zzzPAT database. See Section 4.2.8 for more details.

4.2 Managing Patient Studies

WP200(U)/WP300: After a WP sleep study is done, connect the WatchPAT into the USB, open zzzPAT application and press File>Load Study and Analyze from the main menu. The recorded digital data along with patient information are loaded into the zzzPAT database. The recorded data is automatically analyzed. The user can subsequently review, edit, add Diagnosis and Recommendations and produce a Sleep Report. Previously loaded studies can be opened and reviewed.

WP1: After a WP sleep study is done, open zzzPAT application and press **File**>**Load Study and Analyze** from the main menu. A window will be opened with all registered patients that the study was not yet retrieved from the web server (see Figure 43). Select study to download and the recorded data will be loaded into the zzzPAT database for selected patient. The recorded data is automatically analyzed. The user can subsequently review, edit, add Diagnosis and Recommendations and produce a Sleep Report. Previously loaded studies can be opened and reviewed.

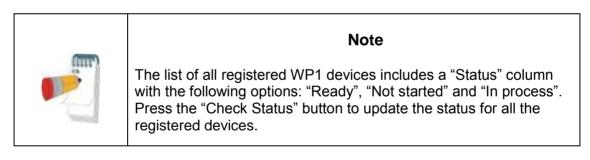



Note

It is possible to search for patients using the Patient ID, and for devices using the Device SN.

Search by	Patient ID:		Device SN:		Find	Find Next		Check Status
Patient ID	Last Name	First Name	Registration D	PIN	WP Device SN(ProbeSN)	Status	^	Download Study
1			Feb-08-2022	1111	104175484			
2			Feb-08-2022	2222	104229042			
3			Feb-08-2022	3333	104245055			
4			Feb-08-2022	4444	104245204			
5			Feb-08-2022	5555	104245436			
6			Feb-08-2022	6666	104245956			
7			Feb-08-2022	7777	104246129			Switch to WP200/300 Device
8			Feb-08-2022	8888	104246707			
9			Feb-08-2022	9999	104246806			Close
10			Feb-08-2022	1010	104247093(104247093)			
10			Feb-08-2022	1010	104247093(104247572)			
11			Feb-08-2022	1111	104258819(104258819)			
11			Feb-08-2022	1111	104258819(104258926)			
12			Feb-08-2022	1212	104258934(104258934)			
12			Feb-08-2022	1212	104258934(104258942)			Send Request to Itamar Medical
3			Feb-08-2022	1313	104258959(104258959)			Senu nequest to Italiar Medical
3			Feb-08-2022	1313	104258959(104259015)			
13			Feb-08-2022	1313	104258959(104259056)		~	

Figure 43 – List of WP1's registered patients

(IIII)	Note
	To switch between WP1's registered patients list to download study from WP200(U)/WP300 device select the "Switch to WP200/300 Device" button.

Note
selected registered patient didn't yet sleep with the WP1 device a oper message shall appear on screen.

	Note
	It is important to login to the correct database in which the data needs to be stored. Typically that should be the Shared database, however if data was stored locally, local login may be required.

4.2.1 File>New Study Details

Prepares the Patient file on the WP for a sleep study (Section 4.1).

4.2.2 File>Load Study and Analyze

This command loads the sleep study data from the device and saves it into the zzzPAT database.

While loading the data the message 'Loading Study' (Figure 44) appears on the screen indicating that the data is being transferred from the WatchPAT to the hard disk and the patient file is saved in the database.

Loading Study		
Loading night study of patient: - John Smith		
Please wait (It may take several minutes depending on your computer) Cancel		

Figure 44 - Loading Study dialog box

At this stage Automatic Analysis is performed and the results are saved in the database. After the Automatic Analysis is completed the results are displayed on screen (see Figure 48). The user has the option to display the "Sleep report" or "Sleep Indices" report box automatically after loading a study (see 3.2.3).

WP200(U)/WP300: When a multi-night study is loaded all the night studies are loaded automatically and the last loaded study will be displayed. Use the Open Study dialog to open and review all the night studies.

4.2.3 File>Open Study

Opens studies stored in the zzzPAT database from previously loaded studies (see Figure 45). Double click on a patient and the studies for that patient are listed with the date/time of

each study. Double click the study icon to load and display recorded information on the screen.

Select Patient Study	
124325161 Rozen, Sara	ОК
Selected studies: Unreviewed, Reviewed, Reported - sorted by "Patient ID" and "Last Name"	Cancel
□ 24325161 Rozen, Sara □	Select Studies
I Number of studies: 2 (All)	

Figure 45 - Select Patient Study dialog box

If several Analysis exist for a study a dialog will appear prompting to select the Analysis to open.

Analyzer	Date/Time Created	Status	OK
	7/9/2018 06:41:22 PM	Reviewed(A)	
Dr. Mark Adams	7/9/2018 06:04:28 PM	Reviewed(A)	Cancel



Select Studies button

Enables the user to define, select and organize the displayed studies in the 'Select Patient Study' dialog box.

• Click Select Studies.

The following Dialog box will open:

Select Studies Options		X	
Show Study Status Study Status Study Status Study Status Status Study Status Status Status Status Status Study Status Status Study Status Study Study Status Study Study Study Status Study Study Study Status Study Study St	Study Date Last All month(s) Study Date Between 2015-09-14 and 2015-12-13	OK Cancel	
Archived Sort Order First Patient ID Second Last Name			

Figure 47 - Select studies options dialog box

Show section - Each study is associated to one of the following statuses (listed below). Checking the boxes to the left of these statuses enables the user to define, according to the following categories, which of the studies from the zzzPAT database will be displayed on the 'Select Patient Study' dialog box:

Unreviewed	_	Study was loaded to zzzPAT and automatic analysis performed on it.
Reviewed	_	User modified events, entered Diagnosis or Recommendations, saved study results or produced a sleep report.
Reported	_	User selected the 'Set as Reported' option in the 'Clinical Diagnosis' dialog box. This study cannot be modified and no additional information can be inserted into the report.
Archived	_	Study was extracted and removed from the database to an archive out-side the zzzPAT database. These studies cannot be reviewed unless retrieved back into the database first.

(IIII)	Note
	The Archived option is unchecked automatically every time the "Select Patient Study" dialog box is opened. In order to view archived studies the "Archived" option must be selected each time in the "Select studies options" dialog box.



Note

The typical cycle of a study is: Unreviewed \rightarrow Reviewed \rightarrow Reported \rightarrow Archived

Sort Order section - Enables the user to define the order by which the studies are displayed. It is possible to display the studies by ID, last name, date of study, date file loaded and additional user inserted criteria.

'First' order field	—	Contains system predefined criteria and user defined criteria inserted in 'Define Study custom Field' (see 3.3.3).
'Second' order field	_	Contains only system predefined criteria.

Study Date section - enables the user to filter which studies are displayed.

'Last xx months'	_	Shows studies recorded during the last xx months (xx=1, 3, 6,12,18 or All months)
'Study Date Between"	_	Show studies recorded between selected ranges.

4.2.4 File>Save Study Results

Saves the patient study results (events) currently being viewed without closing zzzPAT, and sets the study as reviewed. This feature is important when Respiratory events are edited (added or deleted) and the new events need to be saved for future zzzPAT sessions.

4.2.5 File>Save as New Analysis

Saves the current Analyisis with the current events and signals as a new Analysis.

4.2.6 File>Close Study

Closes the patient study currently being viewed without closing zzzPAT.

4.2.7 File>Exit

Closes both the patient study being viewed and the zzzPAT.

4.2.8 Edit>Study Details

Opens the 'View Study Details' dialog box with current patient information. This information can be edited by clicking the **Edit** button. Authorized users can change the patient ID only once.

4.2.9 Edit>Undo

Choosing "Undo" or clicking on the zzzPAT toolbar enables the user to undo the last operation. 'Undo' is enabled only after changing events (manual editing).

4.2.10 Edit>Sleep Stages Editing Using Mouse


Choosing this option or clicking on the zzzPAT toolbar toggles the option that allows to manually edit the sleep stages and CSR times with the mouse. If option is selected and mouse hovers over a sleep stage event (i.e. Light Sleep) the mouse arrow changes to a hand and the selected event may be dragged or resized accordingly.

4.2.11 Edit>Copy...

When a signal section is highlighted, the Copy feature is enabled to allow the user to copy the desired data either to the clipboard as an image or to a file in binary format.

4.3 The Display Screen

The main screen displays the WP recording waveforms with the events that were detected by the automatic analysis. The traces are displayed synchronized to a uniform time base.

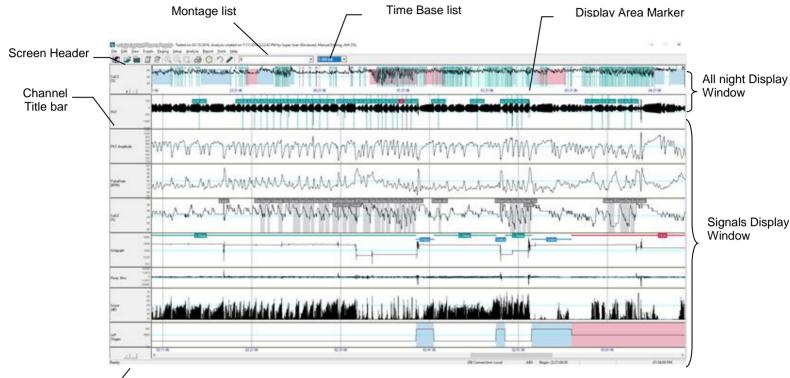

Status bar ____

Figure 48 - zzzPAT display screen

The 'Screen Header' consists of 'zzzPAT', patient's name, ID number, Analysis status, date and time and the zzzPAT user's name.

Events determined by the Automatic Analysis are displayed on the 'Signals Display Window'.

Placing the mouse cursor on an event pops up a tool-tip that contains:

- Event name
- Event source (Analysis or User)
- Start time
- Duration

4.3.1 The All Night Window and View Channels

It is possible to view any channel you select in an 'All Night Window' display even if you change the time base for viewing all the channels of the study.

 To display the All-Night window, navigate through View>All Night Window, check the 'All Night' option. Or View>Channels.

The following dialog box will open:

Se	et Channels View				—
[Channels All Night PAT	Visit	ble		OK Cancel
	Total Size:	100.02			<u>S</u> elect All <u>C</u> lear All
	Name	Area	Name	Area	<u>M</u> ake Same Size
	PAT	18.92	PAT Amplitude	6.49	
	✓ PulseRate	18.92	🔽 SaO2	18.92	
	✓ WP Stages	6.49	🔽 Actigraph	18.92	
	Body Position	5.68	🔽 Shore	5.68	
	Attached Channel f	or General Ever	ıts		


Figure 49 - Selecting the All night Window

- Check the box to the left of 'Visible' in the All Night section.
- Select the channel you wish to display in the 'All Night Window'.

The **All Night** section is used to enable/disable the 'All Night display' and to select the channel for it.

Checking the boxes by the channel names defines the channels displayed on the zzzPAT screen.

The value entered into the 'Area' fields determines the channel's screen display relative size. The total sum of the values in the Area fields should be 100. Clicking the **Make Same Size** button rescales all channels to the same size.

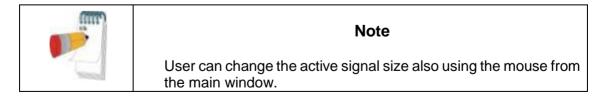

Note
User can change the order of the signals in the view by dragging the active signal to the desired location from the main window.



Channels'	order and	size are sav	ved as part o	of the montage.
• • • • • • • • •		0.20 0.00 00.0		si une interneiger

Note

The **Attached Channel for General Events** section is used to set the channel on which the general events are displayed. General events are events generated by the automatic analysis of the zzzPAT.

To remove the 'All Night Window':

• Use the two options mentioned above or right click on the 'All Night Window' title window (left side of the screen), select 'Remove from View'.

'All Night Window' features:

- The 'Display Area Marker' that appears as a highlighted rectangle in the 'All Night Window' (See Figure 48), is the section being viewed in the 'Signals Display Window'.
- It is possible to navigate through the recorded data through the 'All Night Window' by clicking on a spot on the 'All-Night' window, or by dragging the highlighted rectangle.
- When in the 'All Night Window' display, clicking the mouse right-button opens the following pop-up menu:

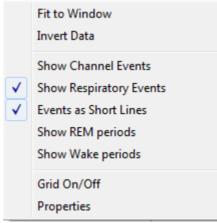


Figure 50 - All Night pop-up Menu

Fit to Window	 Auto fits the All Night window's signal so it is displayed in about 80% of the window's height.
Invert Data	 Inverts the y-Scale and signal of the All Night window.
Show Channel Events	 Displays channel events in the All Night window.
Show Respiratory Events	 Displays respiratory events in the All Night window.
Events as Short Lines	 Option to show full vertical line on the events in all-night window or just short lines so the channel data will be not covered by event lines.
Show REM periods	 Mark REM periods on all night window with background regardless of the channel selected in all night screen
Show Wake periods	 Mark Wake periods on all night window with background regardless of the channel selected in all night screen
Grid On/Off	 Toggles the grid in the All Night window.
Properties	 Opens up the 'Signal Properties - All Night' dialog box (see Figure 51).

Signal Properties - All Night				
General y-Scale Vie	w			
Channel name:	PAT Amplitude			
Sampling rate:	1.00 Hz			
Recording system:	WP200			
Derived from:	PAT			
ОКС	ancel Apply			

Figure 51 - Signal Properties – All Night window


Right clicking the mouse on the left of All Night' channel opens a pop-up menu with the following options:

	Select Channel to Display Fit to Window	► Ctrl+F
	Invert Data	
	Remove from View	
	Show Channel Events	
\checkmark	Show Respiratory Events	
\checkmark	Events as Short Lines	
	Show REM Periods	
	Show Wake Periods	
•	Show Y-axis	
	Properties	

Figure 52 - All Night pop-up menu


Select	- /	Allows the user to choose which channel will be displayed
Channel to		in the 'All Night Display Window'.
Display		

The rest of the options are the same as in Figure 50 explanations.

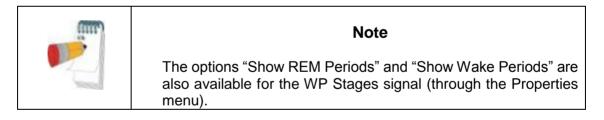

4.3.2 The Active Channel

Clicking on a channel or a channel title activates that channel (the color of the activated signal and titles will change).

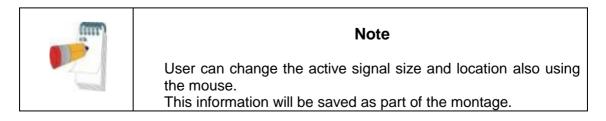


A right-button click on a Channel title activates the channel and opens a pop-up menu with the following options:

	Fit to Window	Ctrl+F
	Invert Data	
	Remove from View	
\checkmark	Active Signal	
\checkmark	Show Channel Events	
	Grid On/Off	
	Show Values	
\checkmark	Show Y-axis	
	Properties	

Figure 53 - Active channel pop-up menu

Fit to window	_	Auto fits the active channel's signal so it occupies about 80% of the window's height.
Invert Data	_	Inverts the y-Scale and signal of the Active channel.
Remove from View	_	Removes the currently active channel from the screen.
Active Signal	_	Allows deactivating the signal.
Show Channel Events	_	Allows option of showing or not showing the events in the Active channel.
Grid On/Off	_	Toggles the grid in the Active channel.
Show Values	_	Shows the values for each data point on the Active channel (effective only under maximum zoom).
Show Y-axis	_	Shows the Y-axis values for the Active channel.
Properties	_	Opens up a dialog box with the General, y-Scale and View properties of the Active channel.
(IIII)		Note

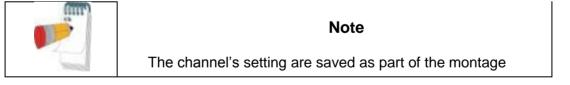

4.3.3 Status Bar

The Status Bar at the bottom of the screen contains the following information:

- Database connection (Shared or Local)
- Real Time Clock
- Highlighted segment start time
- Highlighted segment end time
- Duration
- Time mode (REL/ABS)

REL Begin: 02:12:43.43 DB Connection: Local End: 02:14:34.90 Dur: 0:01:51.47 10:21:54 AM

Figure 54 - zzzPAT Screen - Status Bar

4.4 Signal Display Options

4.4.1 View>Define Channels in Montage

The Montage screen consists of a list of all signal channels available for viewing:

- PAT- PAT signal.
- Pulse Rate Derived from the PAT signal.
- PAT Amplitude PAT signal envelope.
- SaO₂ Arterial blood Oxygen saturation level.
- Actigraph Actigraphy signal.
- WP Stages REM, Light Sleep, Deep Sleep and Wake stages.
- Body Position (optional)
- Snore (optional)
- Resp.Mov (optional)

Montage-Shifted				— ×
<u>A</u> ll Channels Resp. Mov	A <u>d</u> d> < <u>R</u> emove	<u>Channels In Montage</u> PAT PAT Amplitude PulseRate Sa02 WP Stages Actigraph Body Position Snore	♠ Move	OK Cancel
	<u> </u>			

Figure 55 - Montage screen

To add a channel:

• Select the channel in the 'All Channels' window and click Add.

To remove a channel:

• Select the channel in the 'Channels In Montage' window and click **Remove**.

To change the order of appearance of the channels:

• Select a channel and then click \uparrow or \checkmark until you reach the desired order.

(mr)	Note
	User can change the order of the signals in the view by dragging the active signal to the desired location from the main window. However adding a channel to the montage is available only from this screen.

4.4.2 View>Channels

See The All Night Window and View Channels section.

4.4.3 View>Set y-Scale

Opens the 'Set y-Scale' dialog box.

	plitude Maximum 1914 💽	Amplitude	Change +	Moditu
Channel Name	Minimum	Maximum	Ignore Fit	
PAT Amplitude PulseRate	-138 25	1519 82	0 0	
SaO2	91	130	0	
Actigraph	1092	2088	0	
WP Stages Body Position	-14 2	-10 4	0 0	
bouy rosition	2	4	U	
1				

Figure 56 - Set y-Scale dialog box.

The All Night section allows editing the 'y-scale' for the 'All Night' channel.

To edit the 'y-Scale' of all other channels:

- Mark the channel on the 'Set y-Scale' dialog box.
- Click **Modify**. The following dialog box will open:

Set y-Scale - PAT Amplitude	X
Minimum Maximum	Close
Amplitude Change	
Ignore 'Fit to Window Mode'	

Figure 57 - Set y-Scale – per channel dialog box

- Select minimum/maximum ranges for the y-Scale or control the channel's gain with + --.
- Define whether to ignore 'Fit to Window Mode'. (See section 4.4.7).

To edit the y-Scale using the zzzPAT screen:

- Activate a channel by clicking anywhere on it.
- Click on the **H** buttons at the bottom of the channels Title Bar.

To raise or lower the range scale of a channel:

• Left click on the channel title bar and drag the scale range numbers.

To zoom on a specific range of the channel:

• Right click and drag to the desired range.

4.4.4 View>Time Base

Enables to select the time base by which the study is viewed on the zzzPAT screen. It is also possible to select the time base by using the 'Time Base Drop Down Menu' in the zzzPAT tool bar.

4.4.5 View>Activate Channel Fit to Window

Sets the signals display to automatically take 80% of the selected signal's window height. The size of the signal will automatically adjust to take no more than 80% of the vertical axis of its display.

Automatic 'Fit to Window' mode is applicable only for the section being viewed on the screen. It will be disabled when viewing the study by paging (using the Page Up and Page Down keys on the keyboard) or when scrolling (using the Right Arrow and Left Arrow keys on the keyboard).

4.4.6 View >All Night Channel Fit to Window

Activates the Fit to Window mode on the 'All Night Channel'.

4.4.7 View>Fit To Window Mode

This command activates a 'Fit To Window' mode on all the displayed channels. In this mode the size of the signal will automatically adjust to take no more than 80% of the vertical axis of its display.

All the channels will remain in 'Fit to Window' mode when viewing the study by paging (using the Page Up and Page Down keys on the keyboard) or when scrolling (using the Right Arrow and Left Arrow keys on the keyboard).

Note: It is recommended to disable the 'Fit to Window' Mode on the actigraphy channel. For instructions see section 4.4.3.

4.4.8 View>Grid On/Off

Displays the grid lines of the channel values. User may enable or disable the Grid On/Off feature for a specific channel by right clicking the mouse on the highlighted channel.

4.4.9 View>Relative Time

Choosing this option or clicking ⁽²⁾ on zzzPAT toolbar toggles the time mode of the study being viewed between Absolute Time mode and Relative Time mode.

In Relative Time mode the beginning of the study is always defined as 00:00:00 on the x-scale of the zzzPAT screen and the time is measured from the beginning of the study.

In Absolute Time mode the absolute time appears on the x-scale of the zzzPAT screen; i.e. the time registered by the WP device.

4.4.10 View>Zoom In

To enable this option, highlight a section of the waveform by clicking and dragging the mouse. The option **View>Zoom In** and the corresponding toolbar button become enabled.

Activating the 'Zoom In' option modifies the Time Base to fit the highlighted segment into the entire width of the Signals Display Window (Note that the Time Base window and the Status bar details are updated automatically).

When the time base is one second, the zoom in option is disabled.

4.4.11 View>Zoom Out

View>Zoom Out and corresponding toolbar button are enabled when the current view is zoomed in.

Activating this option returns the view back to the previous zoom magnitude.

4.4.12 View>Zoom Original

View>Zoom Original and corresponding toolbar button are enabled when current view is zoomed in. This option returns the view back to the original time scale.

4.4.13 View>All Night Window

Enable/disable the All Night Window (See section 4.3.1).

4.5 Review, Analysis and Report Study

4.5.1 Data Analysis

The zzzPAT software performs an automated analysis of the WP recorded signals, providing extensive information on the patient's sleep. The analysis provides an evaluation of respiratory events during sleep, oxygen saturation, pulse rate and sleep stages statistics. Also, it provides snoring and body position statistics when the SBP/RESBP sensor is used.

4.5.1.1 Generating an Analysis

Recorded study data is automatically analyzed after being loaded from the device. You can also execute automatic data analysis by clicking **Analyze**>**Reload study and analyze**.

This function reloads the saved study data and executes the automatic analysis. If the user changed the file (adding/deleting/modifying events), these changes will be erased and will not impact the analysis.

When used on a file that was previously analyzed and saved with an older version of zzzPAT, this function creates a new analysis using the current version of the zzzPAT software.

REM Analysis is part of the automatic analysis described above.

Under certain conditions, REM analysis is unable to conclusively determine REM periods from the recorded signals.

When this occurs, the display will include only sleep and wake stages, and in the report the REM and sleep stages statistics section shall be disabled stating "Inconclusive REM Detection".

4.5.2 Event Management

Events marked by automatic analysis are shown in color-shaded boxes. Placing the cursor on an event opens a tool-tip with the following information:

- Event name
- Event created by...(e.g. Automatic analysis or user)
- Start time
- Duration


Events are colored by type and have colored background by type. To change the color of a specific type, select an event from the type you want to change, right click on it and choose Event Properties. Selecting a different color will change the color of all events of this type. See Figure 58. The current selected color is marked by 'V'.

Delete Event OK Delete Resp. Event Events in Marked Segment Start Time Draw Event Background Duration	Delete Event Event: A/H unclassified Cancel Delete Resp. Event Events in Marked Segment Start Time 02:48:36 Duration Draw Event Background Duration 10 sec		Event Properties	
	Draw Event Background Duration 10 sec			
Draw Event Background 10 sec	Event Properties	Delete Resp. Event Events in Marked Segment		
		Draw Event Background	Duration 10 sec	

Figure 58 - Change event color

(IIII)	Note
	Changing a respiratory event color will also change the color of the line related to the event in the "All Night" view. Changing the color of Wake and REM events will change the coloring of these events in the "All Night" view and in the WP Stages channel in case these options are selected.

To mark the background of a specific event type or remove an existing background, select an event of the type, right click and select Draw event background.

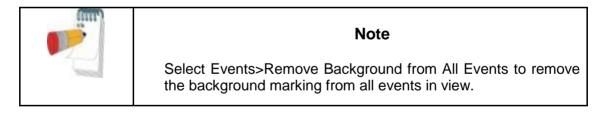

4.5.3 Adding events

Respiratory, desaturation and user defined events can be added to the study by the user. To add events:


- Mark the place you want to insert the event by clicking on the location with the mouse.
- Right click on the location and select the 'Add Event' command. Or by using the zzzPAT tool bar: Events>Add Event. The 'Add Event' Dialog box will open.

Add	Event			—
	<u>S</u> elect Cha	nnel SaO2	•	
	Desaturation			OK
	Long Name	Short Name	Event	Cancel
	A/H central A/H obstructive A/H unclassifi Wake Desaturation Excluded Resp. Event REM Light Sleep Deep Sleep Accept Signal Reject Signal CSR		A/H central A/H obstruct A/H unclassi Wake Desaturation Excluded Resp. Event REM L.Sleep D.Sleep Accept Signal Reject Signal CSR	

Figure 59 - Add Event dialog box

- 'Select Channel' Allows user to define the channel on which the added event will appear, by default the events will appear on the active channel unless a different channel is selected in the 'Select Channel' field.
- To add event Double click on the event type in the list of events. Click **OK** to continue.
- It is also possible to add free text events into zzzPAT signal display using the Free Text editing box. This text will appear on the signal as an event but will not appear in the report

The changes in the database are saved by pressing the **Save study results** button.

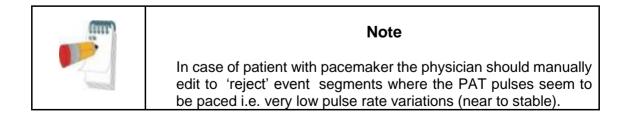

4.5.4 Deleting an Event or changing its type

An event can be deleted by right click on the event and selecting 'Delete Event'.

If a segment is selected and the cursor is on a specific event, you may Delete all events of the same type in the segment. The Undo fuction may not be available after this operation (a message will indicate it before the operation starts).

An event can also be Replaced with another type of event by right clicking on the event and select 'Replace with'.

If a segment is selected and the cursor is on a specific event, you may Replace all events of the same type in the segment with another type of event. The Undo fuction may not be available after this operation (a message will indicate it before the operation starts).



Figure 60 - Rigt Click on event

(III)	Note
	The "Replace with" operation is available only for A/H events on studies with RESBP sensor or for sleep stages.

If a segment is selected and the cursor is not on a specific event, you may Delete all events of the active signal in the segment.

	Undo Marked Segment	
	Zoom In	Ι
	Zoom Out	0
	Zoom Original	
	Add Event	
✓	Show Channel Events	
	Delete Events from Active Channel in Marked Segment	
	Fit to Window	Ctrl+F
	Invert Data	
	Show Values	
	Grid On/Off	
	Properties	

Figure 61 - Rigt Click on segment

(mr)	Note
	Adding or deleting a respiratory or desaturation event affects the results in the report.
	A note will appear in the footer of the Sleep Report indicating that the Automatic Analysis was manually changed by the user.

4.5.5 Events>GoTo Event

This functionality allows the user to navigate through the study by certain event criteria. The following event type search options are available:

Previous Event	Ctrl+Left arrow
Next Event	Ctrl+Right arrow
Previous Same Type Event	Shift+Left arrow
Next Same Type Event	Shift+Right arrow
First Event for channel	Ctrl+Home
Last Event for channel	Ctrl+End

Figure 62 - Event Type Search options

4.5.6 Events>Select Event

Select Event	
A/H central A/H obstructive Wake Desaturation Excluded REM	elected Events 00:00:00 Wake 02:35:30 Wake 02:37:30 Wake 04:35:30 Wake 04:45:30 Wake 04:45:30 Wake 07:56:30 Wake



This option enables the user to select a specific event for viewing. All existing events are listed by type and start time. User may select a specific event by clicking on it. The 'Selected Events' field in the 'Select Event' window is updated to display the page with the specific event.

4.5.7 Events>Remove Background from All Events

This option will remove the background from all events at once.

You may draw a background on any event type for easier viewing of event boundaries. In order to select this option you have to Right Click on the event and select "Draw event Background" option. You may also remove the background from an event type by deselecting the option.

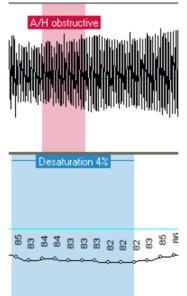


Figure 64 – Draw Background of Events

4.5.8 Editing Sleep Stages

You are able to manually edit the Sleep stages if you already selected to enable this option (see Setup>General Settings>General Options). In order to edit the sleep stages right click the mouse while on top of the sleep stage and then select a new sleep stage from the "Replace with" menu (see Figure 65)

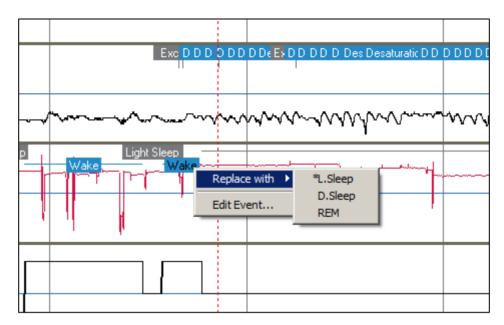


Figure 65 – Editing sleep stages

Note
Editing the sleep stages affects the results in the report. The * in front of the alternative event name denotes the last event preceding the current event. A note will appear in the footer of the Sleep Report indicating that the Automatic Analysis was manually changed by the user.

4.6 Reports

All reports can be reviewed on screen and printed. The toolbar in report view mode includes the following items:



Figure 66 - Report Toolbar

- To print a report, click on the Print button.
- To export a report to a different format, such as pdf, rtf, html, etc., click on the Export button .
- User can see how many pages there are in a report and choose which page to

view by using the icons.

4.6.1 Report>Clinical Diagnosis

This option allows adding a diagnosis and recommendations to the Sleep Report screen (see Figure 67). The information filled in this screen will appear on the first page of the Sleep Report.

- Sign the report by typing in your name or choosing it from the name menu bar.
- Check the 'Set as Reported' option to show that the current study analysis is final and that the Sleep Report is a final report for this analysis.
- Check the 'No AHI Central Statistics' option to omit central apnea statistics in report.
- Check 'Send to HL7' option in case the HL7 service is installed and enabled in order to send the specific study to HL7. Notice that the checkbox needs to be marked for each study that needs to be send to HL7.




Note

Multiple analyses may be associated with a particular study. A new analysis will be created when selecting the 'Analyze>Reload Study and Analyze' option on a study that had been previously 'Set as Reported'.

Medical Information		OK
Medical <u>H</u> istory	Medications	
- Add	- Add	Sleep Bepor
<u>_</u>		Cancel
×	-	
Comments		
	- -	
Diagnosis	Recommendations	☐ No AHI Central Statistics
✓ Add	Let Add	
<u>^</u>	^	Report Signed By:
		Super User
~	-	Send to LIS



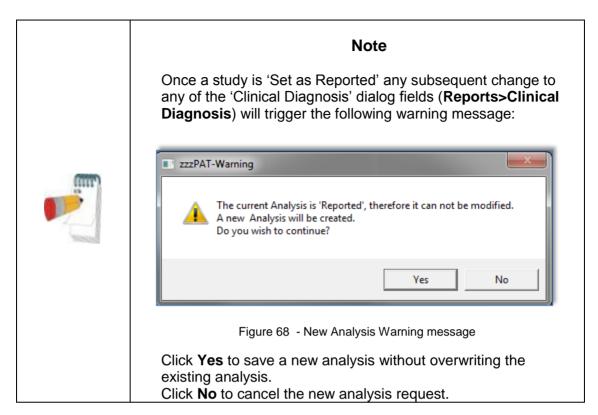

4.6.2 Report>Sleep Report

The sleep report generates a one, two, four, or five page report that provides a summary of the subject's sleep study according to the settings in Setup>General Settings>Report Appearance section.

The first page of the report (Figure 71 - First page of Sleep Report) includes Patient Information, the Sleep Study Information, Referring Physician, Medical history, Diagnosis and Recommendations.

The second page (Figure 72 - Second page of Sleep Report) presents the study results including Sleep Summary, PAT Respiratory Disturbance Index (pRDI), PAT Apnea Hypopnea Index (pAHI), PAT Central Apnea Hypopnea Index (pAHIc)*, Number of events, Percentage of total sleep time with Cheyne-Stokes Respiration (%CSR)*, Oxygen Saturation Statistics including the Oxygen Desaturation Index (ODI), mean, maximum and minimum oxygen saturation levels, Oxygen Saturation analysis, Pulse Rate statistics, REM Sleep analysis and AHI Severity Graph.

The third page (Figure 72 - Second page of Sleep Report) presents graphical displays of Respiratory Events, Snore and Body Position chart (in case a Snore/Body Position sensor was used), Oxygen Saturation, Pulse Rate, PAT amplitude, Wake/Light Sleep/Deep Sleep and REM stages.

The fourth page (Figure 72 - Second page of Sleep Report) presents pie charts of sleep/wake states and sleep stages, Sleep Latency, REM Latency, Number of wakes and Sleep Efficiency and Snore and Body Position statistics (in case a Snore/Body Position sensor was used).

The fifth page (Figure 75 – Fifth Page of Sleep Report (Optional)) is an optional page which is not printed by default. To have this page printed, you must first select the option "Oximetry and Pulse Rate Histogram Page in Sleep Report" in General Settings (see section 3.3.7). The following oximetry and pulse rate histograms will be displayed on this page:

- Oxygen saturation distribution: Number of seconds for each oxygen saturation value
- Number of desaturation events for each oxygen saturation resaturation peak point
- Number of desaturation events for each desaturation depth
- Number of desaturation events for each minimum point at desaturation event (nadir)
- Pulse rate distribution: Number of seconds for each pulse rate value

If there is no valid sleep time, the histograms will be empty.

*pAHIc and %CSR data are supplied in case RESBP sensor was used. The presentation of pAHIc and %CSR is subject to regulatory approval in the country.

Definitions:

Sleep Time: Total time in hours, during which the patient is asleep.

PAT Respiratory Disturbance Index (pRDI): the estimated number of respiratory events divided by the valid sleep time. Provided in Respiratory Events/Hour. The index is calculated during "All Night", REM and Non REM valid sleep time.

PAT Apnea Hypopnea Index (pAHI): the estimated number of Apneas and Hypopneas events divided by the valid sleep time. Provided in Apnea and Hypopnea events/Hour. The index is calculated during "All Night", REM and Non REM valid sleep time.

PAT Central Apnea Hypopnea Index (pAHIc): the estimated number of Central Apneas and Hypopneas events divided by the valid sleep time*. Provided in Central Apnea and Hypopnea events/Hour.

Percentage of total sleep time with Cheyne-Stokes Respiration (%CSR): the estimated percentage of CSR pattern from the valid sleep time*.

*valid sleep time in pAHIc and %CSR might be different from the valid sleep time used in the calculation of other indices, as the RESBP sensor needs to be valid for this calculation as well.

Oxygen Desaturation Index (ODI): the number of oxygen desaturation events (set value of 3% or 4% minimum desaturation) divided by the valid sleep time. Provided in Desaturation Events/Hour. The index is calculated during "All Night", REM and Non REM valid sleep time.



Note

In the **Oxygen Saturation Statistics** section, the first column of the **Oxygen Destur.** % area will be titled "3-9" or "4-9", depending on whether the ODI Index was set according to 3% or 4% (see section 3.3.6).

REM % of Sleep Time: REM sleep stages as percent of total sleep time.

Snore level in dBs: Because snoring can be a sign of sleep apnea, zzzPAT provides snore statistics. The threshold is determined according to DB. The amount of snoring is calculated as the percentage of sleep time over the specified DB threshold. The snoring volume level is graphically displayed (40 - 70 dB range).

Body Position: Five body position levels are graphically displayed (supine, right, left, prone and sit). Because the frequency of apneic events during sleep depends on patient position and sleep stage, zzzPAT provides information about the duration of sleep per each position – supine, prone, left, right, and sit. The corresponding percentage of time spent in each sleep position is displayed in a graph. Moreover, all recorded events such as respiratory disturbance index (pRDI), apnea/hyperpnoea index (pAHI), and desaturation index (ODI) are also provided in the report for each body position as well as non-supine position.

The Snore/Body Position data is displayed in a chart as shown here:

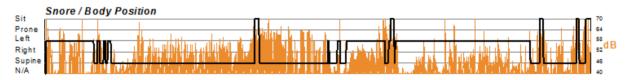


Figure 69 - Snoring and Body Position Statistics

AHI Severity Graph: Indicates the severity of obstructive sleep apnea (OSA). According to the American Academy of Sleep Medicine (AASM) it is categorized into mild (5-15 events/hour), moderate (15-30 events/hr), and severe (>30 events/hr). The severity categorization limits can be modified, see <u>Setup>General Settings>Analysis/Statistics</u> <u>Parameters</u>.

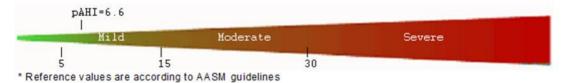


Figure 70 - AHI Severity Graph

Patient Infor	mation				
First Name:	John	Last Name:	Smith	ID:*	123456
Birth Date:	1/ 1/1942	Age:	73	Gender:	Male
Insurer:		BMI:	28.7 (W=211 lb,	, H=6' 0")	
Neck Circ.:	18 "	Epworth:	15		
Address:					
Sleep Study	Information				
Study Date:	10/12/2015	S/H/A Version:	4.6.69.3 / 3.2220	0/69	
Referring Ph	ysician Information	n			
First Name:	Jim	Last Name:	Lark	E-mail:	
Work Phone:		Mobile Phone:		Fax:	
Medical Info	rmation				
Day time sleep Frequent arou Morning heada	biness sals				
Day time sleep Frequent arou Morning heada Medications	biness sals				
Day time sleep Frequent arou Morning heada <u>Medications</u> Aspirin	biness sals aches				
Day time sleep Frequent arou Morning heada <u>Medications</u> Aspirin Summary &	piness sals aches Diagnosis	sleep apnea. It is eviden	t by frequent event:	s associated with ox	ygen desaturation.
Day time sleep Frequent arou Morning head: <u>Medications</u> Aspirin Summary & The patient is	oiness sals aches Diagnosis suffering from severe	sleep apnea. It is eviden	t by frequent events	s associated with ox	ygen desaturation.
Recommend Clinical correl	Diness sals aches Diagnosis suffering from severe lations ation is advised. ion is highly recommen		t by frequent event	s associated with ox	ygen desaturation.
Day time sleep Frequent arou Morning head: <u>Medications</u> Aspirin Summary & The patient is Recommend Clinical correl Weight reducti	piness sals aches Diagnosis suffering from severe lations ation is advised. ion is highly recomment.		t by frequent event: Signature:	s associated with ox	



-	Sleep Study Report	

Sleep Summary				Oxygen Saturation Statistics							
Start Study Time: 10:13:51 PM		Mean:	95 N	/inimum:	89	Maximu	m:	100			
End Study Time: 4:23:53 AM			Mean of Desatu	ratior	ns Nadirs (%)				93		
Total Recording Time: 6 hrs, 10 min											
Total Sleep Time 4 hrs, 56 min		Oxygen Desatu	ır. %:		4-9	10-20	>20	Total			
% REM of Sleep Time: 27.0		27.0	Events Number			13	1	0	14		
Respiratory Indices			Total			92.9	7.1	0.0	100.0		
Tota	I Events	REM	NREM	All Night							
pRDI:	29	8.4	5.1	6.0	Oxygen Satura	tion	<90	<=88	<85	<80	<70
pAHI 4%:	19	5.3	3.4	3.9	Duration (minut	es):	0.0	0.0	0.0	0.0	0.0
ODI 4%:	14	4.6	2.3	2.9	Sleep %		0.0	0.0	0.0	0.0	0.0
pAHIc 4%:	0	0.0	0.0	0.0							
% CSR:	0.0				Pulse Rate Stati	stics	during Sleep) (BPM)			
					Mean:	67	Minimum:	52	Max	imum:	89

Indices are calculated using technically valid sleep time of 4 hrs, 51 min. Central-Indices are calculated using technically valid sleep time of 4 hrs, 43 min.

pRDI/pAHI are calculated using oxi desaturations ≥ 4%

pAHI=3.9 Mild Moderate Severe I I 5 15 30


* Reference values are according to AASM guidelines


4/1/2018,Test47,5/17/1970,Female

Page 2 of 4 Rev.443

Printed on:6/28/2021

*The Automatic Analysis events or stages have been edited.

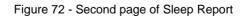

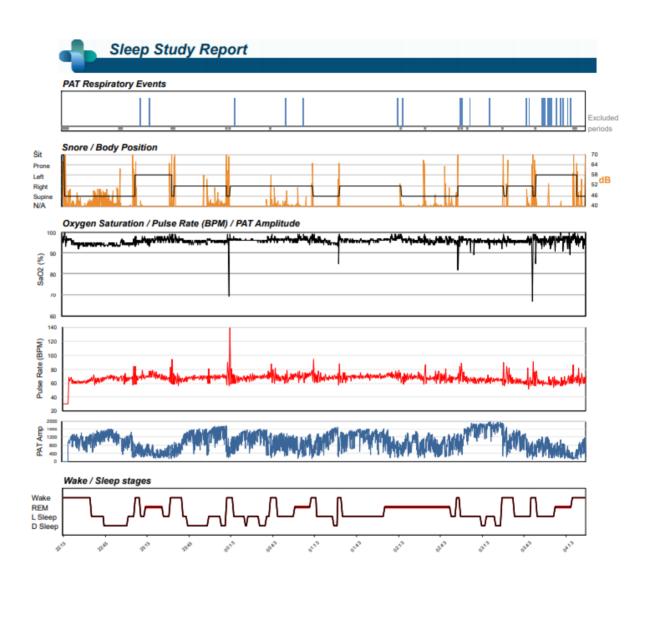

4/1/2018,Test47,5/17/1970,Female Page 3 of 4 Rev.443 Printed on:6/28/2021
*The Automatic Analysis events or stages have been edited.

Figure 73 - Third page of Sleep Report




man and an and a state of the

Sleep Study Report

Position	Supine	Prone	Right	Left	Non-Supine			1
Sleep (min)	84.5	0.0	165.5	46.0	211.5	Left	$\langle \rangle$	
Sleep %	28.6	0.0	55.9	15.5	71.4	Len	<u> </u>	tight
PRDI	0.7	N/A	4.0	22.3	8.0	Sup	ine	
pAHI 4%	0.0	N/A	1.8	18.4	5.4			/
ODI 4%	0.0	N/A	0.7	15.7	<mark>4.</mark> 0			
noring Statistics								
Snoring Level (dB)	>40	>50	>60	>70	>80	>Threshold (45)	Mean:	40
Sleep (min)	40.2	1.7	0.3	0.0	0.0	6.2		
Sleep %	13.6	0.6	0.1	0.0	0.0	2.1		

Sleep Stages Chart

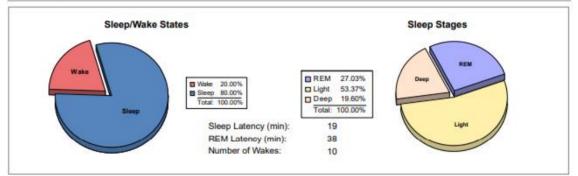

4/1/2018,Test47,5/17/1970,Female *The Automatic Analysis events or stages have been edited. Page 4 of 4

Rev/443

Printed on:6/28/2021

Figure 74 - Fourth page of Sleep Report

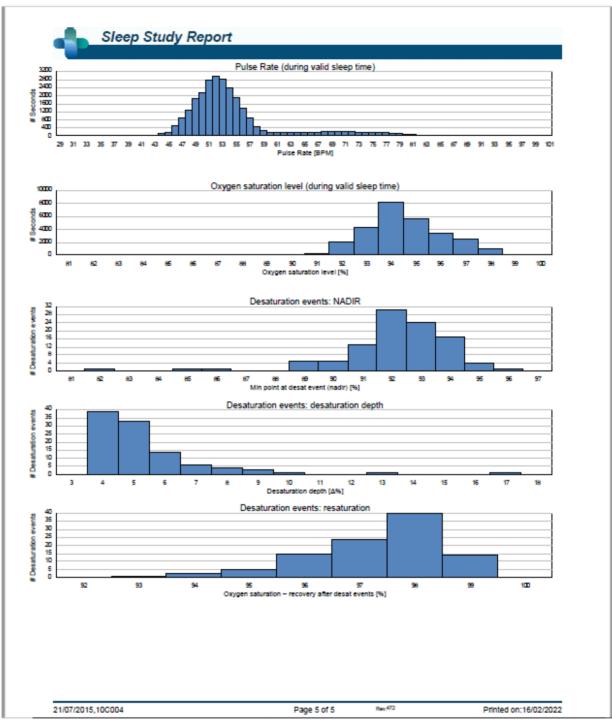


Figure 75 – Fifth Page of Sleep Report (Optional)

4.6.3 Report>Sleep Report for Selected Time Range

This option generates a two-page report that provides a summary of the subject's sleep study in a **Selected Time Range** that is selected by the user.

The Selected Time Range report has the following title on the header of each page:



Figure 76 - Sleep Report for Selected Time Range Title

To generate a report for a selected time range:

- Highlight the desired section of the waveform in the Signals Display Window, by clicking and dragging the mouse.
- Click Report>Sleep Report for Selected Time Range.

The first page of the report includes Patient Information, the Sleep Study Information, Referring Physician, Medical history, Diagnosis and Recommendations.

The second page of the report presents the study results for the **Selected Time Range**, including Sleep Summary, PAT Respiratory Disturbance Index (pRDI), PAT Apnea Hypopnea Index (pAHI), PAT Central Apnea Hypopnea Index (pAHIc)*, Number of events, Percentage of total sleep time with Cheyne-Stokes Respiration (%CSR)*, Oxygen Saturation Statistics including the Oxygen Desaturation Index (ODI), mean, maximum and minimum oxygen saturation levels, Oxygen Saturation analysis, Pulse Rate statistics ,REM Sleep analysis and AHI Severity Graph.

The third page presents graphical displays of Respiratory Events, Oxygen Saturation, Pulse Rate, PAT amplitude, Wake/Light Sleep/Deep Sleep and REM stages, Snore and Body Position chart (in case a Snore/Body Position sensor was used).

The fourth page presents pie charts of sleep/wake states and sleep stages, Sleep Latency, REM Latency, Number of wakes and Sleep Efficiency and, Snore and Body Position statistics (in case a Snore/Body Position sensor was used).

*pAHIc and %CSR data are supplied in case RESBP sensor was used. The presentation of pAHIc and %CSR is subject to regulatory approval in the country.




Note

In the Selected Time Range report, the Start and End Study Times are the margins of the Selected Range.

4.6.4 Report>Event Report

This report provides statistics on different event types identified by the zzzPAT automatic analysis and by the User. A graphical representation provides a quick way of looking at the event distribution, and the summary section provides statistical information. When displayed

on screen, the user can double-click on a particular event name (on the relevant row below the chart) to get a detailed list of all the events of this type (see Figure 77 - Event Report).

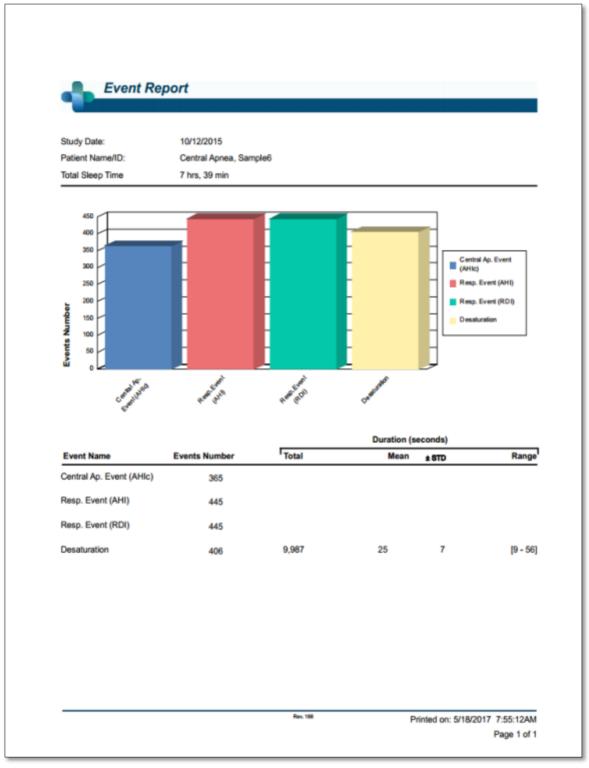




Itamar Medical Ltd.

	Time Start	<u>Time End</u>	<u>Time Duration (s)</u>	<u>Percent</u>	
Desaturation	1:00:46	1:21:22	10	4	
Desaturation	1:59:40	2:20:34	28	4	
Desaturation	2:00:17	2:20:53	10	4	
Desaturation	2:02:01	2:22:56	29	6	
Desaturation	2:03:17	2:23:57	14	4	
Desaturation	2:05:17	2:26:03	20	4	
Desaturation	2:13:41	2:34:19	12	4	
Desaturation	2:17:05	2:37:46	15	4	
Desaturation	5:58:06	6:18:51	19	4	
Desaturation	5:59:27	6:20:24	31	5	
Desaturation	6:02:44	6:23:44	34	5 5	
Desaturation	6:04:38	6:25:17	13	5	
Desaturation	12	235	20	9	[10 - 34]

Figure 78 - Detailed Event Report

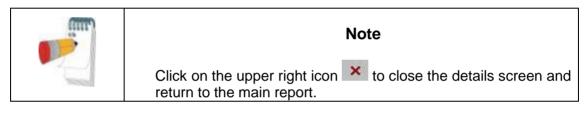

4.6.5 Report>Sleep Indices

This report provides a summary of study results, including pRDI, pAHI, ODI and Sleep Time.

2zzPAT Report			Х
pRDI: 21.7 pAHI (3%): 15.4 pAHIc (3%): 0.4 ODI (4%): 3.9 Sleep Time: 5 hrs, 57 min	%CSR:	0.0	^
	Rev 40		~
<		>	



Note
Generation of the "Sleep Report" will close the "Sleep Indices" report.

4.6.6 **Report>Patient Follow-up Report**


This report provides a way of comparing multiple studies for the same patient. A graphical representation of the pRDI, pAHI and ODI for the different studies provides a quick way of determining a trend through the studies.

The Sleep % over the defined Snoring threshold dB will be displayed as well.



Note

Unreviewed studies will not be presented in the Patient follow up report (see section 4.2.3).

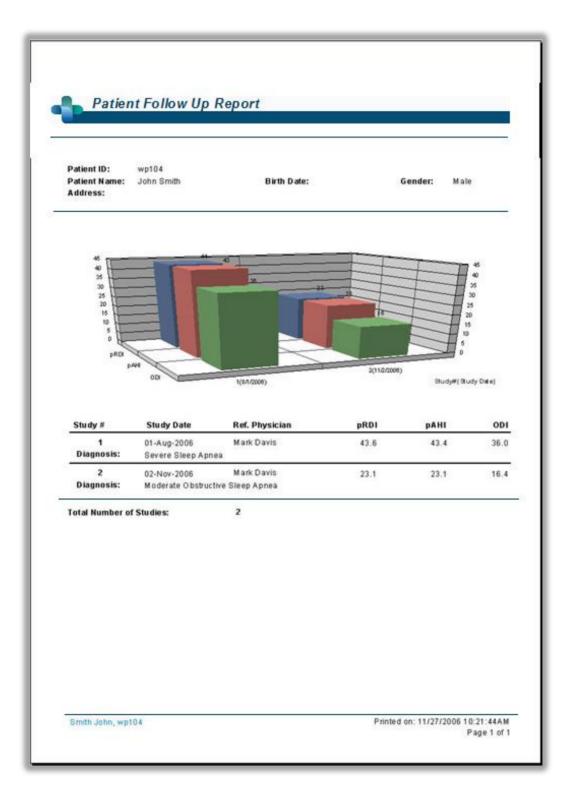



4.6.7 **Report > Report for Patient**

This report is designed in the form of a letter addressed to the patient that informs the patient about the sleep test results. It provides the following patient details:

- Total sleep time
- Apnea/Hypopnea index (AHI)
- Respiratory disturbance index (RDI)
- Desaturation index (ODI)
- Rapid eye movement (REM)

It also compares these to the normal average indices.



Note

The zzzPAT installation will install 3 patientletter.ini files: one for male, one for female, and the default currently used. If the patient's gender is defined, then the relevant template will be displayed when producing the patient report in a language which has gramatical gender; however, if the patient's gender is not defined then the "male" template will be displayed.

To produce the report for the patient:

- 1. In the Report menu, choose Report for Patient.
- 2. Print the report by clicking on the Printer icon

Name of Patient:	John Snow
Name of Referring Doctor: Date of Study:	John Smith 2002-01-10
Dear John Snow,	
l would like to thank you for und review with you the results that	dertaking the Watch-PAT ambulatory sleep study. I would now like to you received.
was 10 hrs, 16 min. But as we a	calculated from the time you turned the device on until you pulled it off) are testing for obstructive sleep apnea and are interested in the time I sleep time was 6 hrs, 21 min. On average, a person with normal sleep urs a night.
Looking at your sleep statisti	ics we can learn some vital information:
of apnea and hypopnea events	bbreviated as AHI) was found to be 51.5. This score is the average numbe per hour of sleep during the night. Apnea event is an episode of fully n 10 seconds, and an hypopnea event is an episode of partial occlusion. core is 5 or lower.
way as the AHI but an additiona abbreviation for Respiratory Eff	dex (RDI) was found to be 52.5. This score is calculated in a very similar al type of respiratory events named RERA are also counted. RERA is the ort Related Arousal and is essentially a very short arousal of a few lusion of the airways. The normal range of the RDI score is also 5 or
the oxygen in your blood dropp	c (ODI) was found to be 33.7. This score is the average number of times ed by 4% or more during an hour of sleep. As you can imagine, the more ne heart must work. The normal range of the ODI is 5 or lower.
your sleep time. REM sleep is v nurtured. The normal range of t	ge (Rapid Eye Movement) we find that you were in this stage 0.0% of ery important as during this stage your body is being replenished and the REM score is between 15 to 30% pending some other conditions. we the time during which we dream.

If I can be of any further assistance, please contact my front desk to make an appointment.


Regards,

Printed on: 2016-03-14

Figure 81 - Report for Patient

4.6.8 Report > Detailed Report

This report's first page is the same as the first page of the sleep report (see Figure 71). The rest of the report's pages present graphical displays of Respiratory Events, Snore and Body Position chart (in case a Snore/Body Position sensor was used), Oxygen Saturation, Pulse Rate, PAT amplitude, Wake/Light Sleep/Deep Sleep and REM stages. Each page represents one hour of sleep time.

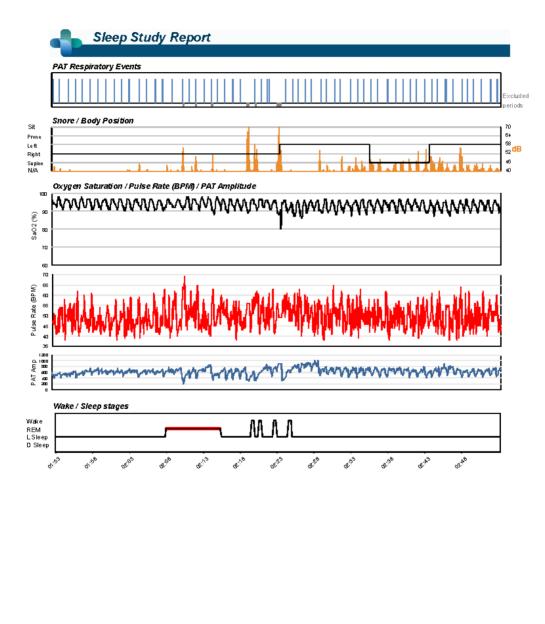


1/18/2022,FTP Study	4	Reu. 27	Printed on: 2/24/2022

Figure 82 – Sample Page from Detailed Report

4.6.9 Report > Multi-night Summary Report

This summary report presents multi-night sleep statistics. It is enabled only if the currently opened study contains WP1 multi-night data (see section 4.1.2.2 regarding preparing a new WP1 study with multi-nights option selected). The report includes up to 3 studies (one study per column), each with the same WP1 device information and patient statistics information. A column with averages of the studies' values (see below) is also included.

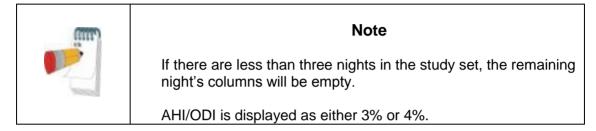

In the Average column, AHI/ODI values will consist of the average of just the nights with the same AHI/ODI configuration. For example, given a three-night study with the following AHI/ODI values: Study1 - AHI3%/ODI3%; Study2 - AHI3%/ODI3%; Study3 - AHI3%/ODI4%. The average column will be of Study1 and Study2 only, and the Average heading will indicate "AHI3%/ODI3%". In the rest of the Average column, average values will be computed from available nightly values, and N/A values will be ignored. If all nightly values are N/A, the average will be empty.



Note

Usually the AHI/ODI percent configuration is defined once per site and would thus be the same for all the studies.

Sleep Summary				
Night #	1	2	3	Averag
Study Date	11/03/2014	12/03/2014	13/03/2014	Average
Oxygen Desatur. %	AHI 3%, ODI 3%	AHI 3%, ODI 3%	AHI 3%, ODI 3%	AHI 3%, ODI 39
Start Study Time	23:34:56	23:23:19	23:31:13	
End Study Time	9:20:08	9:08:31	8:11:02	
fotal Recording Time	9 hrs, 45 m in	9 hrs, 45 min	8 hrs, 39 m in	9 hrs, 23 mi
otal Sleep Time	8 hrs, 26 m in	7 hrs, 46 min	7 hrs, 34 m in	7 hrs, 55 mi
Sleep Latency (min)	6	5	20	1
REM Latency (min)	226	261	53	18
RDI	9.3	9.8	12.8	10.
AHI	0.7	1.3	0.7	0.
DDI	0.7	1.3	0.7	0.
REM/NREM pRDI	19.2/6.5	20.6/6.7	24.3/9.0	21.4/7.
REM/NREM pAHI	2.2/0.3	3.9/0.6	1.1/0.5	2.4/0.
REM/NREM ODI	2.2/0.3	3.9/0.6	1.1/0.5	2.4/0.
DAHIC				
REM/NREM pAHIc % CSR				
o con Dxygen Saturation Statistics (%)				
Mean	97	97	97	9
Minimum /Maximum	95 / 100	93/99	95/100	94 / 9
Mean of Desaturations Nadirs	96	96	96	9
Dxygen Saturation Duration (minutes)	/ Sleep %			
<90	0.0/0.0	0.0/0.0	0.0/0.0	0.070.
<=88	0.0/0.0	0.0/0.0	0.0/0.0	0.0/0.
<85	0.0/0.0	0.0/0.0	0.0/0.0	0.0/0.
<80	0.0/0.0	0.0/0.0	0.0/0.0	0.070.
<70	0.0/0.0	0.0/0.0	0.0/0.0	0.0/0.
Oxygen Desatur. Events Number / Sle	ер %			
3 4 - 9	0/0.0	0/0.0	0/0.0	0/0.
10 - 20	0/0.0	0/0.0	0/0.0	0/0/
> 20	0/0.0	0/0.0	0/0.0	0/0.
PulseRate (BPM)				
Mean	61	61	61	6
Minimum Maximum	43 / 107	41/105	39 / 99	41 / 103
Sleep Stages (%)				
REM	21.6	20.1	25.0	22.3
_ight	56.8	55.3	53.7	55.
Deep	21.6 86.5	24.7 79.7	21.3 87.4	22 / 84 /
Sleep % SnoringDuration (minutes) / Sleep %	00.5	r.v.r	07.4	04.
> Threshold (0)				
Mean (dB)				
Body Position Statistics				
Supine Duration (minutes) / %				
Von-Supine Duration (minutes) / %				
Prone Duration (minutes) / %				
Right Duration (minutes) / %				
.eft Duration (minutes) / %				
Supine pAHI/pRDI/ODI				
Non-Supine pAHI/pRDI/ODI				
Prone pAHI/pRDI/ODI				
Right pAHI/pRDI/ODI				
.eft pAHI/pRDI/ODI				

Figure 83 - Multi-night Summary Report

4.6.10 Printing

The study signals recorded by the WP and the zzzPAT analysis can be printed by: Either clicking on the print icon in the toolbar or selecting **File>Print**. User has a number of options for printing (see Figure below):

- Printing the entire study
- Printing the screen
- Printing specific sections defined by time range
- Printing specific channels

Print	? — × —
Printer	
Name: Name:	-SRV\P_1_Back_LexT644_PCL
	documents waiting
Type: Lexmark	
Where: 1 floor, 22 Comment:	23 room Liat Asaf
Comment.	
Print Range	Options
C AI	Scale signals to fit vertically
G Screen	Print events
C Time from:	: : Print gridlines
to:	
🗹 Rela	tive Time
Channels	
PAT Amplitude	V PulseRate
☑ SaO2	🔽 Actigraph
✓ WP Stages	
	OK Cancel

Figure 84 - Print dialog box

5 Exporting Data

5.1 File>Export Data

This option enables the user to export and save recorded signals data either in binary or EDF format.

The export data as "second by second CSV" creates a comma separated file with one line per each second of the study. The file contains information about the sleep stages, SaO2, pulse rate, snore level, body possition, PAT amplitude, different events type and the analysis excluded times in each second.



Note

The pulse rate in this format is multiplied by 100, so 6000 stands for 60 pulse per minutes. SaO₂ level of 127 indicates a non read at that second.

5.2 File>Export Events - Creating *.txt File

This option enables the user to export marked events into ACSII format. It creates an 'Event.txt' file that lists all events in a study by time of occurrence.

5.3 File>Export Manual Events

This function is used by sleep specialists working with CloudPAT web service. It enables the user to export manually modified events.

5.4 Tools>Export/Delete


See section 6.1.


5.5 Tools>Export General Settings

See section 6.5.

5.6 Transferring a Study to Itamar Medical

This option is used to save a study in a ZIP file, for transferring to Itamar Medical's support department. If zzzPAT has Internet access, this function allows the user to send the file directly, without saving first.

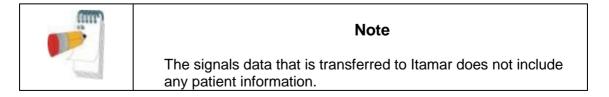

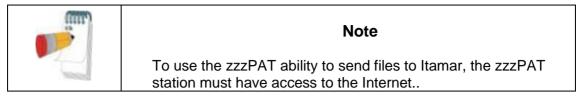

A study can be saved or transferred only if it is displayed in the 'Signals Display Window' or is stored on a device.

- Select a study:
 - o by displaying it in the 'Signals Display Window'; or
 - by connecting the device
- From zzzPAT click Help>Pack and Send Study to Itamar Medical...

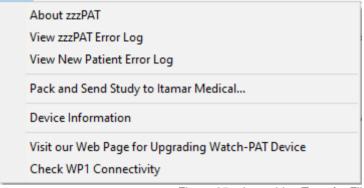


Figure 85 - Launching Transfer Files

The following Dialog Box appears:

	mandatory)		<u>S</u> end	
S <u>u</u> bmitter Name:		Site <u>N</u> ame:	Sa <u>v</u> e	
<u>E</u> -mail:		P <u>h</u> one:	Clear	2
Problem Type	Other		<u>↓</u> <u>C</u> lose	
Problem	1		~	
Description:			5	
Data Source				
	Opened Study (WP S/	N:104446620(SBP=1800686v4.0.2))		
		N:104446620(SBP=1800686v4.0.2))		
C Send Current		N:104446620(SBP=1800686v4.0.2))		
C Send Current C Send Study f □	om WatchPAT	N:104446620(SBP=1800686v4.0.2))		
 C Send Current C Send Study f □ With Wa □ Send Watch 	tom WatchPAT tchPAT Device Logs	N:104446620(SBP=1800686v4.0.2))		

Figure 86 - Prepare and Send Study Dialog Box

- The last entered information is shown by default. Select **Clear** to clear all fields
- Fill in the mandatory fields
- Select the relevant problem type from the dropdown list:

Problem Type	Other	-
Problem Description:	Other Chest Sensor - Missing Snore/Body Position data Chest Sensor - "Sensor might be faulty" warning Study - No indices Study - Short study Study - Short technically valid sleep time Study - Many exclusions in PAT signal Study - Many exclusions in Oximetry signal Study - No sleep stages	

Figure 87 - List of Problem Types

- Check the radio button to indicate the desired option:
 - Check **Send Current Opened Study** if the case study is currently displayed on the Signal Window.
 - WP200(U)/WP300: Check Send Study From WatchPAT if the case study is stored in the WP.
 - WP200(U)/WP300: Check Send WatchPAT Device Logs in case the internal logs from the WP device needs to be sent to Itamar for troubleshooting.

- WP1: Check Send WP-ONE Device Serial Number for any inquiry.
- To save the study click **Save...**. The following dialog box appears:

Browse for Folder				
Select folder for zzzPAT Packed Data				
My Computer				
📃 Desktop 🔺				
Network				
⊿ 📜 Libraries 🗉				
Music				
Videos				
My Videos				
Public Videos				
a 🖹 Documents				
My Documents				
Public Documents				
OK Cancel				

Figure 88 - Saving Case Study

Select the destination for the saved file. The file will have the following name:
 zzzPATPack_DT.zzp

Where

- o D is the current date
- o T is the current time
- **To Send** a study to Itamar automatically
 - Click **Send...**. The following screen appears:


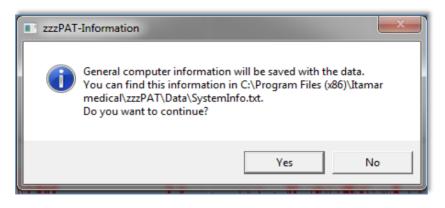


Figure 89 - Sending Study confirmation

• Click YES to continue.

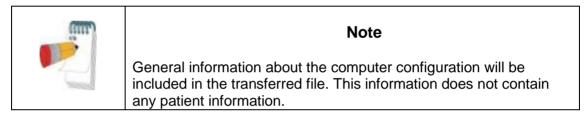

The progress of the file transfer will be reported on the following screen:

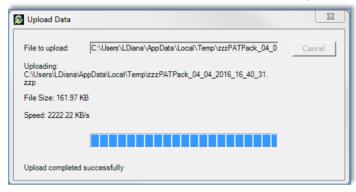


Figure 90 - Data Transfer Progress

Upon completion of the file transfer the following screen will appear:

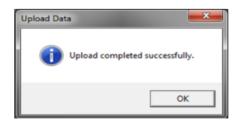


Figure 91 - Data Transfer Successfully Completed

- Click **OK** to continue.
- **To Send** a study to Itamar manually If the zzzPAT station does not have Internet access, or the automatic send failed to complete, the file can be sent manually using the following procedure:
- Save the file as described in Section 5.6
- Copy the file and transfer it manually by your preferred method.

5.7 Displaying WatchPAT Device Information (WP200(U)/WP300)

From zzzPAT click **Help>Device Information** in order to display device information. A dialog box appears with the following information:

- Number of studies made with the connected device (supported from device firmware version >2.2191 or above)
- Probe S/N (supported from device firmware version >2.2191 or above)
- Device S/N

The information can be taken either from the opened study or from the connected WP device. The information will be "N/A" in case of not supported firmware versions.

Device Information		x
C From Current Opened Study	From Connected Device	
	51596 650051113	Refresh
Recorded Studies With Device(WP200): 7	7	
	рк	

Figure 92 – Device Information Dialog Box

5.8 Upgrading WatchPAT Device firmware (WP200(U)/WP300)

From zzzPAT click **Help>Visit our Web Page for Upgrading Watch-PAT200 Device** in order to open a link to Itamar's web page and download the latest firmware version.

6 Tools

6.1 Tools>Export/Delete


This dialog allows the user to select the data of specific studies from the zzzPAT database and extract it to an archive database. This action enables freeing space on the PC hard disk, if needed, and archiving the data in other locations. Upon selection, the 'Export' dialog box will open:

ID	First Name	Last Name	Study Date	Load Date	Status	CHF	Hypertensi N	Select All
546546546 124325161	John Sara	Snow Rozen	2014-03-22 2002-06-22	2016-03-14 2016-03-14	Reviewed Reviewed	In2312	ltamar	Unselect Al
								Study Selectio
								Show All Stud
								Start Export.
۲			III				•	Start Archive.
	it of 2 total stu	dies.	m	OK	B out of 40960) KB total (esti		Start Archive.
	it of 2 total stu	dies.	m	OK	B out of 40960) KB total (esti		Start Export. Start Archive. Start Delete. Export to CSV Export for Interpretation



This dialog box allows to export, archive or delete study data:

- **Export** function: Creates a file containing a copy of the selected study data as well as User's information for users associated with the exported files. Original files are not removed from the database.
- **Archive** function: Creates an archive file of the selected study data as well as User's information for users associated with the exported files, and removes the original data from the database.
- **Delete** function: Deletes the selected files from the database.
- **Export to CSV** function: Creates a file named zzzPAT.csv, which contains in one line all the Patient, Study, Physician, and Analysis fields information. In case that a study has two different analysis, the data will be exported in two lines; one line for each entry.

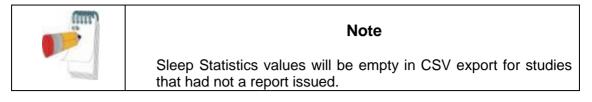

- The Delete function has two options, as seen in Figure 94.
- Delete files and save a temporary copy. The deleted files are saved to a temporary folder, overwriting the previously saved files from the last time the function was used and the option to save a temporary copy was chosen (this function always uses the directory ...\Itamar Medical\zzzPAT\Data\zzDELETE to save the deleted files).
- Delete studies permanently. The deleted files are deleted permanently.

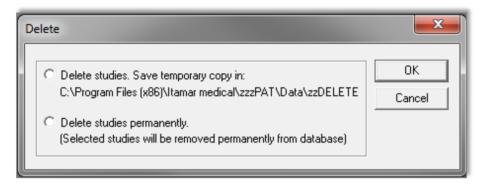


Figure 94 - Delete studies options Dialog box

- Temporarily saved deleted files can be restored by using the Import function of the Database Tools (see section 7.1.2).
- The temporarily saved file can be deleted from the hard disk (e.g. to free disk space) by following one of the below listed procedures:
- Delete one more study and select "Delete studies. Save temporary copy in:". By doing so the previously saved deleted studies will be replaced by the single study deleted now, thus reducing the size of the saved files to one study only.
- Using Windows Explorer navigate to the folder that contains the saved deleted files (\Program Files\Itamar Medical\zzzPAT\Data\zzDELETE) and delete its content in its entirety.

To commence Archive Export or Export to CSV operations proceed with the following:


- Name the archive (optional) in the 'Name' editing box (Name can consist of up to 8 characters).
- Add comments, if desired, in the 'comments' editing box.
- The exported/archived files will be saved in a folder named in accordance with the following convention:
- Exported files: 'zzExport_name_dd_mm_yy_hh_mm' with "name" being the optional data entered by the user

- Archived files: 'zzARCHIVE_name_dd_mm_yy_hh_mm' with "name" being the optional data entered by the user
- Export to CSV files: 'zzzCSVExport_name_dd_mm_yy_hh_mm' with "name" being the optional data entered by the user
- You can apply a filter to the files listed in the main field, by clicking **Study Selection...** The following dialog box will open:

Select Studies By					X
By Loaded Time Betweer By Study Time Between	2016-03-14 💌	and and	2016-03-14	-	Apply All Studies
 By Patient ID By Patient Last Name 					Close
Status Unreviewed Reviewed Reported	Custom Fields CHF Hypertension MSLT			×	

Figure 95 - Study Selection Dialog box

• To select studies by a specific criterion, check the box to the left of the desired criterion. In addition user can select whether to export studies by status: Unreviewed, Reviewed and Reported and by user predefined **Custom Fields** (see section 3.3.3). It is possible to select several criteria by which to export a single group of studies.

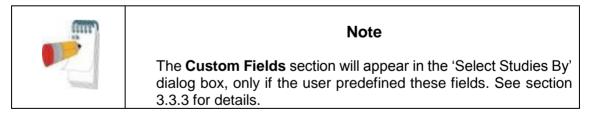

- Select the studies you wish to export/archive/delete from the main field in the Export dialog box. You can click **Select All** to select all files, or you can select individual files by shift or control clicking on them.
- Once ready, click:
- Start Export to export the selected studies, or,
- Start Archive to archive the selected studies, or,
- Start Delete... to delete the selected studies, or,
- Export to CSV to export the selected studies to CSV format.

- Click **Exit** to close the Export dialog box.
- **Export for Interpretation** function: Prepares and copies the raw data files as one zipped file for later ftp upload to a secure server for interpretation services.

6.2 Tools>Import

Use to import previously archived studies that were removed from the zzzPAT database, back into the database. Upon selection the 'Import' dialog box will open.

(mr)	Note
	Together with the imported studies zzzPAT imports user information of users associated with the imported files if such users are not defined in zzzPAT. These users are imported with Basic User Permissions only (see Section 2.6.2).

port									- • ×
			op\zzEXPO	RT_2016-03-1	I4_07_22_03_P№	N	Browse	<u></u>	
	ged in Databas								
Name	No.Studie			reated by	Comment			^	Archive Selection.
	1 1 1 2	2015 2015	-12-23 Si -12-24 Si	uperUser uperUser uperUser uperUser				в	
name 5	ŝ			uper User	comment				
1.44	ĩ			ine la	Massassas			*	
*			_	m				•	
Selected expo	uted studies:								
			Churt Du	ste Load Da	te Status	CHF	1	1	
ID	First Name	Last Name	1 2000 0 5	se Load Da	re status	Unr	Hyperten_	MSLT	Select All
	1	Last Name Snow	2014-03			In2312	Hyperten Itamar	MSLT	
ID	1						1	MSLT	Select All
ID 546546546	1			22 2016-03			1		Unselect All
ID	1						1	MSLT	
ID 546546546	1	Snow	2014-03-	22 2016-03		In2312	1		Unselect All

Figure 96 - Import Dialog Box

It is possible to locate the archived studies you wish to import by using the **Browse** option or by selecting them from a list of logged archives in the database. If previously deleted files were deleted using the "Save a temporary copy" option, selecting the Last Deleted Studies option will allow retrieval of these studies.


Locating an Archive by browsing:

• Select the 'Folder' radio button in the 'Import' dialog box. The **Browse** button will become enabled.

• Click on the **Browse** button and select the file you wish to import.

Locating an Archive from a list of logged Archives in the database:

By default all archives will be displayed in the 'Old Archives Logged in Database' list. The user can filter the archives displayed in this list by certain criteria.

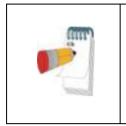

Note

If the database was overwritten while upgrading zzzPAT or the database was erased due to restoring a back up database into it, no archives will displayed in the "Old Archives Logged in Database' Field.

To filter the archives that will be displayed in the 'Old Archives Logged in Database' field:

- Select the "Old Archives Logged in Database' radio button.
- Click on the Archive Selection button. The following dialog box will open:

Image: Weight of the service of th	Select Archives by	×
	By Patient ID By Creator By Comment	All Archives

Figure 97 - Select Archive Dialog box

- Check the criteria by which you want to select the Archive.
- Insert the appropriate parameters for selection.
- Click Apply.
- Click **Close** to return to the 'Import' dialog box.
- You can restore the default display of all archives by clicking the **All Archives** button.
- Select the Archive you wish to import.

Once an Archive has been selected, all its studies will be displayed in the 'Selected exported studies' fields of the 'Import' dialog box.

- You can select individual studies you wish to retrieve by clicking on them with the mouse while holding down the **Ctrl** key on the keyboard, or you can import them all by clicking on the **Select All** button.
- Click Start Import to begin the process.

m	Note
	You cannot import a study that is already available in the database

6.3 Tools>Backup

Enables the user to copy the entire database to another location for back up. Upon selection the 'Backup' dialog box will open.

- Select the path for a backup directory by clicking on the **Browse** button, or by accepting the system default setting.
- Click **Start Backup** to begin the process.

Automatic Backup


A backup of the database for every day of the week will be automatically performed behind the scenes. Backing up this way will minimize loss of data in case of database corruption. The backup of the database will be saved as zzPATday1.bak for Sunday, zzPATday2.bak for Monday, etc. The zzPATday1.bak will be overwritten next Sunday. The files will be saved under backup folder, parallel to the Files folder (default C:\itamar medical\zzzpat\backup).

6.4 Tools>Restore

Use this application for restoring a backed-up database by overwriting the current database. Use this application to recover a database following a major database failure. Upon selection the 'Restore' dialog box will open.

Restore		X
Restore Folder:	C:\ltamar Medical\zzzpat\database_27_12_15_14_19_30\	Browse
Use Restore to o The current data	verwrite the current database with an old copy. will be LOST!	Start Restore
DB Connection: I	Local	Exit



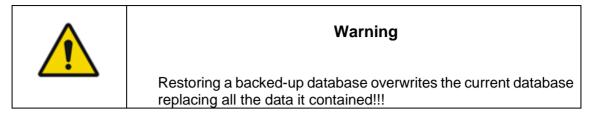

6.5 Tools>Move Files to New Location...

Allows user (Administrator) to move all the data files to a new folder and work from now on with the new files folder.

Note
The zzzPAT software uses a Files folder where all raw data and signal data files are kept. Each study needs around 50 MB disk space. The Files folder path used is kept inside the SQL Server database that holds all patient, study and analysis information and can only be modified using this function.

Move Files	X
Current Files Folder: \\DBFolder\zzzPAT	
New Files Folder:	Browse
The files will be copied to new selected folder. The previous files folder will be renamed to: C:\temp_14_03_16_19_24_48	Start Move
DB Connection: Local	Exit

Figure 99 – Move Files dialog box

"Current Files Folder" – Displays the current Files Folder.

"New Files Folder" – Use the Browse button in order to select a new Files folder "Start Move" – Implements the move (relocating) of all data files to the new Files folder. The previous files folder name is changed to Files_Date_Time (Date-dd_mm_yy Timehh_mm_ss)

6.6 Tools>Export General Settings

Export General Se	ttings	X
Export Folder:	C:\Itamar Medical\zzzpat\	Browse
Use Export Ger	eral Settings in order to save a copy of the current settings.	Start Export
DB Connection:	: Local	Exit

Figure 100 - Export General Settings Dialog Box

- Type the directory path or click '**Browse**' button to select the directory, where the General Setting configuration file will be saved.
- Click 'Start Export'.

6.7 Tools>Import General Settings

Import General Se	ttings	
Import Folder:	C:\Itamar Medical\zzzpat\	Browse
Use Import Gen of the settings.	eral Settings to overwrite the current general settings with an old copy	Start Import
The current ger	neral settings will be overwritten!	
DB Connection	Local	Exit

Figure 101 - Import General Settings Dialog Box

Click 'Browse' to select the directory of the General Setting configuration file.

Click 'Start Import' to continue.

6.8 Tools>User Administration

Please refer to Section 2.6.2

6.9 Tools>Export Activity log

Export Activity Log	×
Image: Select by Date from 6/29/2021 to 6/29/2021 Image: Select by User Login Name Image: Select by Action Image: Select by Action Image: Select by Patient ID	OK
C Select All	

Figure 102 - Export Activity log window

Export Activity log option monitors actions performed by users and that can be exported in csv format based on filters such as: Date, User login Name , Action, Patient ID or All.

Note that only one of such filters can be chosen at a time.

Example: A user with proper permissions can export all the studies that were 'Set as Reported' or he can export all the audit data regarding a specific Patient ID.


m	Note
	In cases when the action is performed on a set of studies (i.e. Export studies) the log keeps general information about actions performed but no information about specific studies. The Export Activity log option is available in Administrator mode only.

6.10 For WP1: Tools>Manage WatchPAT One Registered Devices...

On selecting this option, a dialog will display all registered devices that were not yet uploaded to zzzPAT.

- The dialog will present a list of registered devices with Patient ID, First and Last names, Registration date, Pin number and device Serial number information (see Figure 103)
- "Reset Retries Counter" resets retries counter for any device in the list
- "Hide Registered Device" hides selected device in the registered devices list
- "Show Hidden Registered Devices" displays a list of all hidden devices from the list in order to select the one device to bring it back to the list (see Figure 104)
- "Unhide Registered Device" makes selected device visible in the registered devices list

- "Show Registered Devices" displays a list of all visible devices in the registered devices list
- "Unregister device" unregisters a WP1 device that was registered but was not used for recording. An unregistered device can be registered again.
- "Send Request to Itamar Medical..." sends an information request to Itamar Medical support for this WP1 device. Selecting this option will open up the "Pack and Send Study" dialog where the problem can be described before the inquiry is sent, see 5.6.

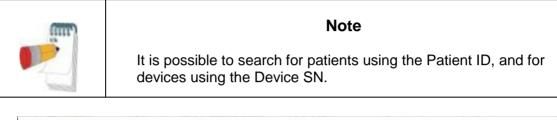

Select WatchPAT ONE Device Serial Number		×
Search by Patient ID:	Device SN: Find Find Next	Check Status
Patient ID Last Name First Name 123456789	Registration D PIN WP Device SN(ProbeSN) Status Jan-25-2022 1111 100009182(100009182)	Reset Retries Counter Unhide Registered Device Show Registered Devices Unregister device Close Send Request to Itamar Medical

Figure 103 - Manage WP1 registered devices window - visible in the list

Search by	Patient ID:		Device SN:		Find	Find Next		Check Status
Patient ID	Last Name	First Name	Registration D	PIN	WP Device SN(ProbeSN)	Status	1	Reset Retries Counter
p4			Feb-08-2022	4444	104245204			
p5			Feb-08-2022	5555	104245436			Hide Registered Device
6			Feb-08-2022	6666	104245956			
57			Feb-08-2022	7777	104246129		-	Show Hidden Registered Device
P8			Feb-08-2022	8888	104246707			
9			Feb-08-2022	9999	104246806			Unregister device
510			Feb-08-2022	1010	104247093(104247093)			Unregister device
o10			Feb-08-2022	1010	104247093(104247572)			1
511			Feb-08-2022	1111	104258819(104258819)			Close
511			Feb-08-2022	1111	104258819(104258926)			
512			Feb-08-2022	1212	104258934(104258934)			
12			Feb-08-2022	1212	104258934(104258942)			
13			Feb-08-2022	1313	104258959(104258959)			
13			Feb-08-2022	1313	104258959(104259015)			
13			Feb-08-2022	1313	104258959(104259056)			Send Request to Itamar Medical.
514			Feb-08-2022	1414	104259064(104259064)			
p14			Feb-08-2022	1414	104259064(104259114)			
p14			Feb-08-2022	1414	104259064(104259213)		~	

Figure 104 – Manage WP1 registered devices window – hidden from the list

7 Database Wizard

The zzzPAT Database Wizard application can be opened by double clicking on Dbtool.exe in the BIN folder (under the Installation folder) and it contains three utilities:

Database Tools - Allows an authorized administrator to update and maintain the zzzPAT database.

User Administration - Allows an authorized administrator to add users to zzzPAT application and define the permissions attributed to them. (See chapter 6.8).

Configuration Tools - Allows an authorized administrator to Export and Import the General Settings.

zzzPAT Database Wizard - Login	
Login Type	ОК
Active Directory Service (ADSI) ZzzPAT Database	Cancel
Login 111 Password	Change
DB Connection: Local 🖃	

Figure 105 - Database Wizard Login

• Enter Login and Password, and select the desired database Connection. The Database Wizard will open.

7.1 Database Tools

Click on the 'Database Tools' icon

Database Tools Wizard	-	
	8	Export/Delete from Database
		Import to Database
		Backup
<u>ק</u>		Restore
it.		Move Files
DB Connection: Local	He	ome Exit

Figure 106 - Database Tools Wizard dialog box

7.1.1 Export From Database:

- Click the 'Export From Database' Icon
 in the 'Database Tool Wizard'
 window.
- Refer to Section 6.1 for a detailed explanation of the exporting procedure.

7.1.2 Import to Database:

- Click on the 'Import to Database' icon in the 'Database Tools Wizard' window.
- Refer to Section 6.2 for a detailed explanation of the importing procedure.

7.1.3 Backup

- Click on the 'Backup' icon in the 'Database Wizard Tools' window.
- Refer to Section 6.3 for a detailed explanation of the backup procedure.

7.1.4 Restore

- Click on the 'Restore' icon in the 'Database Wizard Tools' window.
- Refer to Section 6.4 for an explanation of the restoring procedure.

7.1.5 Move Files



- Click on the 'Move Files' icon Lel in the 'Database Wizard Tools' window.
- Refer to Section 6.5 for a detailed explanation of the Move Files procedure.

7.2 User Administration

Please refer to Section 2.6

7.3 Configuration Tools

Used to copy General Settings from one zzzPAT station to another.

Click the configuration tools Icon . The following screen appears:

Configuration Tools Wizard		×
tamar medical	Export General Settings	
DB Connection: Local	Home	Exit

Figure 107 - Configuration Tool Wizard

• Click the 'Export General Settings' icon to save the current General Settings configuration.

• Click the 'Import General Settings' icon 🗾 to apply a saved General Settings configuration.

7.3.1 Export General Settings

- Click the 'Export General Settings' icon
- Refer to Section 6.5 for a detailed explanation of the general settings exporting procedure.

7.3.2 Import General Settings

- Click the 'Import General Settings' icon
- Refer to Section 6.7 for a detailed explanation of the general settings importing procedure.

8 Troubleshooting

Installation		
Trouble	Possible Cause	Solution
zzzPAT installer fails to run.	Auto run function in Windows is not activated.	Open My Computer>zzzPAT CD and double click 'Setup.exe'.
	Windows version not compatible with zzzPAT.	Use PC with appropriate Operation System.
	Hardware configuration below minimum required.	Must have at least 128 MB RAM, and a Pentium processor for zzzPAT installer to run.
zzzPAT fails to recognize WatchPAT (WP200 only)	USB drive needs to be re-defined.	Select Setup->"Set WatchPAT Drive". Insert WatchPAT device to USB Drive. Click OK. "The system is searching for WatchPAT device" message should appear. At the end, a list containing the WP200 drive should be displayed. Select the WatchPAT drive.and click Ok.
Under Windows XP, user cannot load study or operate 'Database Wizard' utilities, despite having proper zzzPAT user permissions.	User does not have writing permission to the drives these applications are located on.	Check user's writing permissions per section, and redefine as necessary.

Table 2 - Troubleshooting, Installation

zzzPAT		
Trouble	Possible Cause	Solution
Analyze>Reload	User does not have	zzzPAT Administrator can modify user's
Study and Analyze	permission to operate	Extended Permissions. (See section
option in the	this utility.	2.6.2)
zzzPAT window is		
disabled	Insufficient free space on	Free enough disk space to exceed the
	hard disk	minimum requirement of 100MB and try
		again
Cannot Load Study	There is less than	Free enough disk space to exceed the
(function is	200MB of free hard disk	minimum requirement of 200MB and try
disabled)	space	again

	•	
zzzPAT will not	Some zzzPAT files may	Uninstall and reinstall zzzPAT.
start, or behaves	be damaged/overwritten	
unpredictably		
The open file does	File was saved with an	Run the analysis by selecting
not show REM	older version of zzzPAT	Analyze>Reload Study and Analyze
	that did not have REM	
	capabilities or REM	
	could not be calculated	
	because of the algorithm	
	restrictions.	
Cannot generate	Less than 100MB of free	Free enough disk space to exceed the
Sleep Report -	disk space	minimum requirement of 100MB and try
Sleep Report button		again
is grayed out	No study is loaded or	Open the desired study. If the study is
	study is invalid	open, it may have invalid data and
		therefore is not usable
User cannot log on	zzzPAT will not open if	Ensure no other user left an open
to zzzPAT	another session is open	zzzPAT session on the PC. If you cannot
	under a different user	verify, restart PC
	User is not defined in	Define user by zzzPAT administrator
	zzzPAT	
Send report by	Microsoft Outlook was	Define Microsoft Outlook (or Outlook
email fails	not defined as the	Express) as the default mail client
	default mail client.	
Changes to Events	Events names are saved	Run the analysis by selecting
Names do not show	with the saved analysis.	Analyze>Reload Study and Analyze
up on screen	Changes will become	
	visible only after running	
	analysis again	
Errors while printing	Non-compatible printer	A postscript printer driver provides the
	driver	most reliable operation with the zzzPAT.
		Install a suitable postscript driver for the
		printer in use and try again

After elicking New	Inadvertent double	Droce Alt Tab to bring the diplog box
After clicking New		Press Alt-Tab to bring the dialog box
Study in zzzPAT	clicking the New Patient	back to the front
the dialog box	button may cause the	
disappears and	dialog box to be hidden	
zzzPAT is frozen	in the background	
"Enable multi-night	The WP has S/W	Upgrade the WP S/W to the newest S/W
option" does not	version lower than	(call Itamar Help Desk for the upgrade)
show in New Study	2.2182	
dialog		
"Enable tamper-	The WP has S/W	Upgrade the WP S/W to the newest S/W
proof testing option"	version lower than	(call Itamar Help Desk for the upgrade)
does not show in	2.2182	
New Study dialog		
No data of snoring	SBP/RESBP sensor was	Make sure to connect SBP to WP.
and body position	not connected to WP	Make sure to connect RESBP to
presented		WP200U or WP300 only
	RESBP sensor was	Make sure to connect RESBP to
	connected to WP200	WP200U or WP300 only
	instead of	, ,
	WP200U/WP300	
	Embedded SW not	RESBP for WP200U: Make sure to use
	compatible	embedded SW 3.3228 and above
No data of Resp	RESBP sensor was not	Make sure to connect RESBP to
movement	connected to WP200U	WP200U ot WP300 only
presented	or WP300	
Invalid Oximetry	Signal inadequacy due	If the invalid oximetry reading is repeated
reading (oximeter	to Sensor fault / reading	even at rest conditions then change
values of 127%)	during motion / poor	uPAT probe. If still invalid change uPAT
	pulsatile signal strength	cable or contact Itamar Medical customer
		support.

Table 3 - Troubleshooting, zzzPAT

Shared Access Mode zzzPAT		
Trouble	Possible Cause	Solution
User cannot log on to zzzPAT	In Shared Access mode user may be defined in the shared database and not in the local one, or vice-versa	Define user in the second database, or, Exit zzzPAT and log on to the other zzzPAT database (either local or shared)
Cannot find saved file	File saved to the other database (either the local or shared database)	 Verify to which database zzzPAT is connected (the database connection appears in the zzzPAT status bar) Exit zzzPAT Start zzzPAT and select the other database to connect to Select File>Open and search for the desired file
Shared database is not available	Network is disconnected	 Make sure the zzzPAT station is properly connected to the network, and that network services are available to it. Consult your system administrator if necessary
Cannot open selected study	Study is in use by another zzzPAT user	 Wait until the other user closes the study and try again

Table 4 - Troubleshooting, Shared Access Mode zzzPAT

Utilities		
Trouble	Possible Cause	Solution
Preparing for new study failed	The device was disconnected from the USB too soon.	Do not remove the device before Figure 37 dialog box appears
WP200(U) only: New Study or zzzPAT do not recognize the WP200(U)	The volume name has been erased	Format drive with volume name "WP200"
WP300 only: New Study or zzzPAT do not recognize the WP300	The FTDI drivers were not properly installed	Make sure the FT4222H Interfaces A, B, C, D appear on the "Universal Serial Bus controllers" section of Device Manager when a WP300 is connected to USB. In case these interfaces do not appear please contact Itamar Medical customer support
Database Tools button in the 'Database Wizard' window or Tools in zzzPAT is disabled	User does not have permission to operate this utility	zzzPAT Administrator can modify user's Extended Permissions (See section 2.6.2)
User Administration button in the 'Database Wizard' window or Tools>User Administration is disabled	User does not have permission to operate this utility	zzzPAT Administrator can modify user's Extended Permissions (See section 2.6.2)
Database Tools does not open	zzzPAT or New Study is running	Close zzzPAT or New Study and open Database tools
Super User forgot his password	-	Contact Itamar Medical Representative

Table 5 - Troubleshooting, Utilities

Appendix A: LICENSE AGREEMENT

This License Agreement represents the complete and exclusive understanding between you and Itamar Medical. The document can be viewed at https://www.itamar-medical.com/lmages/licensewp.pdf

Should you have any questions concerning this License Agreement, or if you desire to contact Itamar Medical for any reason, please write to:

USA: Itamar Medical Inc. 3290 Cumberland Club Drive, Suite 100 Atlanta, Georgia 30339, USA Tel: 1 888 748 2627

Worldwide: Itamar Medical Ltd. 9 Halamish Street, PO 3579 Caesarea 3088900, Israel Tel: +972 4 617 7000

Appendix B: TECHNICAL SUPPLEMENT

The zzzPAT uses a set of algorithms and provides automatically the following indices and events:

- Sleep-wake events using the Automatic Sleep-Wake Algorithm (ASWA)
- Oximetry algorithms to calculate saturation level (WP200U/WP300 only).
- Respiratory events index which includes Apnea Hypopnea and RERA: pRDI (PAT Respiratory Disorders Index).
- Apnea and Hypopnea Index: pAHI (PAT Apnea and Hypopnea Index).
- Central Apnea and Hypopnea Index: pAHIc (PAT Central Apnea and Hypopnea Index).
- Cheyne-Stokes Respiration detection algorithm to calculate %CSR during sleep
- Oxygen Desaturation Index ODI.
- REM (REM) events using the Automatic REM Detection Algorithm (ARDA).
- Deep and Light Sleep (s1,s2 is light) and s3-s4 is Deep sleep.

Sleep-Wake

The sleep-wake output, obtained in 30 seconds epochs, is used by the other three algorithms to apply calculations in sleep sections only, while skipping over the wake sections.

pRDI and pAHI

pRDI expresses the number of PAT respiratory events per hour of sleep, the index includes the following events: Apnea and Hypopnea and RERA (respiratory effort related arousal). **pAHI** expresses the number of Apnea and Hypopnea per hour of sleep.

These events are derived from the following physiological parameters measured by the WatchPAT

- PAT signal amplitude acquired by a pneumo-optical finger probe that measures the vasomotor changes of the arterial blood vessels in the finger. This reflects changes in sympathetic activity.
- Pulse rate derived from the above PAT signal.
- Blood Oxygen saturation level determined by an embedded pulse Oximeter.

The first two parameters are associated with sympathetic activity related to respiratory episodes. The third parameter, oxygen saturation level decreases (desaturation) during a respiratory event. Actigraphy movement is often associated with respiratory episodes. These four physiological parameters are incorporated into two different decision- making processes that define, for each epoch identified as a sleep epoch and breathing disorders. These processes are described in the attached flow diagram.

pAHIc and CSR%

pAHIc expresses the number of Central Apnea and Hypopnea events per hour of sleep. These events are classified as central out of all apnea/hypopnea events.

• These events are identified based on the RESBP sensor, snoring, oximetry, PAT waveform and using actigraphy analysis.

%CSR expresses the relative time in which periodic breathing was detected based on RESBP sensor, oximetry, PAT waveform and using actigraphy analysis.

Note: The calculation of pAHIc and %CSR is subject to regulatory approval in the country

ODI

This index expresses the number of Oxygen desaturation events during an hour of sleep. Desaturation event is determined when there is a reduction of 4% or 3% (based on user configuration) of the oxygen saturation baseline. The index includes the events that occurred during sleep time, and it does not includes events occurred during wake periods.

REM

REM events are determined for sleep epochs only, based on information extracted from local windows applied to the amplitude and pulse-rate time-series of the PAT signal. For each epoch four parameters are extracted:


- Mean PAT amplitude time-series
- Scaling-exponent of the amplitude time-series using Detrended Fluctuation Analysis (DFA).
- Ratio of peak low-frequency-band to high-frequency-band in the PAT amplitude timeseries spectrum.
- Ratio of peak low-frequency-band to high-frequency-band in the PAT pulse-rate timeseries spectrum.


These algorithms were optimized in clinical studies using simultaneous study of Watch-PAT with automated zzzPAT analysis, and in-lab standard polysomnography (PSG) recordings, which were scored manually according to the American Academy of Sleep Medicine (AASM) criteria. This set of sleep studies was defined, according to correct practice, as a training set. Once finalized, a separate set of studies was used to validate the algorithms.


DEEP and Light sleep

Epoch of Deep and light sleep are identified using the very same transformation of PAT amplitude and Pulse rate than for REM. This Provides the full PAT Hypnogram.

ZZZPAT analysis flow diagram

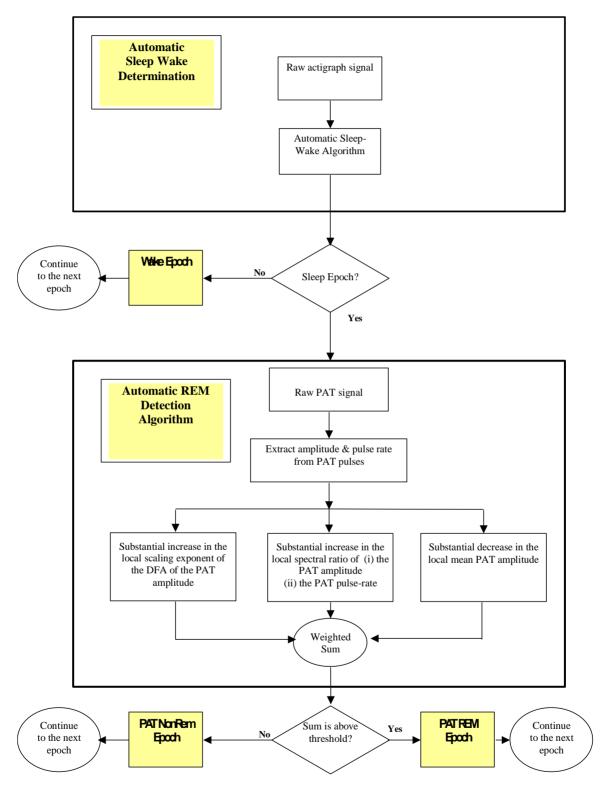

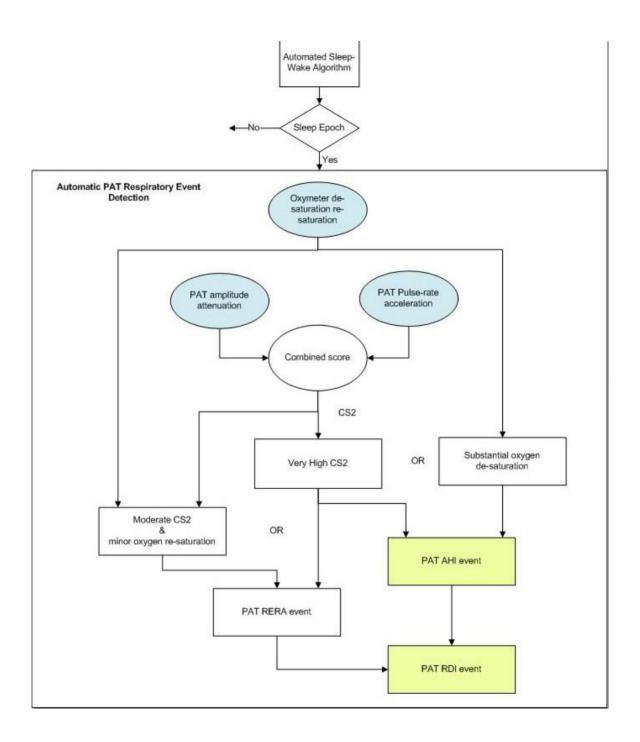

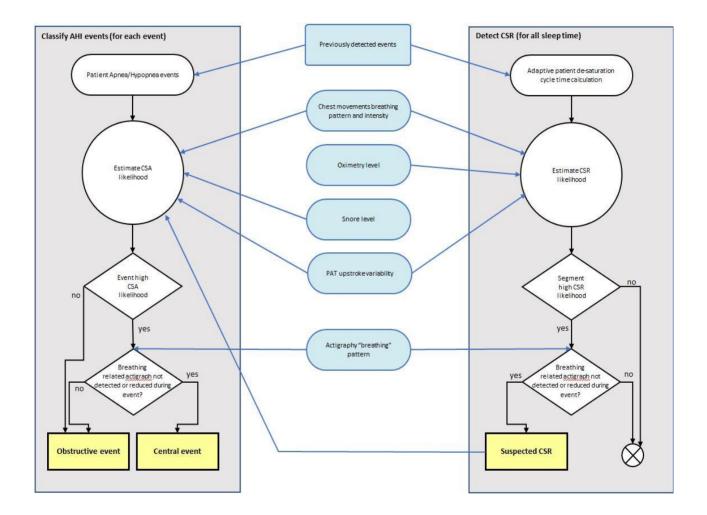

Appendix C: Keyboard Shortcuts

General:

In all dialogs or menus:

Press Alt +X (X is the character in the word that has underscore)

Special Keys:

Special Keys:			
F1	– Help		
Ctrl+N			
Ctrl+O	– Open study		
Ctrl+F4		se study	
Ctrl+L	– Loa	d from flash and analyze	
Ctrl+P	– Print		
Ctrl+C	– Сор	,	
Ctrl+F			
Shft+F		o window all signals	
Shift+O		rt an A/H obstructive event on the highlighted section of a signal	
Shift+C		rt an A/H central event on the highlighted section of a signal	
Shift+U	– Inse	rt an A/H unclassified event tag on the highlighted section of a signal	
I O	– Zoom in – Zoom out		
	arrow e	 Previous Event Next Event First Event for channel Last Event for channel Sleep Report Sleep Indices 	
Alt+Plus Alt+Minus Alt+ Right Alt+ Left a			
Home End Page Dow Page Up Left arrow Right arrow Delete		 Go to First Page Go to Last Page Go to Next Page Go to Previous Page Scroll left Scroll right Delete selected event 	

Appendix D: Regulatory representative

Itamar Medical's authorized regulatory representative:



The Squaire 12, Am Flughafen, 60549 Frankfurt am Main, Germany

Appendix E: Index

A

Analyze>Reload study and analyze · 82 Archive · 115, 116, 117, 119, 120, 121

B

Backup · 120, 126, 127

С

Compact Flash Defining drive · 21

D

Database Tools · 116, 125, 126, 133 Database Wizard · 20, 21, 52, 125, 126, 127, 129, 133

E

Edit>Copy · 70 Error! Hyperlink reference not valid.>Add Event · 83 Event Management · 82 Events Deleting · 85 Events>GoTo Event · 86 Events>Select Event · 86, 87 Export · 26, 36, 109, 115, 116, 117, 118, 119, 120, 121, 126 Export a report · 88 Export From Database · 126

F

File>Close Study · 69 File>Exit · 69 File>Export Data · 109 File>Export Events · 109 File>Load Study and Analyze · 66 File>New Study Details · 53, 57, 66 File>Open Study · 67 File>Print · 108

Η

Hardware Requirements · 13, 14 Help Prepare and Send Study to Itamar Medical · 110

Ι

Import to Database · 118, 126

0

ODI · 90, 91, 98, 101, 136

P

pAHI • 90, 91, 98, 100 pRDI • 90, 91, 98, 101, 135 pREM • 92, 135, 136 Printing • 108 Print a report • 88

R

Report · 88 Clinical Diagnosis · 89 Event Report · 98, 99, 100 Patient Follow-up Report · 101 Restore · 120, 121, 127

S

Setup>Directories · 27 Setup>Settings · 27, 33 Colors · 29 Events · 35 History · 35 Montage · 28 Report Appearance · 43, 45 Report Translation · 48 Status Bar · 76, 77 Study Adding clinical information · 61 Adding Demographic information · 60

T

Transferring a Study to Itamar Medical · 109

U

User Add User · 24 Deactivate · 26 User Administration · 23, 26, 125, 133

V

View>All Night Channel Fit to Window · 80 View>Activate Channel Fit to Window · 80 View>All Night Window · 71, 81 View>Channels · 71, 79 View>Fit To Window Mode · 81 View>Grid On/Off · 81 View>Grid On/Off · 81 View>Relative Time · 81 View>Study Details · 70 View>Study Details · 70 View>Time Base · 80 View>Zoom In · 81 View>Zoom Original · 81 View>Zoom Out · 81

Ζ

zzzPAT Installation · 14 Upgrading · 20 Using · 52